Table of Contents

- General warning and cautions.....1
- General information, indication for use.....2
- Specification, intended.....3
- Device description4
- Environmental Conditions that Affect Use5
- Installation.....6
- Operating Instructions.....7
- Storage.....10
- Troubleshooting.....11
- Cleaning12
- Check out.....13
- Symbols14
- Dimension and data.....15
- Summary.....16
- Index.....17

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WARNING AND CAUTIONS:

Warnings mentioned in this directory are for recognizing possible risks in the conditions of using the bed inaccurately; that may cause patient death, dama

Cautions mentioned in this directory are for recognizing possible risks in the condition of using the bed inaccurately; that may cause to damage the bed or make it out of order.
ge or other possible risks.

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WARNINGS

Do not use subsidiary equipments which are not designed for bed.

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general information

general information

- GROUP SINA as emergency stretcher with the various positional specialty of the patient and other specifications are an assistant for diagnosing and treating diseases.
- these beds have been built in accordance with the applicable national and international standards and the regulations reflecting the current state of technology . These beds are tested in accordance with international standards and bear the CE mark, which documents the beds' compliance with essential requirements for medical products.
- Please read the General Safety notes with particular attention to any warnings and cautions .
- We accept no liability for damage and operational faults caused as a result of misuse .

Intended Use

- Hospital, Healthcare Centers, General medicine, general and specialty surgery, adult cares,

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specification/ intended use

Clinical application

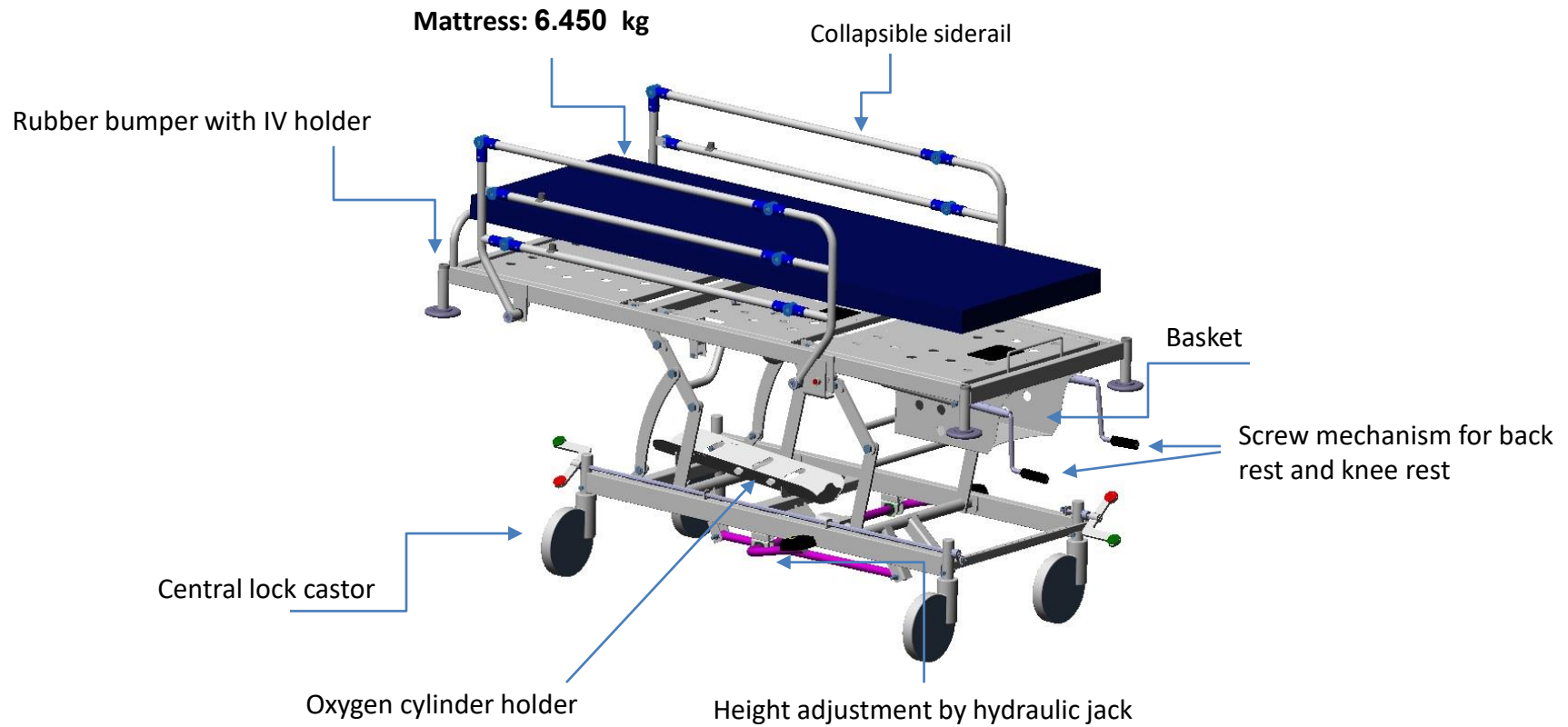
S4100.1 as stretcher with the ability of adjusting Hi/ Low, backrest, kneerest and other features is an assistant for diagnosing and treating diseases.

• **General specification**

- Mattress platform is three section, two of them adjustable with airflow capability under the mattress
- Detachable, antistatic mattress with foam of 70(mm) thickness and artificial washable leather cover
- Mattress platform covered with metal sheet
- Backrest and knee raise adjustments are manually with two freewheeling and foldable screw mechanisms
- Height adjustment is available by hydraulic jack system
- Collapsible siderails
- Washable and disinfectable
- Four castors with central/directional brakes
- Four rubber bumpers at corners
- IV stand holder at each corner
- Electrostatic powder coating
- Fixed pushing handle at head end
- Security belt for protecting patient during transport
- Basket for patient personal goods
- DIMENSIONS and BASIC INFORMATION
- oxygen cylinder holder positioned on the base with fixing straps
- Overall 2024 x 850 (mm)
- Mattress platform 1810 x 630 (mm)
- Hi - Low 550 - 800 (mm) without mattress
- Backrest angle 0° - 65°
- Knee raise angle 0° - 15°
- Castors 200 (mm) diameter
- Stretcher net weight: 115 kg
- Safe working load capacity 170 kg
- Max. load capacity 250 kg

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environmental conditions

➤ Security

- When using the bed, be sure that there is no obstacle such as furniture near the bed.
- Before using bed be sure that patient is placed on bed correctly to prevent unbalancing.
- In fix state, always apply brakes .
- When patient is without care giver, keep the height of bed in Min. height.
- When patient is without care giver, all functions of bed should be locked through ACP by a qualified person.
- The mentioned guidelines should be available to refer.
- Atria, general hospital bed is delivered fully assembled. If needed, the expert company staff- after sale services staff- assembles the bed on site.
- The completeness should be checked against delivery documents

 Maximum safe working load for the bed is 250 kg.

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➤ Installation and Electrical Activation

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When a product is purchased and sent to customer, it will be installed and used. For Sina stretchers series , trained operators of *after sale service department* are responsible for setting up these beds.

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Mention the following while installing the bed:

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- I. Check the bed for any breakage may caused while transporting. Be sure that bumpers are not missing .
- II. First install detachable parts and accessories.

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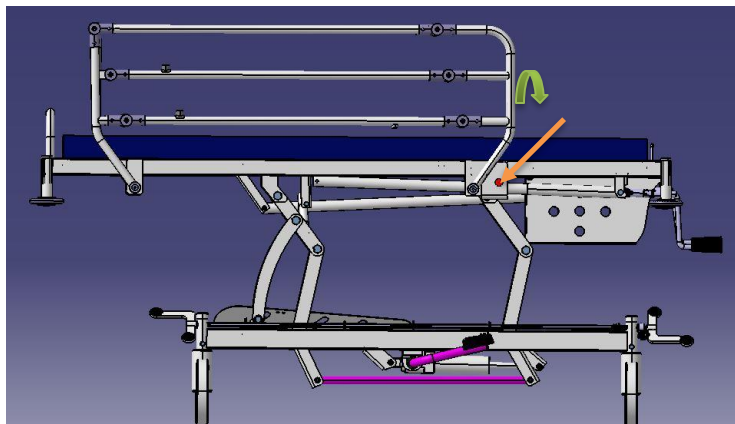
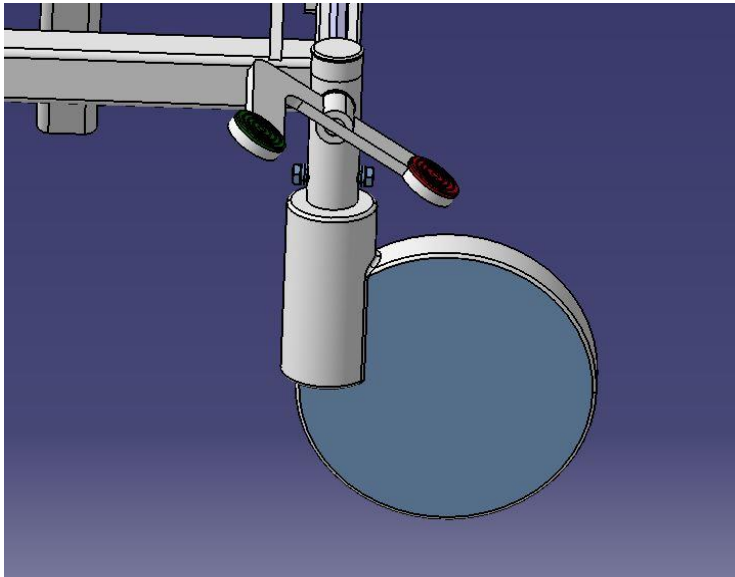
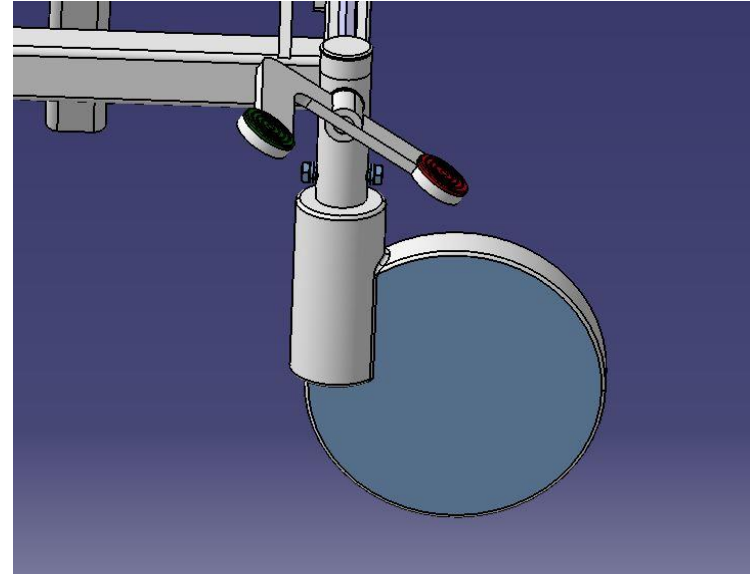
➤ Castors

S4100.2 castors are central locking. On both wheels, a dual plastic pedal is used to determine the direction of movement and the fixed position of the bed. The three wheel modes are as follows:

Lock mode: When the pedal is pulled down, the two wheels are locked diagonally and the bed does not move.


Free mode: When the pedal is parallel to the ground. In this case, moving in all directions is possible.

Direction mode: When the pedal is raised, it locks one of the wheels on the head and prevents it from turning. In this case the motion of the bed is on a straight line.


➤ Collapsible side rails

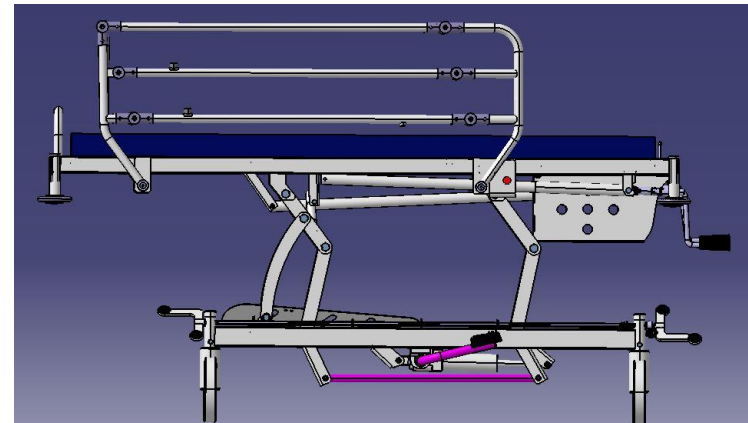The side rails are on both side of the stretcher. By pulling the side bar upwards, it will be at its highest level and will lock automatically. To lower the side, hold it with one hand and press the plastic button with the other hand to guide the side down.

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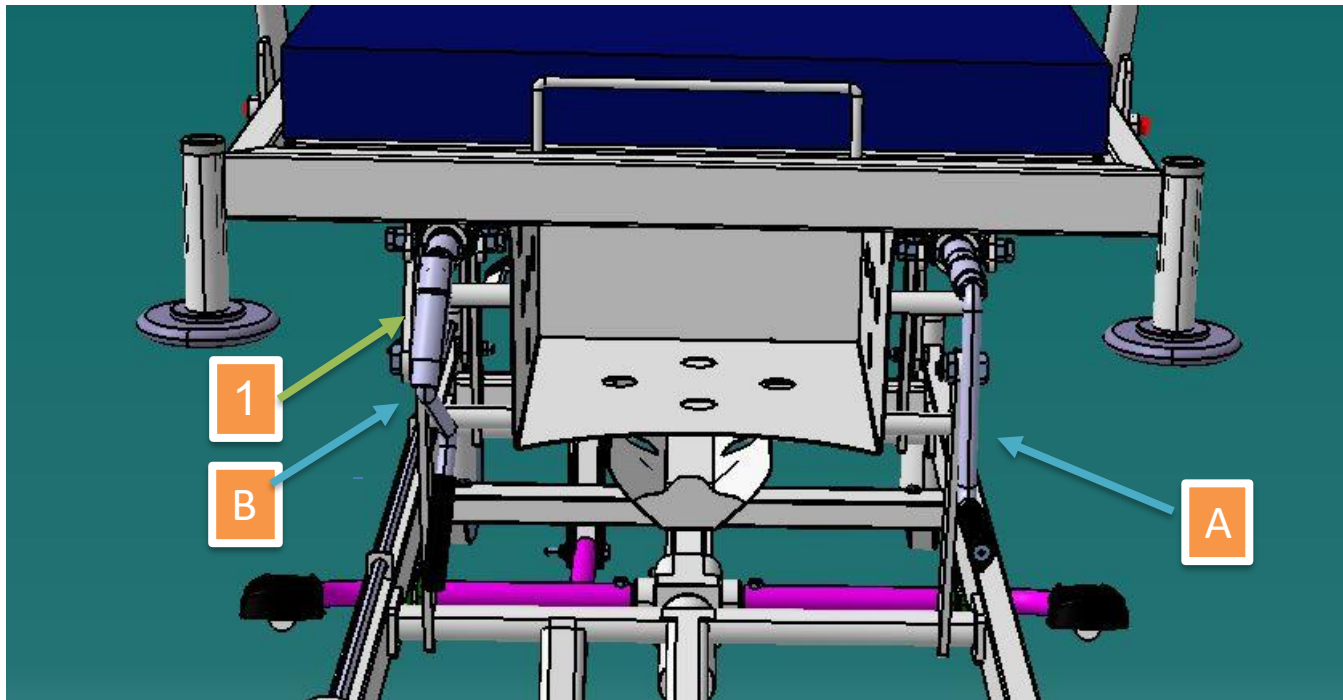
Operating instruction

➤ mechanical jack

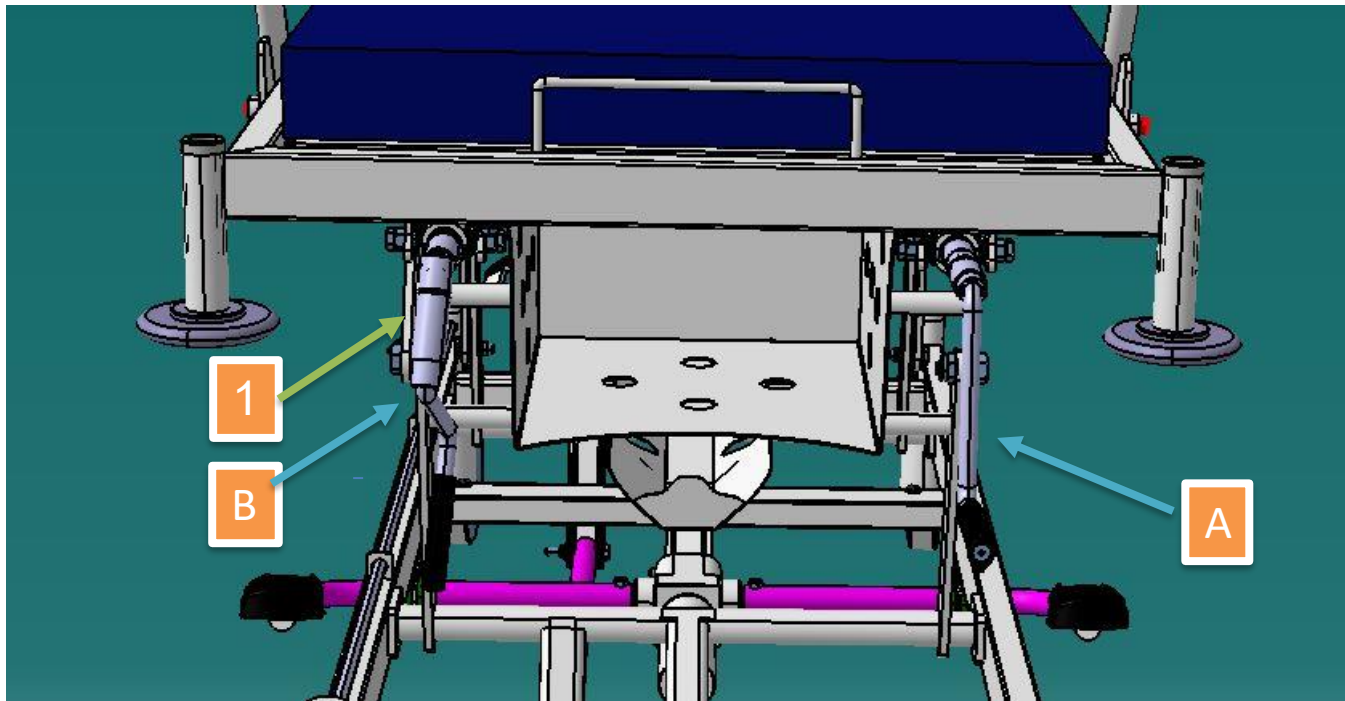
The S4100.2 stretcher is equipped with a mechanical jack for backrest and kneerest positions. To use the mechanism jack, simply turn the handle shown clockwise or counterclockwise. When using the mechanism jack, the mechanism handle should be in the A (open) position. After use, the handle can be folded and guided to the sub-stretcher. To move from position A to B, piece 1 must be pulled out and the hand guided under the bed.

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
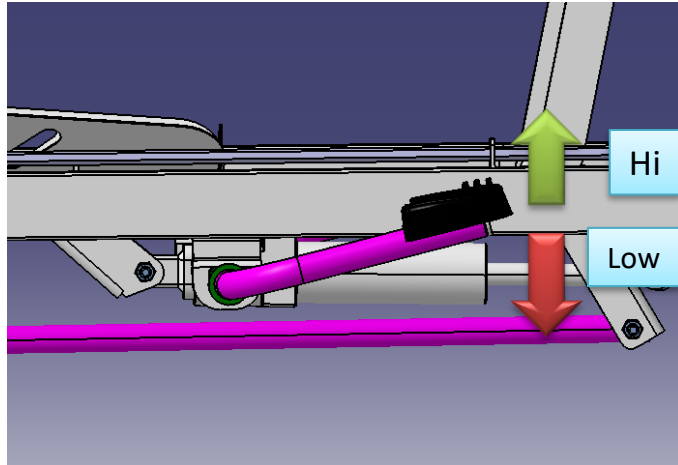
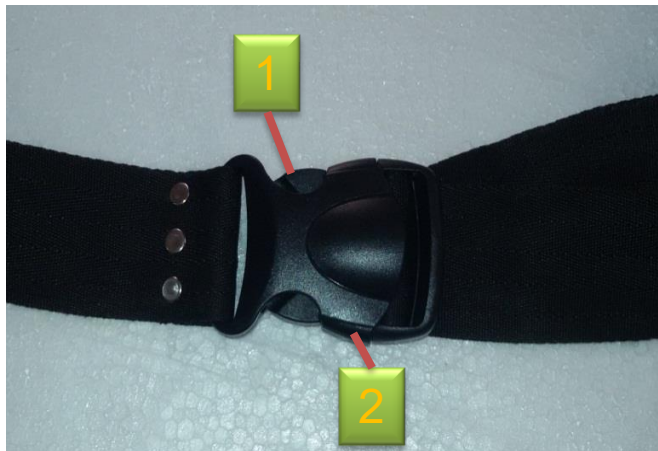
Operating instruction

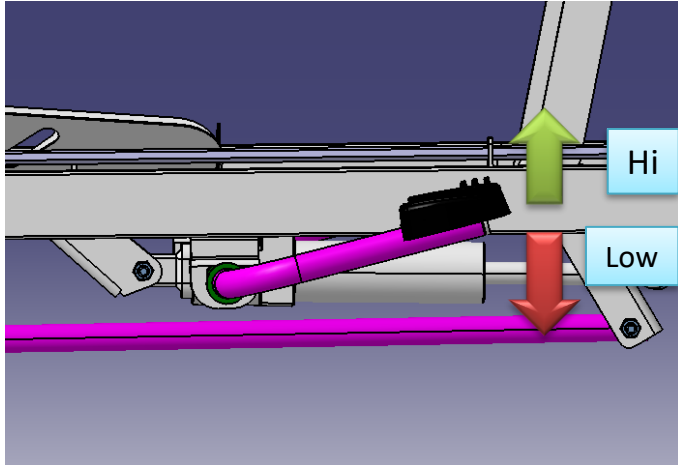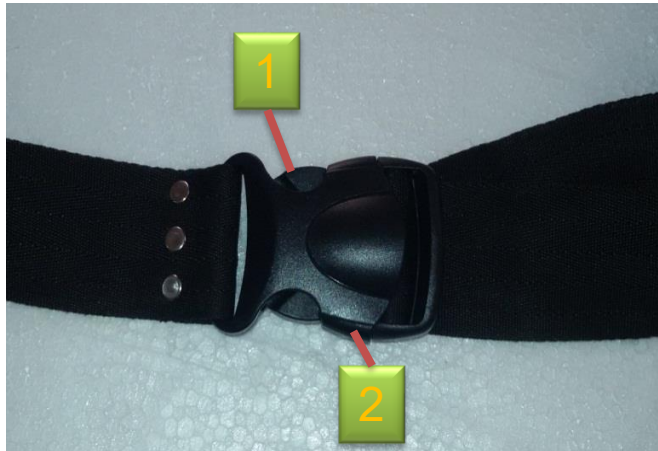
- Hydraulic jacks
- The S4100.2 stretcher is equipped with a hydraulic jack for Hi-low positioning.

- Safety belt
- To prevent the patient from slipping, the seat belt is installed on the stretcher. To use, we first push part number 1 inwards and push part number 2 backwards.

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
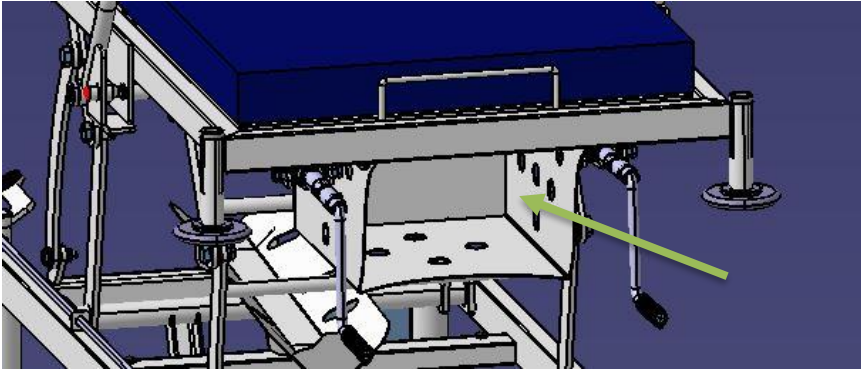
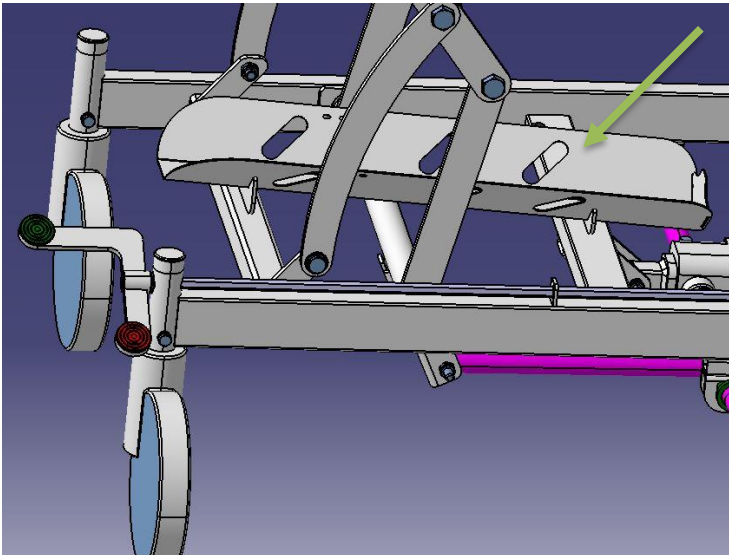
Operating instruction

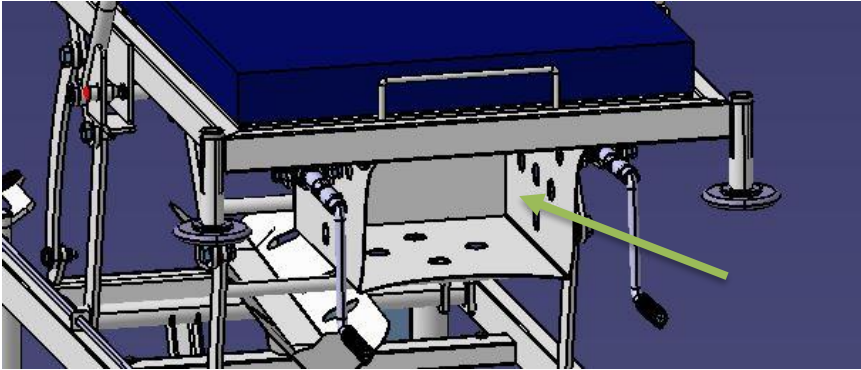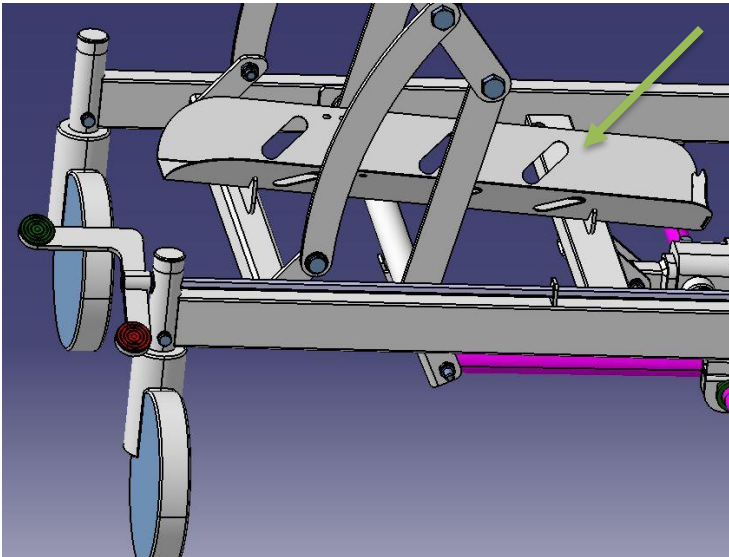- **Basket**
- The basket holder is designed to hold sick items on a stretcher.

- **Oxygen cylinder holder**
- To place, place the oxygen capsule on the sheet marked in the figure and fasten the belt around it, and do the opposite to remove the capsule.

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➤ **Note the following while Transporting the Bed**

Handle with care, do not drop. To protect the stretcher in the best manner, pack it like its original packing. The equipment should be stored in a clean, dry and ventilated area.

The following limits apply to operating conditions or longer periods of storage:

Ambient Temperature	from 10° C to 40° C
Relative Humidity	30% - 75%
Air Pressure	70kPa – 104kPa

Troubleshooting

- The table below suggests some simple checks and corrective actions. If these actions fail to solve the problem, consult an after-sales representative.

DEFECTION	REASON	DEBUGGING
The bed is not fixed in its place.	<ul style="list-style-type: none"> brakes are free brakes do not work 	<ul style="list-style-type: none"> Use the brakes of castors replace castors
Back rest angle does not change when it is in up position	<ul style="list-style-type: none"> the crank has been locked (because it has been extended extremely or because of exertion of strength). 	<ul style="list-style-type: none"> shake the handle several times and then turn it calmly.
Back rest angle does not change when it is in low position	<ul style="list-style-type: none"> worn ABS nut of crank 	<ul style="list-style-type: none"> Replace the crank
Folding side rails do not work.	<ul style="list-style-type: none"> loosen or damaged plastic button 	<ul style="list-style-type: none"> Replace plastic button
Manual crank handle does not shrink.	<ul style="list-style-type: none"> dislocated spring or damaged locker pipe. 	<ul style="list-style-type: none"> Replace crank handle.

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Cleaning and Disinfection

WARNINGS

- Before cleaning disconnect the bed from the mains power supply
- Gloves and protective clothing should always be worn when carrying out cleaning procedures.

These instructions also apply to accessories but not to mattresses.

Wipe all surfaces with a soft cloth moistened with hot water and a mild detergent (or the manufacturer's recommended cleaning solution). Take extra care with harbor dirt or dust.

Rinse with clean water and dry with paper towels.

Allow the cleaned parts to dry before replacing the mattress.

Caution

- Do not use cleaning solutions such as alcohol, gasoline or substances including benzene rings.

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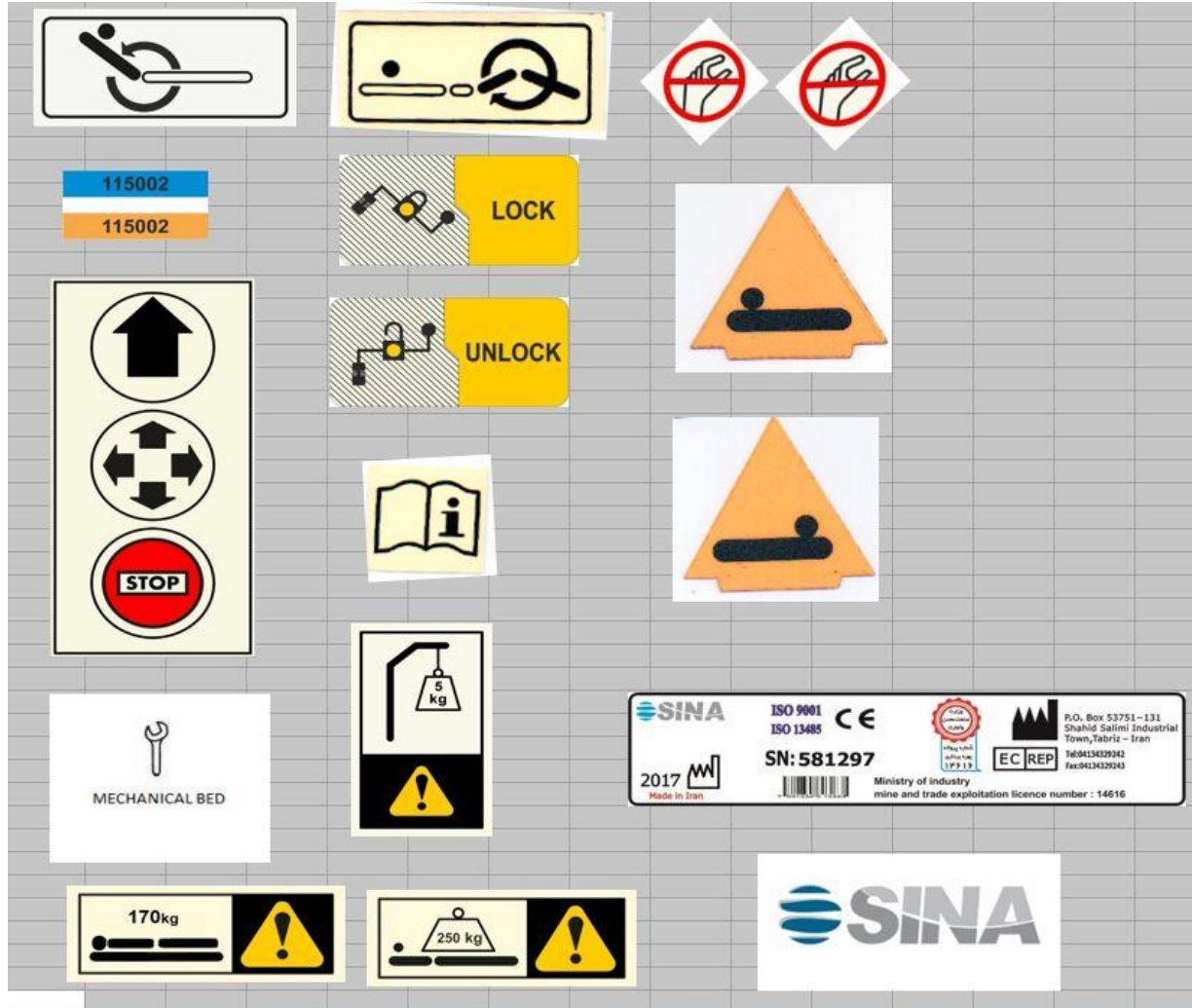
Check-out

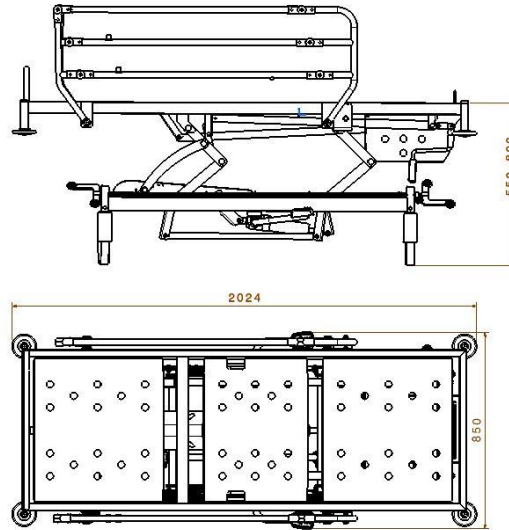
Check that the bed does not have any deflection. Make sure that all nuts and screws or other bolts are fastened and not missing.

- Always use the mattress with suitable size and type.
- When using the bed, be sure that there is no obstacle such as furniture near the bed.
- Before using bed, be sure that patient is placed on the bed correctly to prevent unbalancing.
- In fix state, always apply brakes.
- Using security sides should be under the operator consideration.
- Keep these instructions handy for reference.
- Use apply push handle for transfer and moving the stretcher.

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Summary

- Sina series stretchers are produced in several models.
- Backrest and knee raise adjustments are manually with two freewheeling and foldable screw mechanisms
- Height adjustment is available by hydraulic jack system
- For adjusting bed positions roll the crank in clockwise direction.
- Folding side rails are in both sides of the bed. On each side, prepared a plastic button for guiding the bed downward.
- Stretchers can be equipped with different accessories including IV pole, urinary bag, chest bottle ... which their place can be customized by the customer need.
- Do not use subsidiary equipments which are not designed for bed.
- we accept no liability for damage and operational faults caused as a result of misuse or non-observance of these instructions for use.

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Index

Accessories	19
Castors	15
Cleaning	25
General specification	7
Graphical design	31
Intended use	7
Installation	13
mass of the bed	31
Side rails	15
Standards	11
Storage	21
Summary	33
Symbols	29
Table of troubleshooting	23
Technical dimension and data	30
Workload	11

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