Table of Contents

☐	Warnings and cautions	1
☐	Devise Description	2
☐	Technical Specification	3
☐	Installation Instruction	4
☐	Maintenance	5
☐	Storage	6
☐	Troubleshooting	7
☐	Failure	8
☐	Symbols	14
☐	Block Diagram	15

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WARNING S AND CAUTIONS:

- **Warnings** given in this manual identify possible hazards in procedures or conditions which , if not correctly followed could result in death, injury or other serious adverse reactions.
- **Cautions** given in this manual identify possible hazards in procedures or conditions which, if not correctly followed could result in equipment damage or failure.

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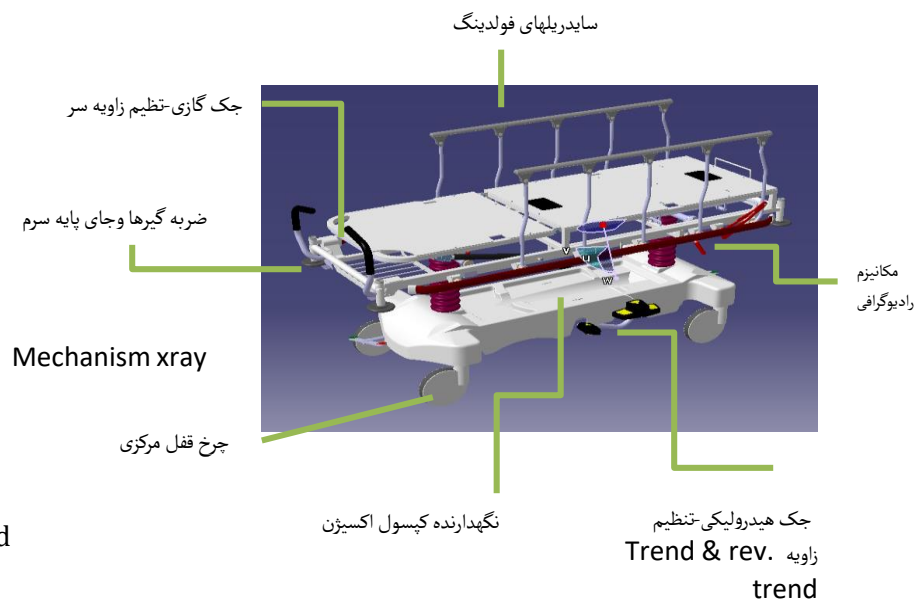
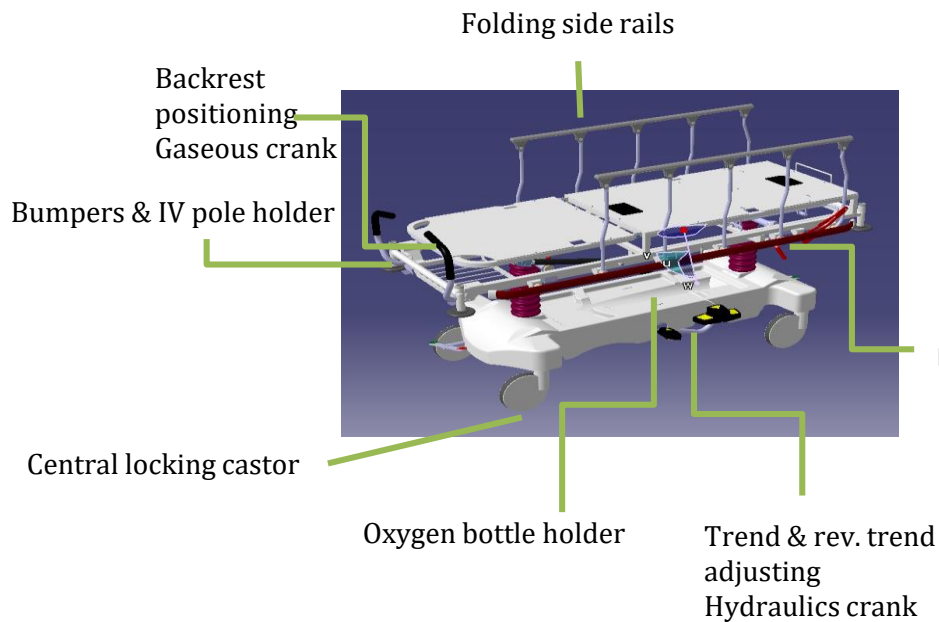
Devise Description

GROUP stra4243as emergency , transfer and ward stretchers are an assistant for diagnosing and treating diseases used in hospitals, care centers and adult cares. Stra 4243is emergency stretcher with the ability of adjusting backrest, trend & rev. trend

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Technical Specification

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- Mattress platform is two – section, one crack.
 - High-low trend / reverse trend adjustment is possible by hydraulic jack.
 - Folding siderails.
 - Backrest adjustment by gas spring.
 - Washable and disinfect-able mattress.
 - Four castors with central/directional brakes
 - With IV stand in four corners of the bed.
 - With bumpers in four corners of the bed.
 - With protection guard for side rails.
 - Oxygen cylinder holder on cover of the bed.
 - Detachable mattress
 - Radiographic capability
 - backrest xray casset holder
 - Has an insoles hpl
- Dimensions and Basic information:
- Overall: 2000×754×980 (mm)
 - Mattress platform: 187×64cm
 - Backrest angle: 0-90
 - Hi-lo: 66-98 cm
 - Castors: 200 mm diameter central/directional brakes
 - Max. Load: 250 Kg

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Installation Instruction

Installation of stra4243 stretchers series are done by Sina Hamd Aria operators. These beds are delivered to the users and customers as fully assembled .

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Maintenance

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Any service or repair activity except the following ones must only be performed by a qualified and trained person approved by Sina Hamd Aria. Product recovery and its components should be complaint by local regulations. Check that the bed does not have any defection. Make sure that all nuts and screws or other bolts are fastened and not missing.

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No.	Control Steps	Period	Manner of Control
1	Side rails	biannual	Open and close side rails several times to be sure of its hinges and jacks operation correctly.
2	Jacks, hydraulic and gas	Annually	Jokes under Barqradadh In this case you should not have to reduce the height and angle.
3	Castors and Brakes	Annually	Use the castors' brake and force the bed forward. The bed should not move.
4	Mattress	6-month	Check the whole part of the mattress . Make sure mattress is not torn.
5	x-ray Mechanism	6-month	Open and close several times and check the linear motion along the bed and ensure the spring and joints performance.

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Storage

Avoid shock during transporting the bed. Do not drop it when unloading.. To protect the bed in the best manner, pack it like its original packing.

The equipment should be stored in a clean, dry and ventilated area.

The following limits apply to operating conditions or longer periods of storage:

Ambient Temperature	from 10° C to 40°
Relative Humidity	30% - 75%
Air Pressure	70kPa – 104kPa

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Troubleshooting

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The following table suggests some simple checks and corrective actions.
If these actions fail to solve the problem, consult an after-sales representative.

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DEFECTION	REASON	DEBUGGING
The bed is not fixed in its place.	<ul style="list-style-type: none"> brakes are free brakes do not work 	<ul style="list-style-type: none"> Use the brakes of castors replace castors
Non-moving gas jack	<ul style="list-style-type: none"> Gas leak 	<ul style="list-style-type: none"> Replace the crank
Non-moving hydraulic jack	<ul style="list-style-type: none"> Oil leakage 	<ul style="list-style-type: none"> Replace the crank
Folding side rails do not work.	<ul style="list-style-type: none"> Joint damage plastic 	<ul style="list-style-type: none"> Plastic joint replacement

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Troubleshooting

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The following damages may occur in Stra4243 stretchers :

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- 1)Hydraulic jack failure
- 2)castor damage
- 3) Torn mattress
- 4) Dropping back and missing bumper
- 5) Side rail damage
- 6) Gas springs failure
- 7)Do not move the radiography mechanism
- 8)Do not lower the mechanism frame

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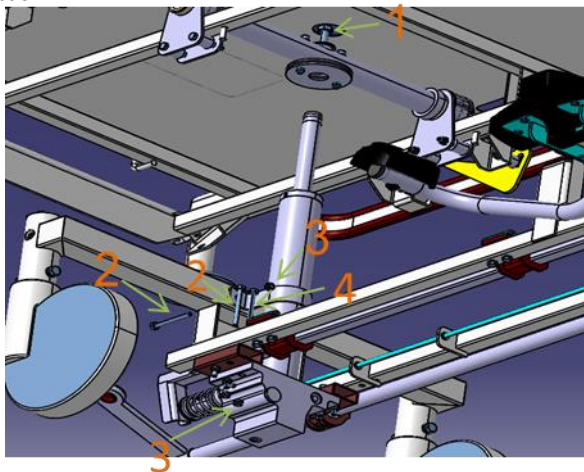
Troubleshooting

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
1) Hydraulic jack failure

To change the first part 1 (screw M10x25) and then open the A (frame) from the base removed and parts 2.3.4 Number (M6X60 screw nut M6 felt-screw M6X50) open and then we replace Jack .

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
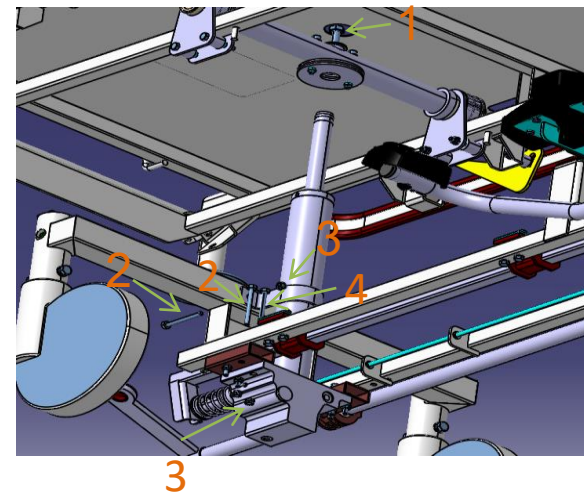
2) CASTORS DAMAGE

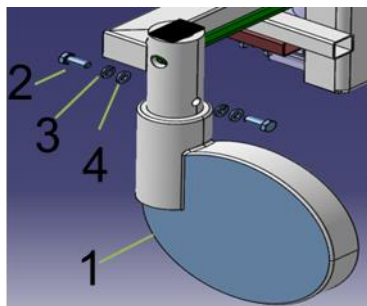The wheel assembly 1 piece wheel base of the tube stretcher through, piece by 2 (screw M8x20) and No. 3 (Spring washer M8) and (4) (Flat washer M8) after passing through the broken wheel close tightly.

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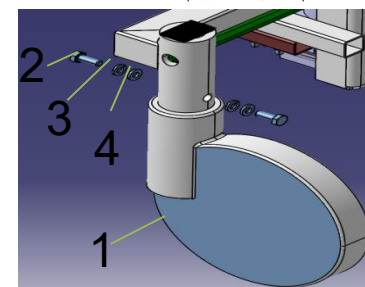Troubleshooting

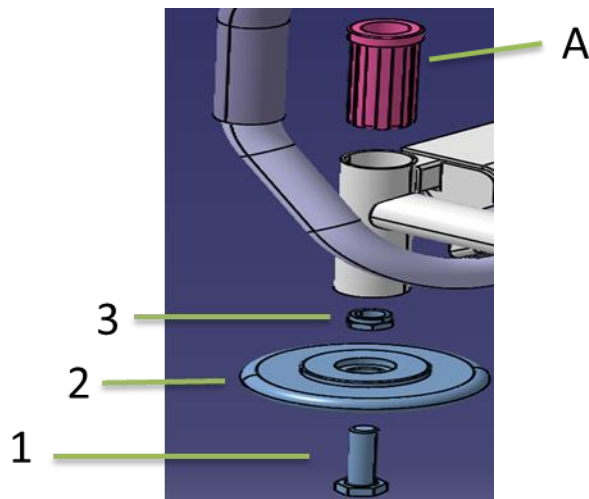
3) TEAR IN MATTRESS

Mattress stretcher includes a removable piece is:

At the time of rupture of the mattress, the mattress to remove damaged and healthy mattress to replace our old mattress.

4) BUMPER MISSING

•first plastic piece A's out Pass the screw M12 under the bumper and the nut over it. Enter the second nut into the tube under the stretcher. Fasten the set to its place. do not tight firmly that the bumper can rotate in it.

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

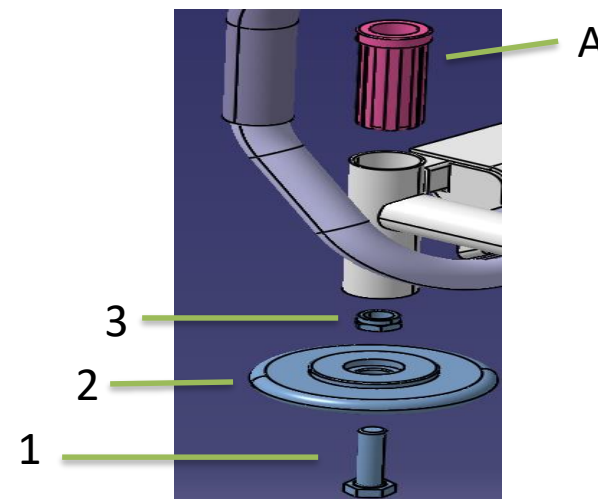
a) take the screw

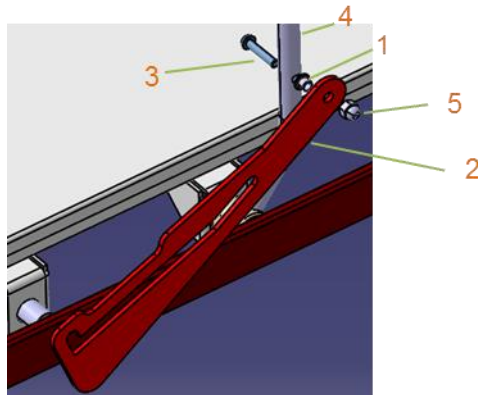To assemble the piece (1) (Bush) as shown below in Part 2 (lever steering), and then part 3 (screw) in Part 4, 1 and 2 from the hardware through the cap to close.

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
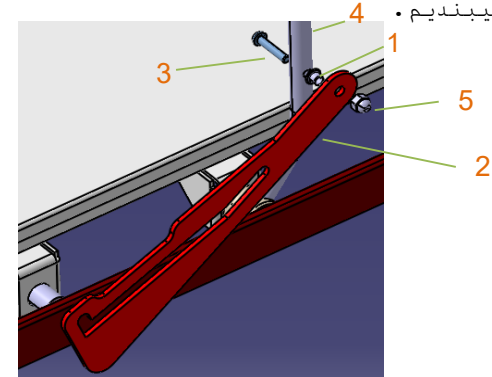
b) Joint breakage in folding sideboard

Two piece number 1 (pin plastic) of the piece 2 (plastic hinge), and 5 (tube) inserted then close fitting piece (3) (screw the lens dispenser) through the hole part 2 (plastic hinge) and cross pieces (4) to close.

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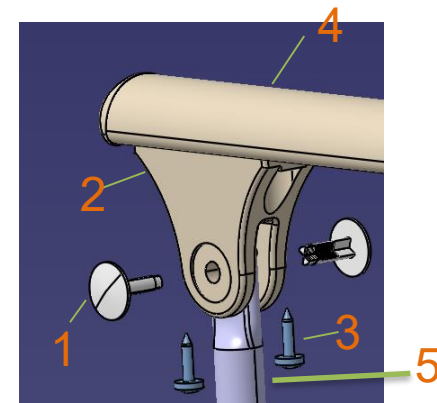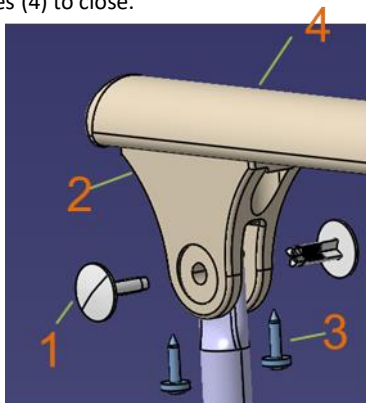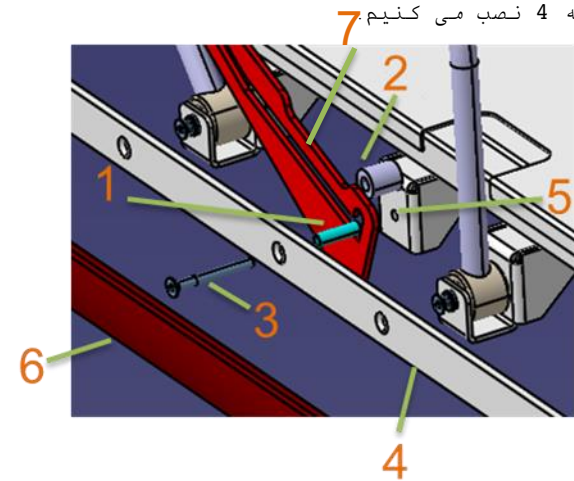
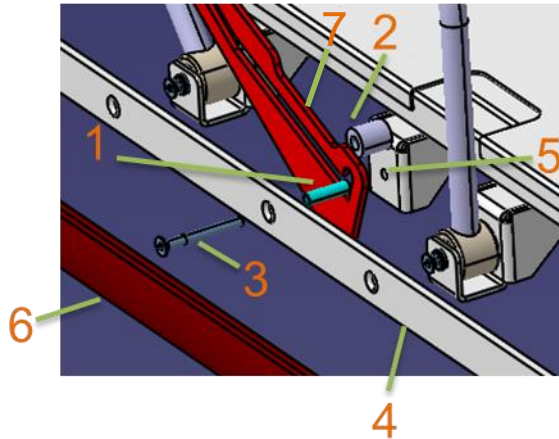
To assemble the piece (1) (Brass pipe) from the piece (2) (Metal Bush), and 7 (lever metallic lead) and then passed through the pieces (3) (a treasure of screws M6x60) hole section 4 (Belt Guard) enter to close the piece after piece (5) (6) in part 4 installed on it.

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7) Do not move the radiography mechanism:

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Polymer bush blade mechanism on the rails makes the radiography mechanism not move on the rails, therefore Polyamide rails and linings lubricate the mechanism.

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The breakdown of the spring causes the mechanism frame to not be lowered, in which case the spring changes the mechanism.

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9) How to open the mechanism for repair as follows

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To open the mechanism, first open the rails of figure 1 and then remove the coil.

Then, to open Procedure 2, open the mechanism of the rails and the screws.

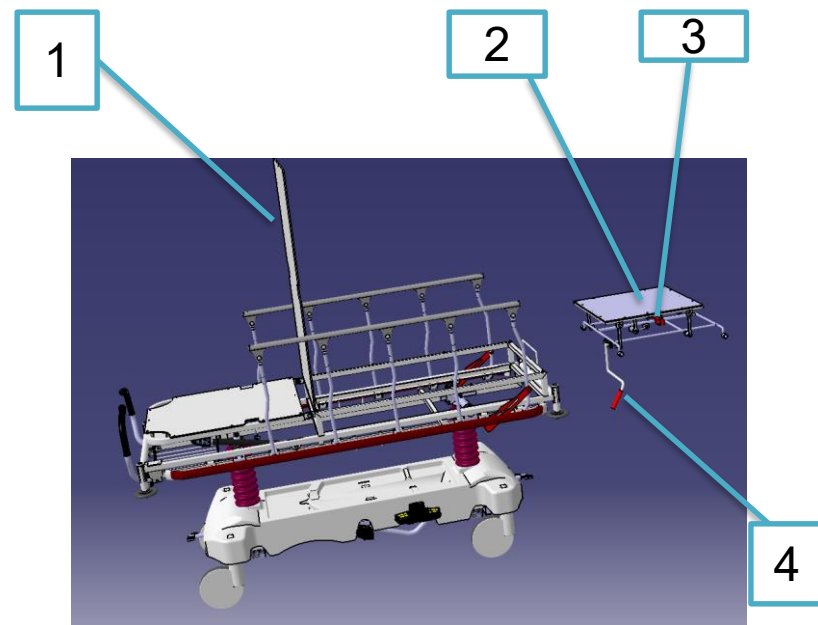
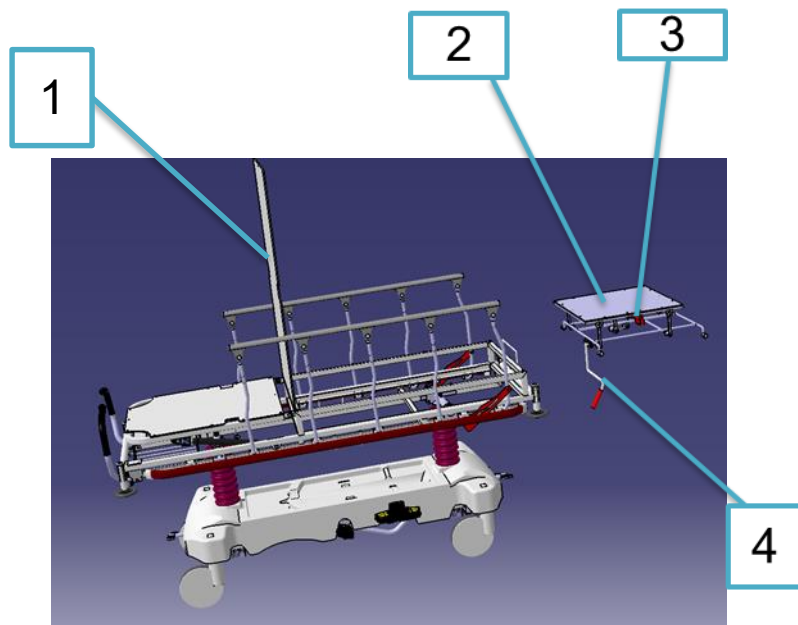
Which separates the procedure and separates the number 3.

Which is to bring the mechanism Open the mechanism through the knobs of the knobs on handle No. 4 and separate the handle and place the mechanism in

The ends of the rails we put on the collar, in the part that is above the rails. Mechanism

We raise it up and remove it from the coil.

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Symbols

Serial Number, manufacturer, company logo و Manufactured date

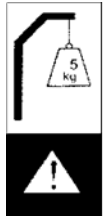
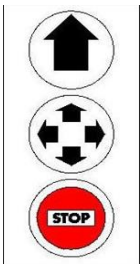
Max. safe working load for the bed is 250 kg

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