MOBIAK®

MEDICAL • GASES • FIREFIGHTING est. 1977

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. PRODUCTS

PATIENT LIFTS

0804935 PATIENT LIFT MBK ELECTRIC ATLAS II (150kgr)

0804936 PATIENT LIFT MBK ELECTRIC TALOS II (200kgr)

0804937 STAND UP MBK (200 kg)

2. INTRODUCTION

Thank you for choosing a MOBIAK patient lift / transport patient lift. Their design and quality are a guarantee of comfort, safety and reliability. MOBIAK's patient lift / transport cranes are designed and built to meet all your requirements for practical and proper use. This user guide provides some suggestions on how to use the patient lift of your choice correctly and offers valuable safety tips. You should read the entire manual carefully before using the patient lift / transport patient lift. If you have any questions, you can contact him retailer for advice and support.

3. INTENDED USE

MOBIAK's patient lift / transport lifts are intended for lifting and transporting people with mobility difficulties. An assistant operates the machine. It is not possible to use patient lifts by one person only. MOBIAK disclaims any responsibility for damage caused by improper use of the device or by use other than that specified in this manual.

4. DECLARATION OF CONFORMITY

We declare our sole responsibility that the Medical Devices mentioned in this statement are of low Risk Class (Risk Class I) and meet the requirements of European Regulation 745/2017 and where applicable the mentioned standards and legislation.

BEFORE USE

The patient lifts are suitable for transporting people with mobility problems over short distances, such as from the bed to the wheelchair or to a toilet seat and vice versa. It is not possible to use the patient lifts in an upright position to transport the patient over longer distances.

- Before lifting the patient, make sure the base is wide open, the opening pedal is locked in the open position and the rear wheel brakes are activated.
-
- When moving the patient, do not cross obstacles that could cause the hoisting patient lift to tip over.
- Always check that the sling is in good condition. Do not use slings with signs of wear or damage

Note: Before use, make sure that the emergency button is not pressed. If it is on, turn it off so that the control unit can be activated.

5. GENERAL WARNINGS

WARNING!

Always pay close attention when there are moving parts, which could cause limb entrapment and injury.

- For proper use of the device, refer to this manual, checking the pre-use warnings very carefully.
- Consult your doctor or therapist for proper use of the device.
- Keep the packaged product away from any heat source, as the packaging is made of cardboard.
- The useful life of the accessory is determined by the wear of the parts that cannot be repaired and / or replaced.
- Be careful when there are children in the area.

Do not exceed the maximum payload

- The user and / or patient must report any serious accident that has occurred in relation to the device to the manufacturer and to the competent authority of the country in which the user and / or patient resides.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Attention, should read the instructions

Production dates

Waste Electrical & Electronic Equipment (WEEE)

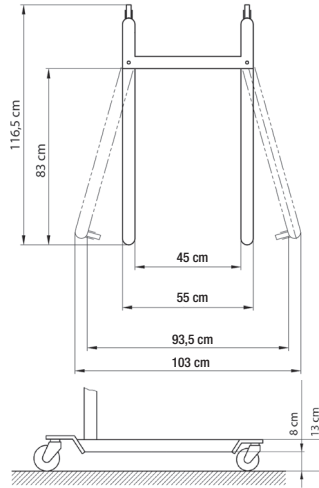
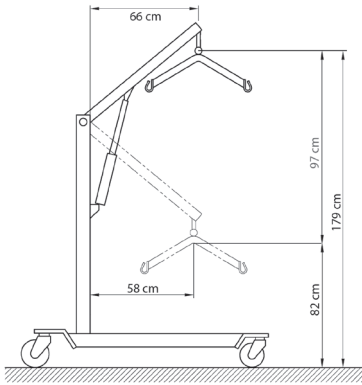
Protection against dust and water splashes

B Type



7. DIMENSIONS

7.1 Dimensions of product 0804935 PATIENT LIFT MBK ELECTRIC ATLAS II (150kg)


Patient lift with a total weight of 150 kg with the electric kit: 35 Kg

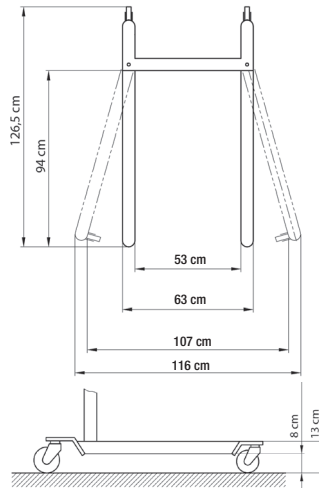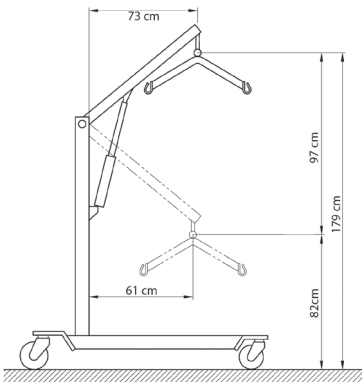7.2 Dimensions of product 0804936 PATIENT LIFT MBK ELECTRIC TALOS II (200kg)

Patient lift with a total weight of 200 kg with the electric kit: 36 Kg

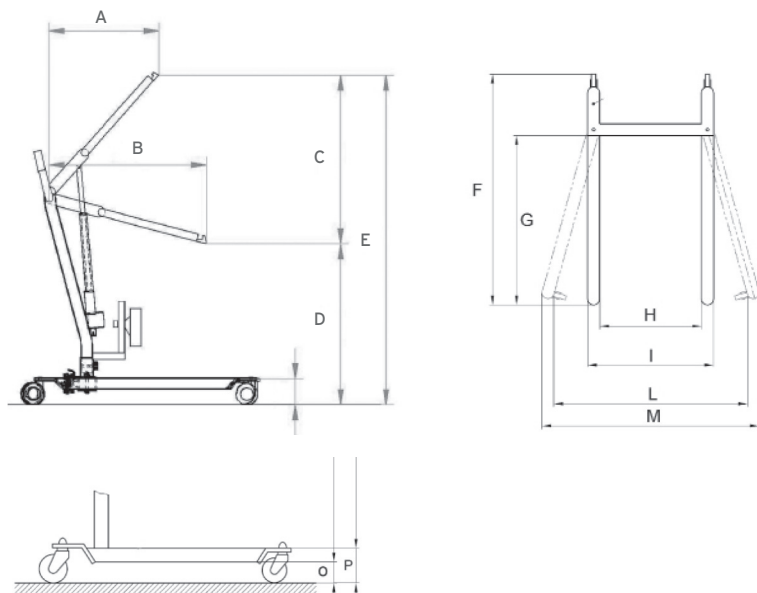
7.3 Dimensions of product 0804937 STAND UP MBK (200 kg)


- A Sagittal dimension at max.verticalization: 2 cm
- B Arm lenght: 80 cm
- C Maximum stroke: 73 cm
- D Minimum arm height: 107 cm
- E Maximum arm height: 180 cm
- F Maximum base length: 101 cm
- G Distance between front castors and base mast: 68 cm
- H Minimum internal width: 45 cm
- I Minimum external width: 55 cm
- L Maximum internal width: 84 cm**
- M Maximum external width: 94 cm**
- O Minimum legs height from ground: 8 cm
- P Maximum legs height from ground: 13 cm**

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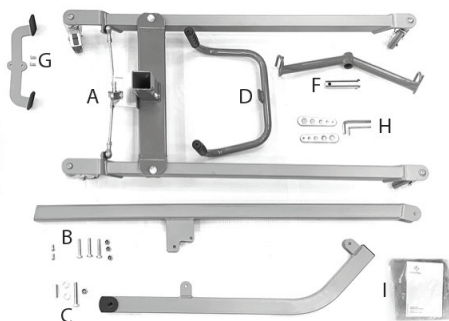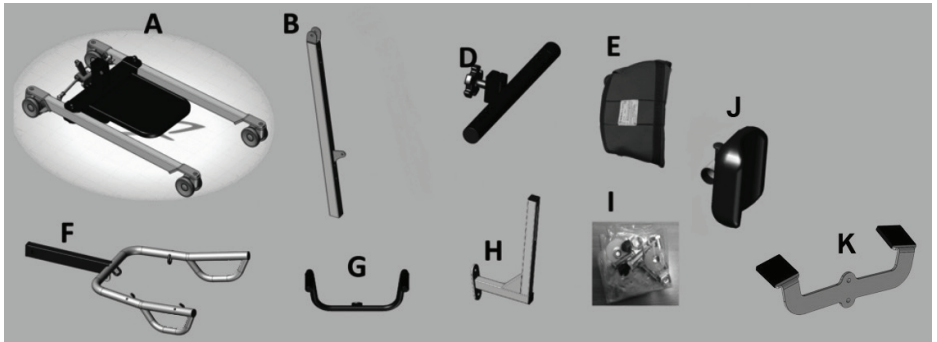
9. DESCRIPTION OF THE ACTIVATION KITS**9.1 Electrical kit (150/200 Kg)**

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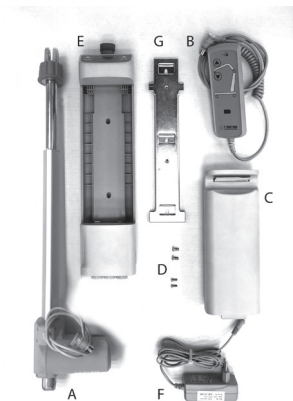
10. ASSEMBLY INSTRUCTIONS (FOR 0804935 & 0804936)

1. Unscrew the screw at the base of the column. (Fig.4)
2. Attach the column to the corresponding socket on the base. (Fig.4)
3. Insert the screw that secures the column to the base and tighten it securely. (Fig.4)
4. Insert the ring into the hole of the arm intended for joining the joint with the column (Fig.5a)
5. .. Connect and fasten the hinge between the column and the arm with the supplied screws (Fig.5b)
6. Install and secure the column handle with the supplied screws (Fig.6)

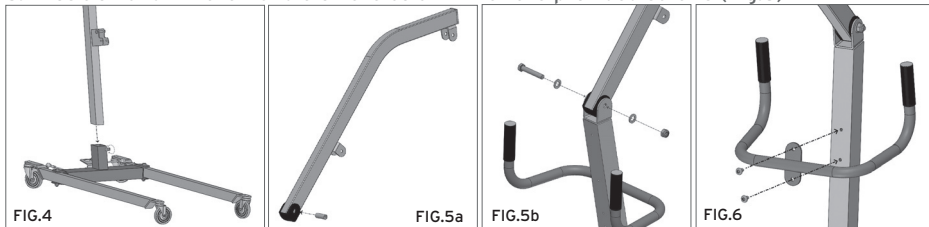5. Position and Fix the Handle on the column with the provided screws (Fig.6)



(image for illustrative purpose only)

6. Position and fix the sling bar with the provided screw on the corresponding holed bracket (Fig.7)

7. Fit the pedal for base legs opening and block it with the provided screw.

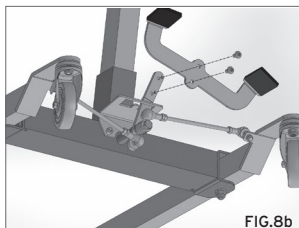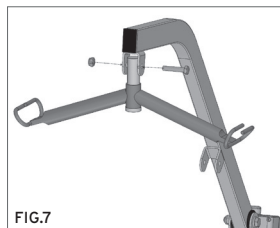8. You have completed the assembly of the patient lift. You will now install the electric lifting kit that is suitable for the selected model.

Installation instructions for electric mechanism (suitable for models 0804935 & 0804936)

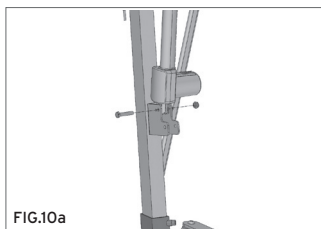
FIG.10a

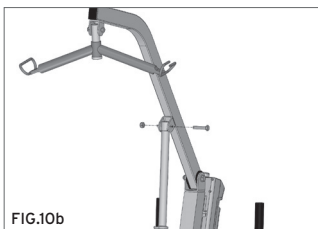
FIG.10b

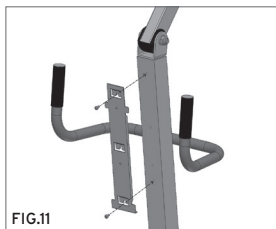
FIG.11

1. Remove the bolts intended for the arm and the holes in the column to secure the actuator. Install the actuator in the correct position and secure it using the previously removed bolts. (Fig.10a-10b)
2. Attach the control module support hook to the column with the supplied screws and screw in the corresponding holes. (Fig.11)
3. Attach the control unit to the arm, allowing it to slide from top to bottom and securing it with the supplied screws. (Fig.12a-12b)
4. Attach the battery to the control module socket. (Fig.13a-13b)
5. Connect the actuator connector to the central socket on the control unit and the remote control to the right socket. (Fig.14). The electrical kit is now properly mounted and the transport patient lift in the upright position is ready for use. (Fig.15)

FIG.12a

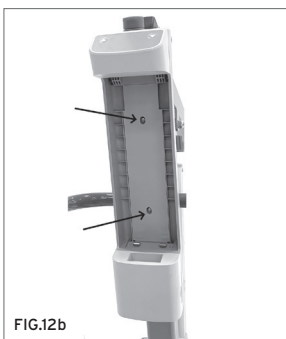
FIG.12b

FIG.13a

FIG.13b

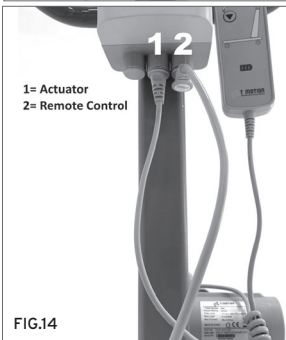
FIG.14

FIG.15

(image for illustrative purpose only)

Note: Before use, be careful to check that the emergency button is not pressed, and if activated, release it to allow the switch on of the control unit.

ASSEMBLY INSTRUCTIONS OF PRODUCT 0804937

1. Unscrew the screw at the base of the column. (Fig.1)
2. Attach the column to the corresponding socket on the base. Insert the screw that secures the column to the base and tighten it securely. (Fig.1)
3. Insert the ring into the hole of the arm intended for joining the joint with the column (Fig.2a)
4. Attach and fasten the hinge between the column and the arm with the supplied screws (Fig.2b)

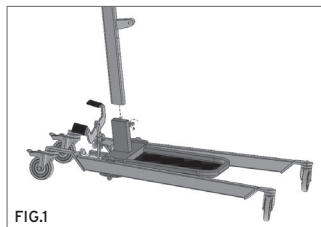
FIG.1

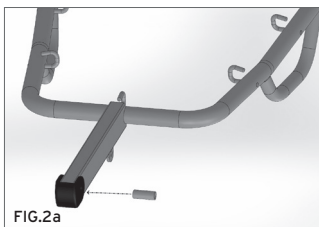
FIG.2a

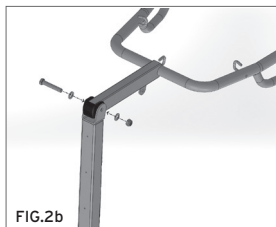
FIG.2b

5. Install and secure the column handle with the supplied screws. (Fig.3)
6. Adjust the pedal to open the base legs and secure it with the supplied screw. (Fig.4a)

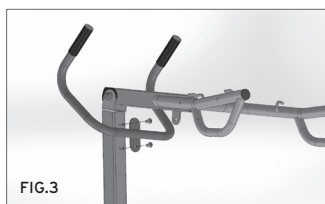
FIG.3

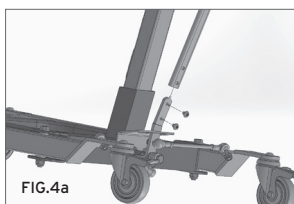
FIG.4a

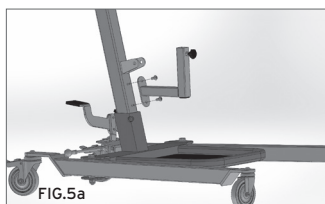
FIG.5a





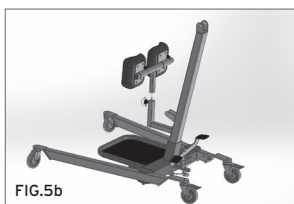
FIG.5b

7. Install and secure the knee bracket to the column with the supplied screws and position the knee brackets by adjusting their height with the supplied handle (Fig. 5a-5b).
8. You have completed the assembly of the patient lift. Now install the electrical lifting kit (see §9.2).

9.2 Electrical Kit Assembly

1. Remove the bolts prepared from the arm and from the column holes to fix the actuator. Fit the actuator in the correct position and fix it using the bolts previously removed. (Fig.6a-6b)
2. Fix the support bracket of the control unit to the column with the provided screws, screwed up the correct fitted holes. (Fig.7)
3. Fit the control unit on the bracket, letting it slide from top to bottom and blocking it with the provided screws. (Fig.8a-8b)
4. Fit the battery into the control unit cavity. (Fig.9a-9b)

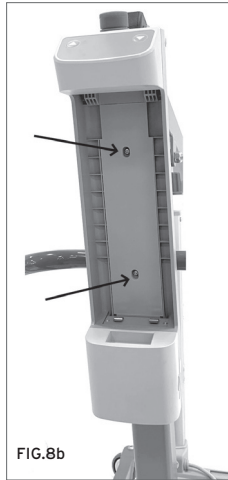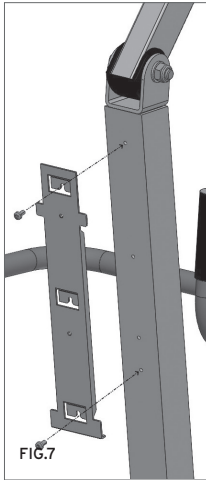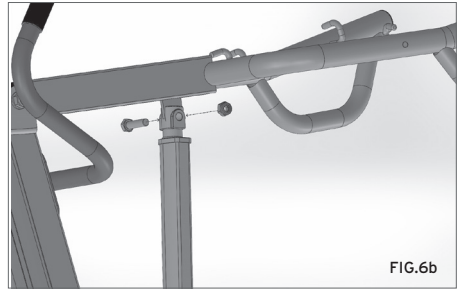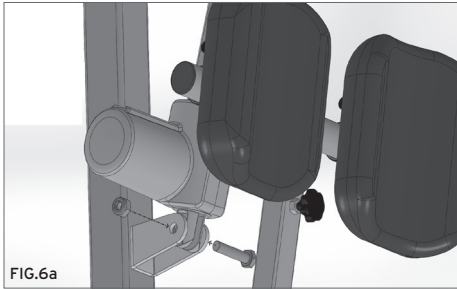5. Connect the actuator connector to the central socket of the control unit and the remote control to the right socket. (Fig.10). The electric kit is correctly mounted and the stand-up lift is ready to use. (Fig.11)

(images for illustrative purposes only)

12. INSTRUCTION OF USE FOR 0804935 & 0804936

ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCEDURE!

ALWAYS HELP THE PATIENT DURING THE DOWNLOADING PROCEDURE TO MAINTAIN HIS CORRECT POSITION!

WARNING!

While lowering the patient, remove any obstruction that could impede the downward movement. If there is an obstacle, lift the patient again and remove the obstacle to lower it.

BEFORE USING THE ELECTRIC LIFTING patient lift, IT IS RECOMMENDED TO CHARGE THE BATTERY FOR ABOUT 6-8 HOURS. BEFORE USE MAKE SURE THE BATTERY IS CHARGED

1. Before lifting a patient, open the base to the maximum possible width:
Pull the pedal slightly to open the foot towards you and then rotate it to the left and lock it in the open position.
(Perform the same steps in reverse to return the base to the closed position)
2. Activate the rear wheel brakes
3. Keep the patient in the center between the legs of the base and always facing the assistant
4. Secure the sling straps to the hoist
5. Press the up arrow button to lift the patient
6. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCEDURE!
7. Raise the patient to the desired height, so that he always maintains eye contact with his assistant
8. Release the rear wheels and move the patient
9. Before lowering the patient, lock the rear wheels again
10. To download the patient, press the down arrow button
11. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCEDURE!

ALWAYS HELP THE PATIENT DURING THE DOWNLOADING PROCEDURE TO MAINTAIN HIS CORRECT POSITION!

WARNING!

While lowering the patient, remove any obstruction that could impede the downward movement. If there is an obstacle, lift the patient again and remove the trade to lower him.

AUTOMATIC CONFLICT PREVENTION SYSTEM

The electric kit is equipped with an automatic collision prevention system, which stops the patient from lowering if there is unintentional contact of the patient with any obstacle.

EMERGENCY TERMINATION

The control unit has a red "STOP" button which, when pressed, deactivates all controls immediately. To return to normal operation, turn the knob to the right and pull it out.

EMERGENCY BUTTONS

There are two buttons on the control unit (one with an up arrow pointing and one with a down arrow) which can be pressed instead of the corresponding ones on the remote control to perform actions when the remote control does not work properly.

MECHANICAL LOWERING OF THE ACTIVATION MECHANISM

If you are applying manual / mechanical lowering, make sure it is performed in completely safe conditions for the patient. To activate the manual / manual descent, pull down the red ring on the top of the actuator and turn it to the left. After the download process is complete, return the red ring to its original position.

BATTERY CHARGING STATUS INDEX

There is an indicator with 3 LEDs for the battery charge status. This indicator is located on the remote control. The 3 LEDs indicate full charge of the battery, while if only one LED is lit, the battery is low and you need to charge it. The supplied battery, in new condition, with full charge and external temperature of use and storage 20 ° C, has an autonomy of more than 40 operating cycles with a load of 200 kg. When the load decreases, the battery life increases and consequently the possible operating cycles.

RECHARGE BATTERIES

Connect the adapter to the main socket and then to the socket on the control unit after removing the protective cap.

- There is an orange light on the charger adapter which lights up in orange during the charging process. Charge the battery until the LED turns green.
- Before turning on the hoisting patient lift again, be sure to disconnect the battery charger!
- Charge the batteries for at least 6 hours before using the patient lift

INSTRUCTIONS ON USE FOR CODE 0804937

BEFORE USING THE ELECTRIC LIFTING patient lift, IT IS RECOMMENDED TO CHARGE THE BATTERY FOR ABOUT 6-8 HOURS. BEFORE EVERY USE CHECK THAT THE BATTERY IS CHARGED.

Before lifting a patient, spread the legs of the base to the maximum possible width:

- pull the pedal slightly to open the foot towards you with your foot and then turning it to the left and locking it in the open position

Perform the same steps in reverse to return the base to the closed position

1. Activate the rear wheel brakes.
2. Keep the patient in the center between the legs of the base and always facing the assistant.
3. Adjust the position of the footrest by height and width using the adjustment handles.
4. Place the sling on the patient, adjusting the belt to the abdomen and armpit straps.
5. Carefully hook the sling straps to the hoist, as shown in the illustration (for the pelvic sling the central hook is used. For people under 1.70 m. The middle hooks must be used. For people above 1.70 m. The hooks at the ends of the arm must be used)

NOTE: The pelvic sling is recommended for patients with sturdy physique or great mobility difficulties. Thanks to the pelvic bands, the pressure in the armpit is reduced and the movement in an upright position becomes more comfortable for the patient

NOTE: If necessary, for tall people you can hang the sling on the middle hooks of the arm, passing the straps through the extreme hooks. In this case, the armpit straps shrink and allow the patient's chest to approach and rotate toward the arm.

6. Press the up arrow button to lift the patient.
7. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCEDURE!
8. Support the patient to the desired height, making sure that he always maintains eye contact with his assistant.
9. Release the rear wheels and move the patient.
10. Before lowering the patient, lock the rear wheels again.
11. To lower the patient, press the down arrow button.
12. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCEDURE!

WARNING

- Always help the patient during the lowering process to maintain his correct position!
- While lowering the patient, remove any obstruction that could impede the descent. If there is an obstacle, lift the patient again and remove the trade to lower him.

Automatic collision prevention system

The electric kit is equipped with an automatic collision prevention system, which stops the lowering of the lifting patient lift if there is inadvertent contact of the patient with any obstacle.

Emergency stop

The control unit has a red "STOP" button which, when pressed, deactivates all controls immediately. To return to normal operation, turn the knob to the right and pull it out.

Mechanical lowering of the actuating mechanism

If you are applying manual / mechanical lowering, make sure it is performed in completely safe conditions for the patient. To activate the manual / manual descent, pull down the red ring on the top of the actuator and turn it to the left. After the download process is complete, return the red ring to its original position.

Battery charge status indicator

There is an indicator with 3 LEDs for the battery charge status. This pointer is located on the remote control. The 3 LEDs indicate a full charge of the battery, and if only one LED lights up, the battery is low and you need to charge it.

Recharge batteries

Connect the adapter to the main socket and then to the socket on the control module after removing the protective cap.

- There is an orange light on the charger adapter which lights up in orange during the charging process. Charge the battery until the LED turns green.
- Before turning on the transport crane in the upright position again, do not forget to disconnect the battery charger!
- Charge the batteries for at least 6 hours before using the transport crane in an upright position.

INSTRUCTIONS FOR USE OF REMOTE CONTROL (FOR ALL CODES)

The remote control is connected to the control unit with a spiral extension cable

Buttons:

1. „UP“ key for lifting
2. „DOWN“ key for lowering

Visual indications:

3. LED for max. load exceeded: the lift movement is interrupted if the load exceeds the max. load threshold and the control unit generates an acoustic signal.
4. Led for monitoring and signaling the battery charge status

13. WARNINGS

- A lifting patient lift is a device designed to move patients with mobility problems for short distances, such as from the bed to the wheelchair or to a toilet seat and vice versa. It should not be used to transport the patient over longer distances.
 - When lifting the patient, make sure that the legs of the base are as far apart as possible moreover, the control lever is locked securely and the brakes are activated for the rear wheels.
 - When moving the patient, do not cross obstacles that could cause overturning of the lifting patient lift.

WARNING!

Always check that the sling is in good condition. Do not use bags with signs of wear or damage.

14. CLEANING

To clean the sling and the frame, use an alcoholic solution and neutral soap (in equal parts) mixed with water.

DISINFECTION: If you must disinfect the product, use a common neutral disinfectant detergent.

Note: Never use acids, alkalis or solvents, such as acetone or diluent. Keep away from light, moisture and heat.

15. MAINTENANCE

- It is extremely important to check the hoisting patient lift before each use. In particular, check that all screws and bolts are tight.
 - Check that the patient lift can move comfortably when the wheels are not applied to the brakes.
 - Check that the rear wheel brakes are working properly.
 - It is very important to check the connection pins and suspension rings regularly are in the correct positions (the pins connect the actuator to the top and bottom of the patient lift and the hoist where you attach the sling).
 - Mechanical parts must be lubricated with a drop of oil to facilitate their movement and to enhance their wear resistance. In particular, pay special attention to the mechanical retractor for the legs of the hoisting patient lift. Lubricate as indicated above every two to three months.

16. SPARE PARTS AND ACCESSORIES

For spare parts and accessories, you can refer to our catalog.

ACCESSORIES

0808094 Standard Sling with Headrest

0808738 Sling for Stand-Up Lifter

YOU USE ONLY ORIGINAL MOBIAK ACCESSORIES. THE COMPANY IS NOT RESPONSIBLE FOR ANY ACCIDENT THAT HAPPENS DUE TO IMPROPER USE.

17. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to dispose through the joint municipal ecological islands for the expected operations of recycling the materials used.

17.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the

product itself bears the barred bin symbol.

17.2 Treatment of dead batteries (Directive 2006/66/CE)

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from whom you purchased the device.

17.3 INCIDENT REPORTING

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

18. WARRANTY

Each product has a 2 (two) YEAR warranty from the date of purchase, except for the exceptions specified below. The warranty will not apply in the event of damage due to improper use, misuse, product modifications or any other use that does not comply with the user manual. Proper use of the product is indicated in the user manual. MOBIAK bears no responsibility for material damage or injuries due to incorrect assembly / improper use of the device that does not comply with the user manual. MOBIAK does not guarantee its products for damage or damage in the following cases: natural disasters, unauthorized maintenance or repair, damage caused by power supply problems, use of components or spare parts not provided by MOBIAK, non-compliance with the guidelines and instructions for use, unauthorized modifications, damage due to transport or maintenance of the device in a manner contrary to what is stated in the manual. Components that are subject to normal wear and tear are not covered by this warranty if the damage is due to normal use of the product.

18.1 Guarantee for rechargeable batteries (if supplied)

Original and replacement batteries are covered by a 90 (ninety) day warranty on performance and a 6 (six) month warranty on manufacturing defects or as defined by the law. The guarantee for fully charged batteries that are not used for more than 3 (three) consecutive months canceled automatically. The guarantee for fully discharged batteries that are not used for more than 3 (three) consecutive days is automatically canceled.

18.2 Repairs

If a product suffers material or structural damage within the warranty period, the customer will review whether the product is covered by the warranty. MOBIAK, under its sole discretion, it can replace or repair the product covered by the warranty at a MOBIAK resale point or at its own premises. MOBIAK may assume the labor costs for the repair of the product if it is ascertained that the repair is covered by the guarantee. A repair or replacement does not renew or extend the warranty.

A product that is not covered by the warranty can be returned for repair only after obtaining approval from the customer service department of MOBIAK. Labor and shipping costs for one warranty repair is fully borne by the customer or reseller. Repairs for products without warranty have a warranty of 6 (six) months from the date the customer receives the repaired product.

The customer is informed if MOBIAK decides that the product is not defective, since MOBIAK examine and test a returned product. The product will be sent to the customer, who must bear all shipping costs.

18.3 Spare parts

Original MOBIAK spare parts have a warranty for 6 (six) months from the date of receipt.

18.4 Exemption clauses

Apart from what is clearly defined in this warranty and in accordance with the law, MOBIAK does not offers to its customers any other statement, warranty or condition, express or implied, including any declaration, warranty or condition of marketability, non-infringement or non-interference, suitability for a particular purpose. MOBIAK does not guarantee the use of products will be continuous and trouble-free. T The duration of any implied warranties that may be enforced by law is limited to the warranty period. Some countries do not allow restrictions on the duration of an implied warranty or exemption or restrictions on unintentional or indirect damage to products for consumers. In such countries, some of the exemptions or limitations of this warranty may not apply to the user. This warranty is subject to change without notice.

WARRANTY FORM

BUYERS DETAILS:	
FULL NAME:	
PURCHASING DATE:	
SERIAL NUMBER:	
LOT or SERIAL NUMBER	
DEALER DETAILS	
FULL NAME:	
ADDRESS:	
PHONE:	
DEALER SIGN AND SIGNATURE	

NOTES

NOTES