**GR  
EN**

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T: +30 28210 63222 | WEB: www.mobiaicare.com

## **Intended Use:**

The examination bed is intended for use in a Medical Office, in a Physiotherapy Clinic or in any other place for the diagnosis of patients and the short-term application of physiotherapy. Ideal for transport because it is foldable, has a small size and light construction.

## **General Warnings:**

- If you don't read this user manual, it is preferable not to use this product or another available part.  
If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the device from heat sources.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.

## **Package Includes:**

1. The Examination Bed
2. Headrest cushion for the hole.
3. Inner Head Pillow
4. Side Armrests
5. Front Armrest
6. Carrying bag

## **Before use:**

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check, if the product is properly assembled.
3. Make sure all screws are secure and the cable has not deteriorated or frayed.
4. Make sure that the floor surface that will support product is flat and all four legs are open in the correct position.
5. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.

## **During The Use:**

Avoid contact of the product with creams and massage oils, always placing a bed cover or other protection between the bed mattress and the patient.

After each patient you should change this cover or other protection that you have used and before the next session put a new one.

## **Maintenance & Cleaning:**

- Clean the product surface after each use with a soft cloth.
- Dirt, oil or debris must be removed from the product surface.
- Use only a cloth and mild soap on surfaces that need cleaning.
- Then use a clean cloth and dry well before using the product.  
Do not leave the product wet.
- If you need to disinfect the product, use a common, mild disinfectant
- Protect the item from scratches, cuts and punctures.

### **CAUTION (!):**

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAC S.A reserves the right to change the information contained in this document without previous notice
- There is a  $\pm 3\%$ . Deviation for the Dimensions of the Products.

### **Environmental Protection:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

### **INCIDENT REPORTING:**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

### **Declaration of Conformity:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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Unfold the foldable examination bed

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*Release the buckles of the folded examination bed as Figure #1.*

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***Side armrests installation***

*Insert the side armrests stem into the bed side holes as Figure #4.*

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***Installation and operation for the headrest and front armrest***

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*Put the headrest stem through the front armrest straps and insert into the headrest holes as Figure #5.*

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*Place the remaining single strap over the middle part of the headrest as Figure #6.*

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Headrest is adjustable by using the handle as Figure #8.

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*Slide the strap up and down to adjust the desired height for the front armrest*

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*There are 9 adjusting holes on each leg. Press down the grip knob, prolong or shorten the leg, fix the grip knob to the adjusting hole in the desired position. Repeat the above process to adjust the same height for the other legs. See Figure #9.*

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*When not in use or during transporting, adjust the bed leg to the lowest position, detach the headrest and armrests, attach them under the bed as Figure #2. Then pack the folded bed in the bag after locking the buckles.*

**Warranty:**

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use or over time, like the mattress upholstery, screws, knobs and cables.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

**Warranty Claim Form:**

<p><b><u>Buyers Data</u></b> Name: ..... Date of purchase: .....S/N..... LOT:.....Phone: .....</p> <p><b><u>Resellers Data</u></b> Name :..... Address:..... Phone: .....</p> <p style="text-align: right;"><b>Reseller Stamp</b></p>
---

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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MOBIAK S.A