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INTENDED USE:

The Beds 0805425 & 0805427 is used for patients who have health problems and they need to stay lying in bed. They are designed for people who need to be treated for health reasons or disability in nursing homes, hospitals or even in healthcare facilities.





CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts

PICTURES OF CORRECT AND WRONG USE OF THE PRODUCT:

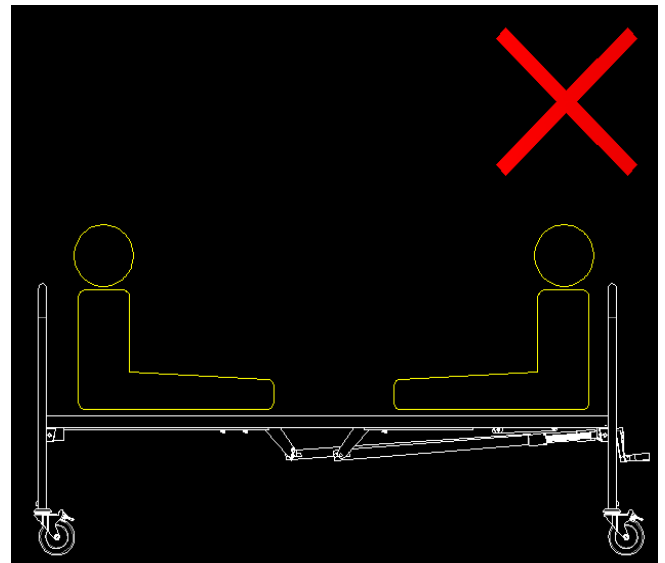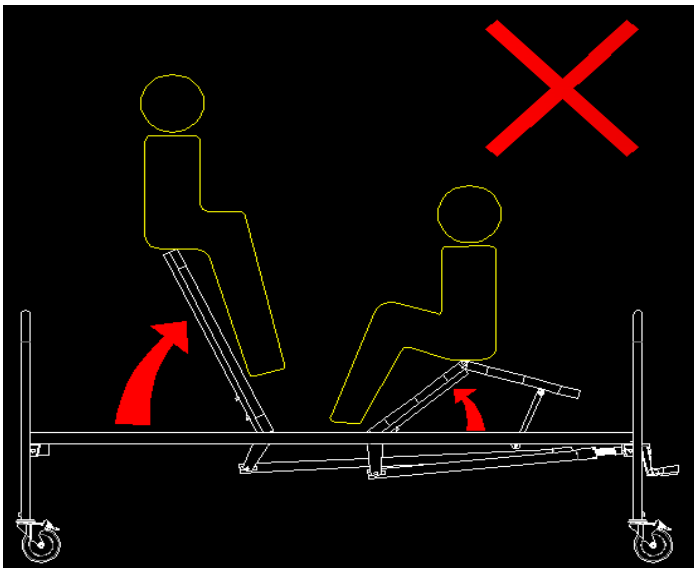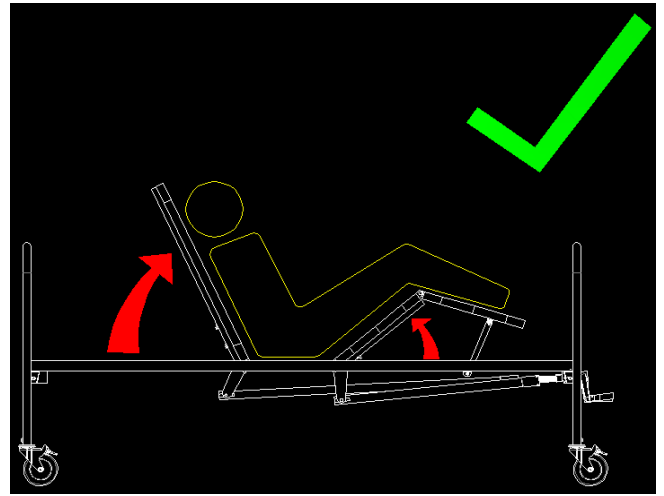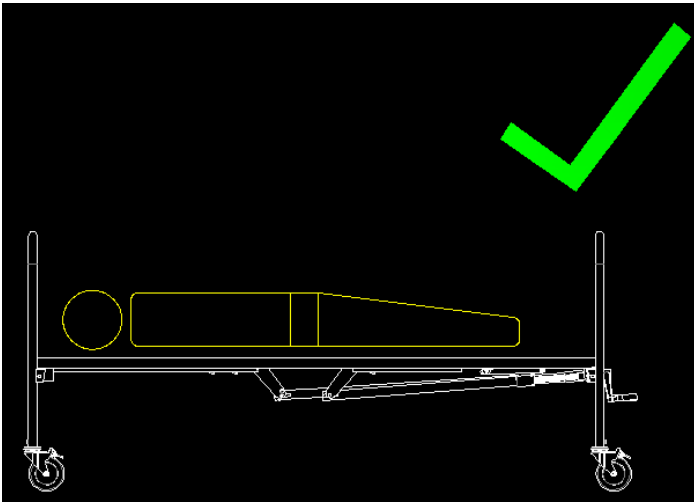DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

PRECAUTIONS FOR USE


- Do not exceed the maximum load capacity for any reason.
- When lowering the head or foot section, make sure that nobody inserts hands, legs or other body parts under the bed base
- Make sure that the floor surface supporting the bed is flat.

BEFORE USE:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
2. In case of damage do not use the product and contact your dealer for further instructions.
3. Check, if the product is properly assembled and all screws are secure and well screwed on.
4. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects


MATTRESS SELECTION

Use mattresses or anti-decubitus mattresses according to the prescription of the treating Physician. The size of the mattress should not be larger than the frame of the bed. All MOBIAK mattresses and anti-decubitus are suitable for beds.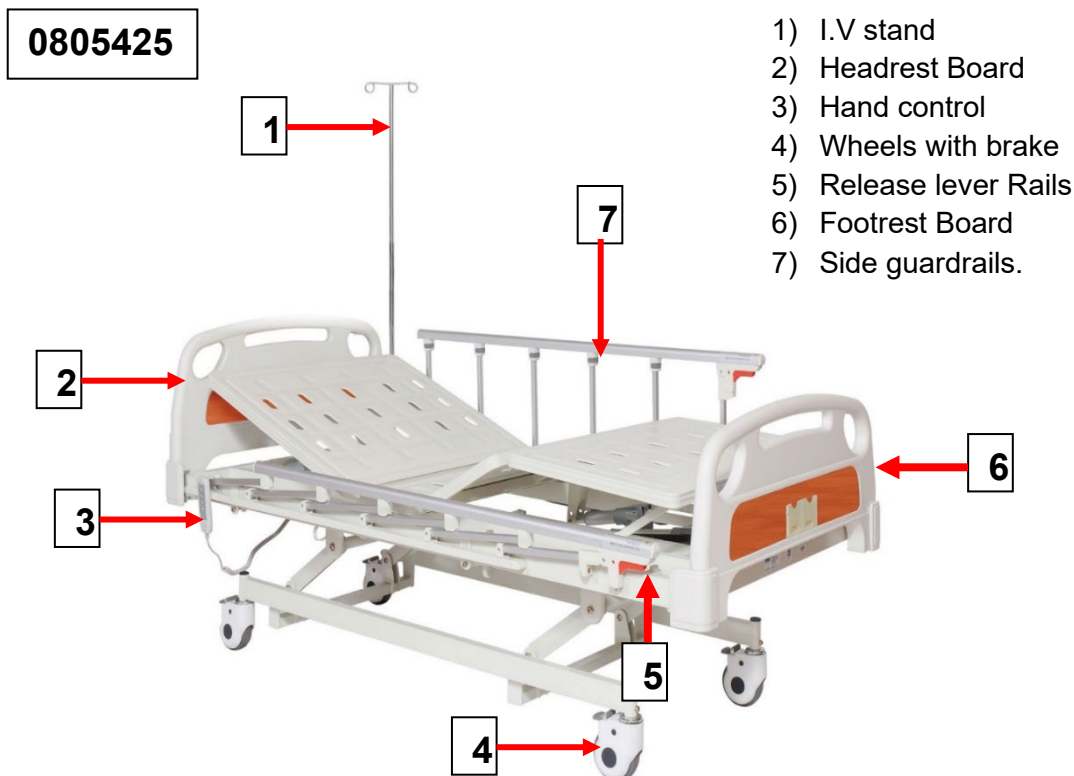

0805427

- 1) I.V stand
- 2) Headrest Board
- 3) Hand control
- 4) Wheels with brake
- 5) Rails Release lever
- 6) Footrest Board
- 7) Side guardrails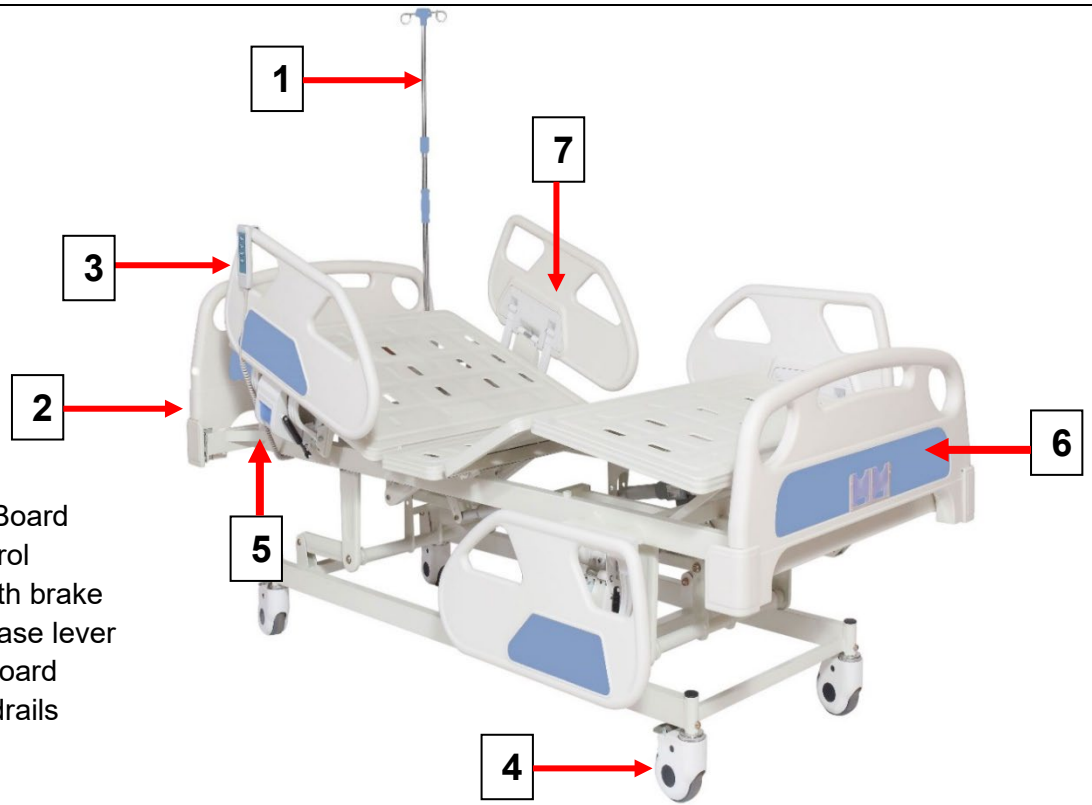

ASSEMBLING:


- The product is assembled ready for use
- Remove the product from the carton.
- Release all connections that have tire-up.
- Plug in the 220 Volt power supply.
- Press the appropriate button on the remote control to adjust the inclination of the back, legs & knees as well as the desired height of the bed from the floor.
- You may need to install the rails and plastic board (headboard and footrest) for product 0805425 or 0805427.

NOTE: The illustrations for the assembly are indicative to make the assembly easier. May will be differences from reality. Assembly tools are included in the package.

TO INSTAL THE PLASTIC SIDE BOARDS (HEADREST AND FOOTREST) :

- 1) Install the Plastic Side Boards (headrest and footrest) in the special iron sockets of the bed..
- 2) Press lightly down to secure.
- 3) Put the safety lock.


As the pictures #01 to #05 shows.

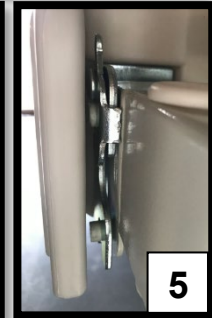I.V STAND:

Insert the IV. Stand "7" that are in the package, into the special slots on the edge of the bed.

As the pictures #06 to #08 shows.

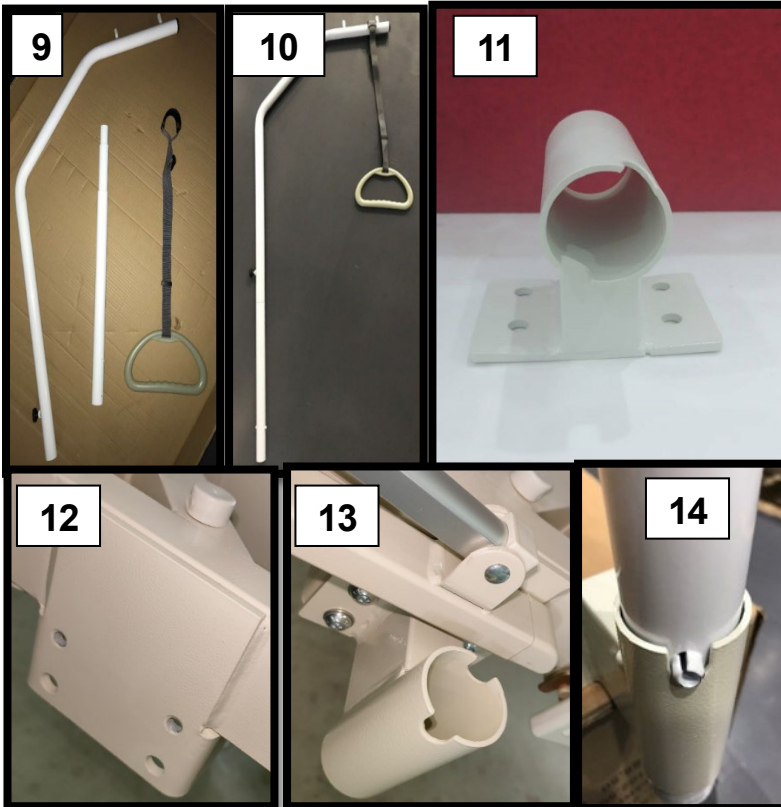To place the Monkey Pole with plastic handle on the bed*:**

5. You must separately purchase the Monkey Pole 0805423 and Base 0805432.
6. Assemble the Monkey Pole 0805423 that comes in three parts.
7. Screw the Base 0805432 on the special bed socket that are on right or left.
8. Insert the Monkey pole 0805423 into the notched tube to stabilize it.


As the pictures #9 to #14 shows.



***** ONLY FOR BED 0805425. We can install the Monkey pole base 0805432 on beds with production date from March 15, 2021 and after. The Bed 0805427 does NOT have the socket to install the monkey pole base.**

IN CASE THAT THE RAILS NEED ASSEMBLING:

1. For the item 0805425, Install the rails at the desired height position, with the safety handle to be on the side of the legs.
2. Screw and secure the rails with the screws & with the tools included in the package.
3. Lift the side rails until you hear the characteristic locking sound. To release the rails, press the button to unlock them and lower them.

As the pictures #15 to #21 shows.

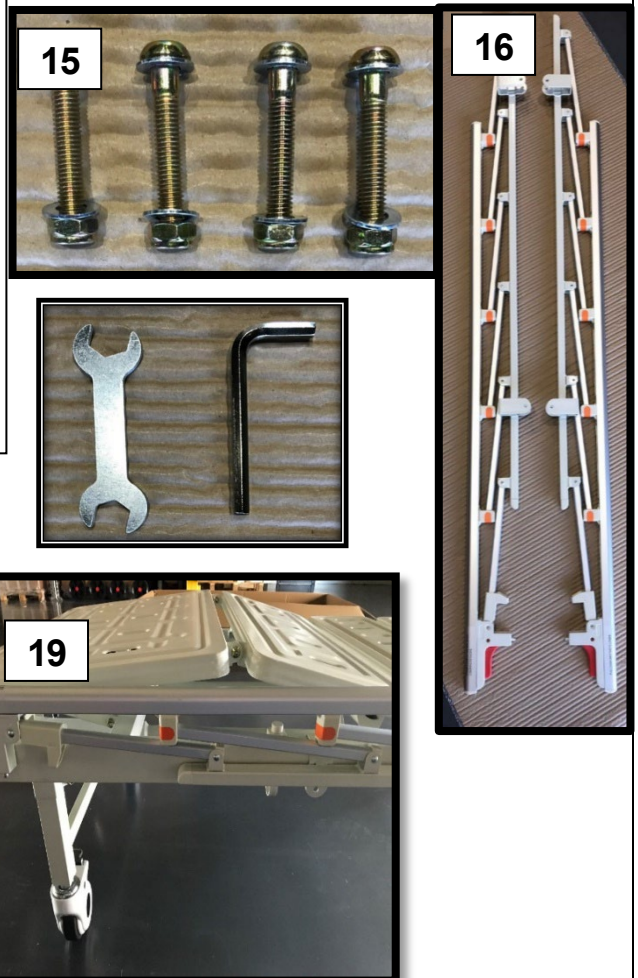
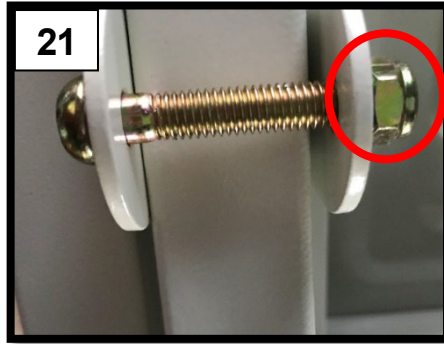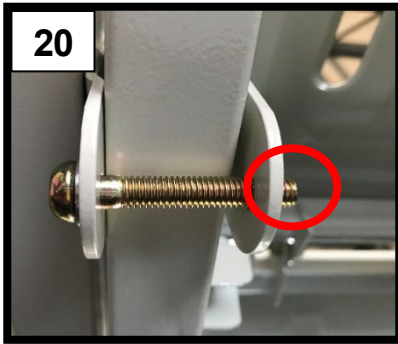
FOR THE SIDE RAILS OF 0805427:

3. Pull Handle "A" from the side of the bed and push down to lower.
4. Do the reverse procedure to move up.

As the pictures #22 to #23 shows.

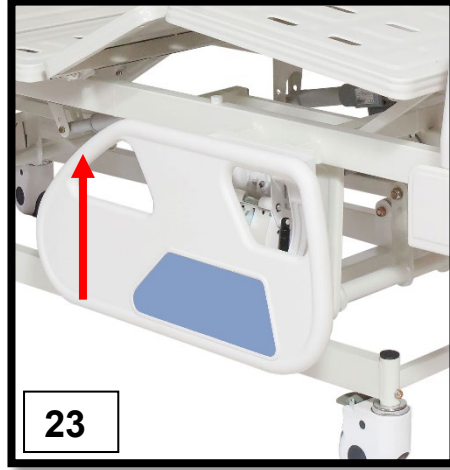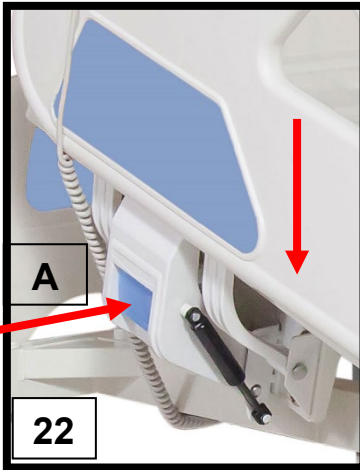Brake activation: push the brake activation lever down with your foot.

Brake deactivation: push the brake deactivation lever down with your foot.

DISASSEMBLY:

Follow the steps described above in reverse order.

Up & Down button for Backrest positioning

Up & Down Button for Backrest and Foot (together) support positioning

Up & Down button for Foot support


Up & Down button for Bed High positioning

HANDCONTROL

For the operation of the Hand controller, after connecting the product to a 220 Volt power supply, press the "up arrow" button to adjust the moving up and the "down arrow" to adjust the moving down, as shown in the side image.

MAIM CONTROL UNIT EXPLANATION:

- M0: Hand control
- M1: Backrest inclination adjustment motor
- M2: Legrest inclination adjustment motor
- M3: Total Heigh adjustment motor.

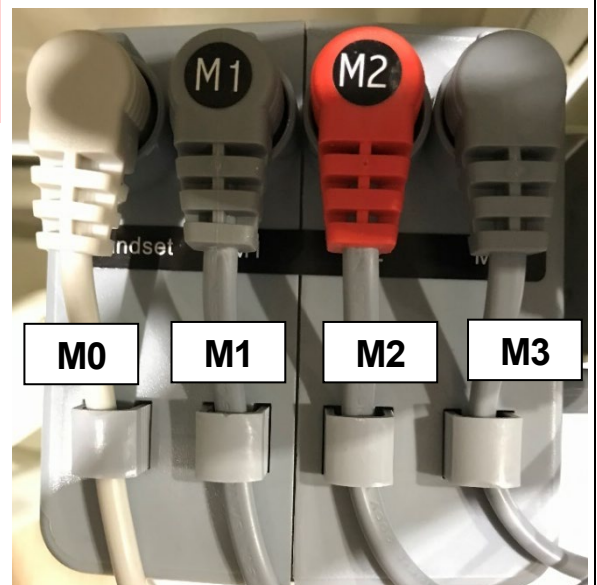CLEANING AND DISINFECTION:

Use only a wet cloth and mild soap on surfaces where they need cleaning.

Then dry thoroughly before you use the product.

DO NOT leave the product wet.

Do not use chemical cleaners to clean the frame. This could cause damage to the surface.

If you need to disinfect the product, use a common, mild disinfectant.

Protect the item from scratches, cuts and punctures.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, brakes, wood, or plastic parts. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty. No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user-customer or by dealer.

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
DDRESS:			
PHONE:			
			SIGN AND SIGNATURE

 **MOBIAC S.A**
KAGIANA AKROTIRIOU, XANIA - KPTH
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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