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EN

WALL-MOUNTED BATHROOM SEAT "PRIME" INTENDED USAGE:

The Wall-Mounted bathroom seat "PRIME" is meant to be used while showering, allowing the user to achieve greater independence, reducing muscle fatigue due to standing position as well as slip and injuries risks. The folding mechanism make these products suitable even for small bathrooms and help the user to keep the shower tray tidy and free from any obstacle without effort.

INDICATIONS AND CONTRAINDICATIONS:

The Wall-Mounted bathroom seat "PRIME" is aimed to elderly people, wheelchair users, people with reduced mobility of lumbar, knee or hip, and to all those who have difficulty to maintain the standing position during the shower stages of daily personal hygiene. The user is required to have autonomy of balance during the sitting and standing procedures as well as to have adequate ability to keep sitting. The wall seat must not be used if the user has problems related to balance or perception disorders.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceed.
- Before every use, be sure the wall seat is correctly assembled.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting process.
- Do not modify the wall seat and its components in any way.
- Do not use the wall seat if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- After use, the wall seat should be folded in its non-use configuration.
- Do not use the wall seat for a different purpose from the intended one.
- The wall seat must not be stored outdoors.

MAINTENANCE INFO:

The products maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.

INCIDENT REPORTING:

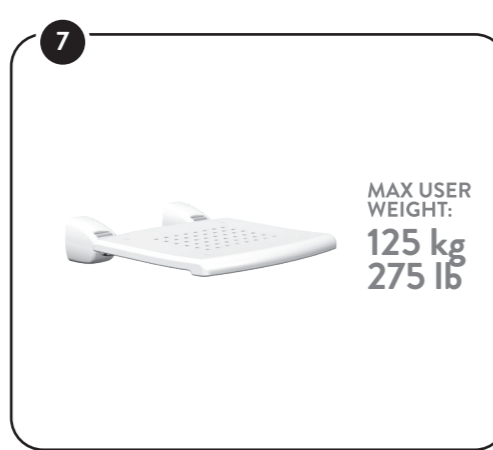
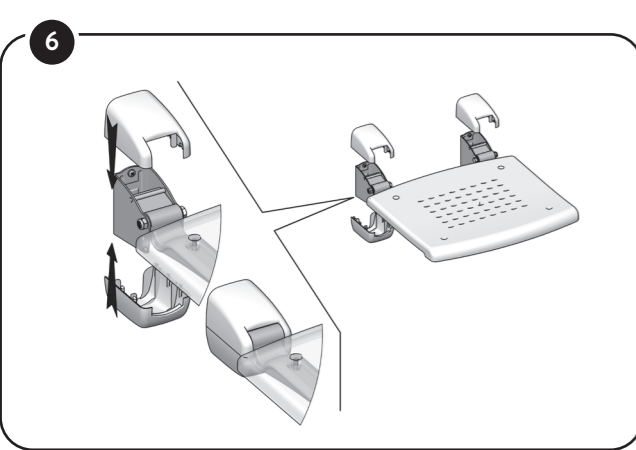
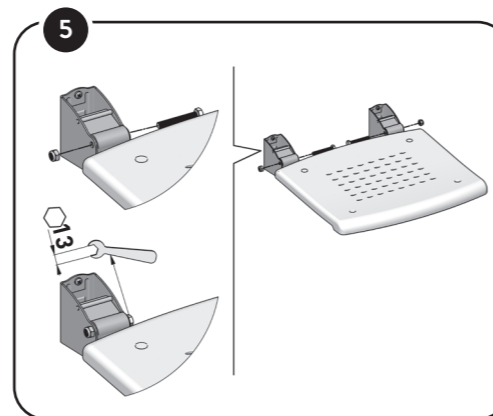
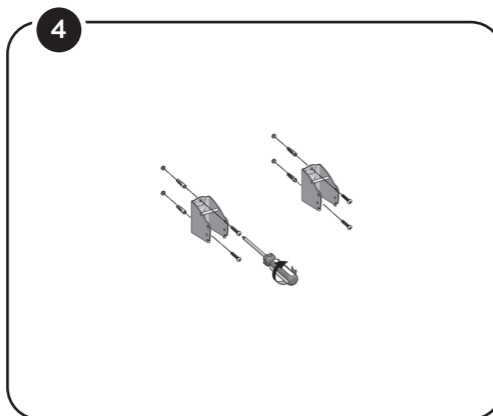
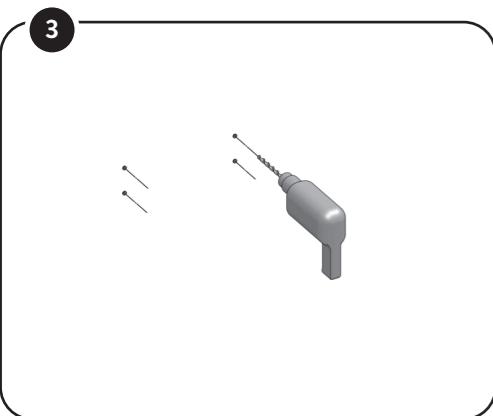
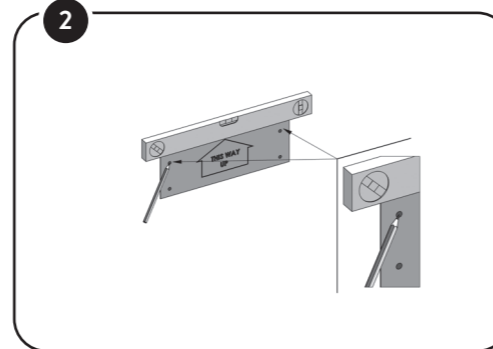
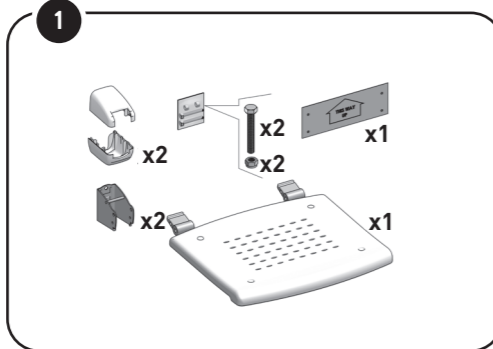
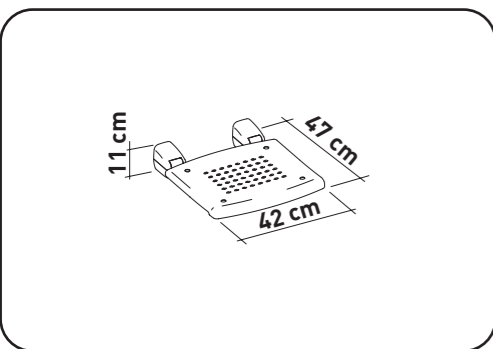
If there was a serious incident involving this device, contact the manufacturer of this product at mobiakcarequality@mobiak.com and the responsible authority in your country.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the standards and legislation referred to.

ATTENTION!

The holes for fixing the wall mounted seat 0805519 to the wall must be place in correspondence of the vertices of a rectangle of height 70 mm and width 300 mm. See the template included in the packaging for further information. Please note that the screws kit for fixing the product to the wall, is not included, the type of screws to be used changes according to the specific material of the wall of the final user. Providing a generic screws kit might be dangerous if it is not made to be used with the specific wall. Hence, the final user must provide themselves to choose the appropriate screws kit, according to the type of the wall of bathroom.

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