GR EN

MOBIAK CARE[®]
MEDICAL EQUIPMENT

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T: +30 28210 63222 | WEB: www.mobiakcare.com

MOBIAK®
MEDICAL • GASES • FIREFIGHTING est. 1977

Intended Use:

The Mattress Cover creates a waterproof protection for the bed mattress. The material of the cover is Nylon, which offers durability and protection of the mattress from liquids. Its waterproof sides do not allow any form of moisture to penetrate the mattress.

The cover is accessory for the mattress and bed which are offered separately.

The dimension of the Cover is 200x90 and is suitable for all the mattresses with these dimensions and thickness up to 15 cm.

The net weight of the Waterproof cover is : 142gr.

Further Instructions:

- Place the mattress inside the cover.
- Use the zipper to close the cover and secure the mattress.
- DO NOT use a mattress larger than the size of the cover

Maintenance & Cleaning:

- Hand washing at 30-40 °C.
- No chlorine.
- No dry cleaning.
- No dryer.
- Dry it on a hanger.
- No ironing.
- If necessary you can use a cloth and mild soap in areas where need cleaning.
- Then dry well with a dry cloth before using it again
- If you must disinfect the product, use a common, mild disinfectant.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

Environmental Protection:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

Declaration of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides

WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

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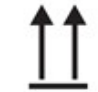
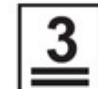
Description of symbols that have been printed on the label, Master Carton and or User Manual

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