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**INTENDED USAGE:**

The Commode Chair can be used by people with limited mobility who are unable to go to the toilet in the house and as an aid on the toilet bowl without the container. Designed for indoor use only.

Do not use the product if you have severe balance problems and you are unable to stay seated.

**CAUTION (!):**

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- If you don't read this user manual, it is preferable not to use this product or another available part.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a  $\pm 3\%$ . Deviation for the Dimensions of the Products.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts
- Protect the product from scratches, cuts and punctures.

**SAFETY WARNINGS:**

- The maximum user weight indicated in the instructions must not be exceeded.
- Before every use, be sure that the product is correctly assembled in a stable position.
- The product can be used only on surface that are free from obstacles.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting or assembling process.
- Do not swing on the product, this may lead to slipping, overturning or subsidence of the legs. When you are using the product, always keep the seat on its 4 legs.
- Do not modify the product and its components in any way.
- Do not use the product if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the product.
- After use, the product should be removed from the shower tray to avoid any obstruction.
- Do not use the product for a different purpose from the intended one.
- The product must not be stored outdoors.
- Do not place or remove the commode pan while the patient is sitting in a chair: there is a high risk of injury!

**MAINTENANCE INFO:**

In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.
5. If you need to disinfect the product, use a common, mild disinfectant
6. Clean the toilet container immediately after use. Initially, rinse with warm water. Then use a soft cloth impregnated in a mild detergent or antiseptic solution. Wipe with a clean, soft cloth.
7. ATTENTION: Do not use acidic & caustic dilutions

**ENVIRONMENTAL PROTECTION:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

**INCIDENT REPORTING:**

If there was a serious incident involving this product, contact the manufacturer of this product at [mobiakcarequality@mobiak.com](mailto:mobiakcarequality@mobiak.com)

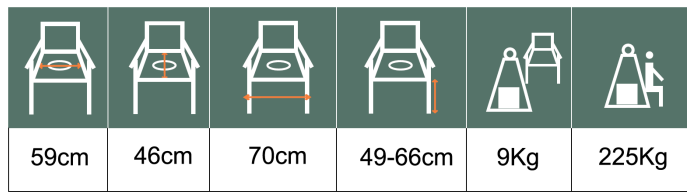
**DECLARATION OF CONFORMITY:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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**Assembling:**

The product is delivered assembled & ready to use. You may need to assemble the seat with the backrest. The product that you have in your hands may differ from the one in the following photos. The assembly process is similar. **NO** tools needed for the assembly.

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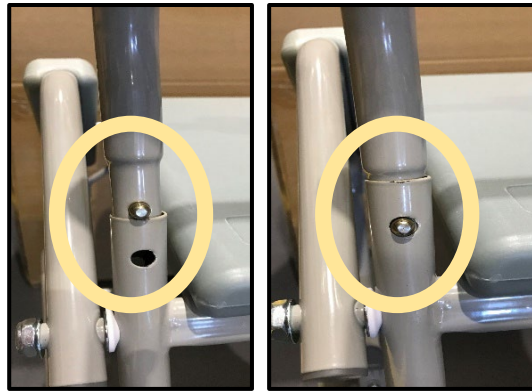
**Packaging contents:** Inside the package is the main frame of the product, the backrest, and the commode pan, as shown in the photos below

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
To install the Backrest onto the seat, insert at the same time the tubes of the backrest into the special slots on the seat until they lock in place.

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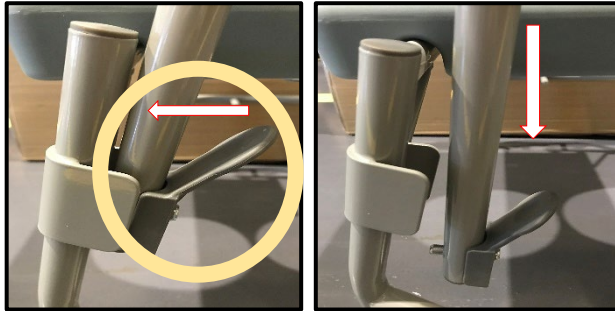To fold armrest, press the safety lever and push the armrest downwards.

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From the front, Insert the plastic commode into the guides and push inwards all the way under the seat.

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**WARRANTY:**

The product is guaranteed for a period of one (1) year from the purchasing date. The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

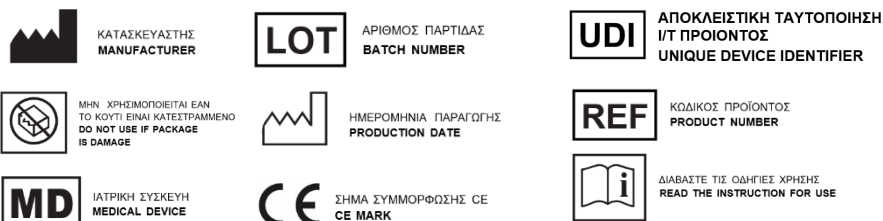
Also DOES NOT cover the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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Description of symbols that have been printed on the label, master carton and or user manual

**REF** : 0808524

**MOBIAK S.A**  
ΚΑΘΙΑΝΑ ΑΚΡΩΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ  
KATHIANA AKROTIRIOU CHANIA - CRETE  
T: +30 28210 63222 | WEB: www.mobiakcare.com

MULTILINGUAL  
MANUALS