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INTENDED USE:

This product is an orthopedic splint and joint stabilizer placed in the ankle area. Immobilizes, Blocks or Controls Movement (Depending on the version) During the Treatment of Ligaments or Fixed Fractures.
The “Range Of Movement” version, Allows the control of Extension from 0 to 30 ° and Flexion from 0° to 45°, in steps of 10 ° or 15 °.
It is applied in Acute ankle sprain, Stress fractures of the lower leg, Stable delayed union or non-union fractures of the distal tibia and fibula, Achilles tendon repairs.

CAUTION (!):

- The use of the product is indicated only after doctor's prescription.
- The product requires assembly and prior adaptation by an orthopaedic technician or qualified personnel. After the orthosis has been adapted to a patient, it should only be modified by qualified personnel.
- The product is for single patient use, only given it must be specifically adapted to the said patient. Do not reuse on other patients.
- Monitoring must be frequently revised by the physician. If any secondary effects are noticed, inform the physician immediately
- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a ± 3%. Deviation for the Dimensions of the Products.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

GENERAL SAFETY WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Its application is the sole responsibility of the treating physician.
- The Customization of the product must be done by qualified health personnel.
- The frequency & duration of use should be determined by the treating physician.
- For maximum effectiveness you must respect the daily usage duration according to the instructions of the treating physician or specialized health personnel.
- To ensure the full therapeutic effect, you must use the correct size.
- This product is intended to be used by one patient only.
- In pressure bearing areas, the skin must not be injured or hypersensitive.
- Do not wear the product while having a bath or a shower.

BEFORE EVERY USE:

- Check if the frame is damaged in order to guarantee a safe use of the product.
- In case of damage do not use the product and contact your dealer for further instructions.
- Always check the condition of wear for the parts to ensure the absolute safe use.
- Regularly check that the straps are tense and in good condition (free of grease, cuts, tears, etc.) and that the buckles fasten well.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

MAINTENANCE, CLEANING & DISINFECTION INFORMATION:

To guarantee a safe use and adequate standard of hygiene, the user should perform these cleaning procedures.

1. Clean the product frequently.
2. Remove the evident dirt before to proceed with the most accurate cleaning.
3. Clean the hard surfaces with a damp cloth.
4. The fabric parts of the product are easily removed allowing them to be washed. Wash in hand at a maximum of 30 °C.
5. Do not use bleach.
6. Do not iron.
7. Do not dry clean.
8. Do not tumble dry.
9. DO NOT leave the product wet.
10. If you need to disinfect the product, use a common, mild disinfectant.
11. It is recommended, to careful cleaning of the Velcro and to eliminate threads, lint, etc., and thereby preserve correct adhesive properties.

APPLICATION INSTRUCTIONS:

The product is assembled and ready for use.

It's fitting and application is the sole responsibility of the treating physician and must be done by him or by specialized health personnel.

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- For first time application, loosen straps and remove liner from boot.
- Place foot in liner and secure with contact closure. Fasten the foot flaps on the liner first, making sure they are snug. Wrap and fasten the leg portion of the liner, again making sure that it is snug from the bottom to top. Some trimming of the liner may be necessary to accomplish optimum fit.
- Spread the uprights using both hands and step into boot, aligning uprights with midline of the ankle.
- If uprights have plastic tube liner, remove them now and affix upright to liner.
- Secure boot straps at the toes and working up the leg.
- For hygiene and comfort, it is recommended to wear a cotton garment or a sock, preferably seamless, underneath the brace.

For the Items with ref. 0809097, 0809098, 0809099:

Limitation of joint flexion/extension:

To adjust range of motion, Press the button to adjust the degrees of restriction indicated by the physician and secure with the tire-up that are included in the package. The "0" lock setting are used to lock the brace in full Flexion or Extension.

To **inflate** the air chambers, first make sure the deflate knob is screwed, press the Inflate Knob (poir) repeatedly until the air chambers will inflated to a comfortable level.

To **deflate** the air chambers, first make sure the deflate knob is unscrewed, press the Inflate Knob (poir) repeatedly until the air chambers will deflated to the desired level.

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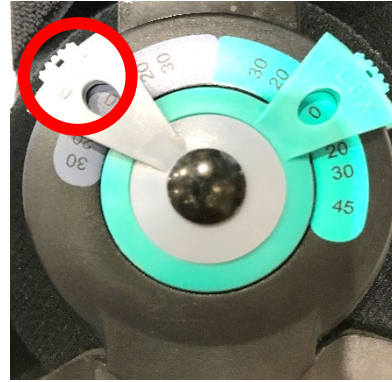
WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date.

The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by the dealer.

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