GR EN

ΜΟΒΙΑΚCARE®
MEDICAL EQUIPMENT

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INTENDED USE

These wheelchairs have been designed for physically or mentally handicapped people with limited mobility.

It makes possible to transport people with mobility problems or limited mobility and can be used for movement with the help of an attendant.

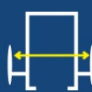
When the wheelchair has a rear wheel with a diameter of 13 to 40 cm and does not have a hand rim on the wheel then the user needs an attendant to move the Wheelchair through the handles located on the back of the Wheelchair.

SPECIFICATIONS

FRAME	ALUMINUM
SEAT	REINFORCED FABRIC
ARMRESTS	DETACHABLE WITH ADJUSTABLE LENGTH
FOOTRESTS	DETACHABLE, RECLINING & ADJUSTABLE
REAR WHEELS	16" SOLID
FRONT WHEELS	6" X 1" SOLID (152X25MM)
MAX LOAD	100 KG

- 45° Reclining Seat,
- 90° to 165° Backrest Tilt
- Crossed Safety Belt
- Height-adjustable, Anti-tilt Support Casters

DIMENSIONS

													GROSS WEIGHT
0811982	43-49 cm	42 cm	40 cm	45 cm*	62-64 cm	102-114 ** cm	104-117 *** cm	50 cm	16" 6"	100kg	21.5kg	24.7kg	
0811983	38-44 cm	36 cm	40 cm	45 cm*	58-60 cm	102-114 ** cm	104-117 *** cm	50 cm	16" 6"	100kg	21kg	24kg	

*without the headrest. ** with the footrests in the lower and fully reclining position.

*** with the headrest in the lower and higher position.

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the wheelchair by any removable parts.
- Sitting down in and standing up from the wheelchair may be performed independently, provided you have an adequate mobility and upper body strength.

ASSEMBLY

Please refer on **ANNEX A**

SYMBOLS

Please refer on **ANNEX B**

IMPORTANT!

The construction of this wheelchair is designed to provide maximum safety. However, users may be at risk by using their wheelchairs unsuitable. For your own safety, the following rules must be observed. Nonspecific or incorrect adjustment work can increase the risk of accidents.

- Always wear light-colored clothes when moving in dark places so that you can be reached more easily from others.
- Be careful during your first course in this wheelchair.
- Meet your wheelchair.
- Make use of your wheelchair on level ground as well as uphill and downhill.
- Do not attempt to reach objects if you need to lift them from the floor reaching between your knees.
- Do not attempt to reach objects if you need to move on to the seat.
- Do not use on roads or motorways.
- Do not attempt to climb or descend with water, ice, or oil membrane.
- Do not use unauthorized parts, accessories, or adapters other than those approved by MOBIAK.
- Do not stand in the wheelchair.

BEFORE USE:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check, if the product is properly assembled.
3. Check If the seat is completely open. If not push the seat down from the lateral frame.
4. Check that the brake works correctly.
5. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.
6. If there are quick release axes, check that they have been secured correctly.
7. Check tires, tire pressures (if needed) and wheel locks (brakes). The brakes will not work properly if the pressure is too low or the gap between the tire and the wheel locks is too long.

WARNING!

The following methods must be used only from experienced patients. If you are not experienced, ask for help from an attendant.

SITTING DOWN TO THE WHEELCHAIR:

1. Position the wheelchair as close as possible to you.
2. Engage wheel locks.
3. Lean on the frame with arms and slowly go down towards the seat.

STANDING UP FROM THE WHEELCHAIR:

1. Position the wheelchair as close as possible alongside the seat to which you are transferring.
2. Engage wheel locks.
3. Must rely on both handles equally. Supporting only one hand (and unattended) can cause injury.
4. Shift body weight into the seat that you are going to transfer.

OTHER INSTRUCTIONS:

1. Always use lifts and ramps. If these are not available, you can reach your destination with the help of two escorts. They must lift the wheelchair by holding stable parts of the frame. A wheelchair should never be lifted while a passenger is seated. It should be pushed only.
2. Avoid stairs or obstacles, otherwise ask help from your attendant:
3. This wheelchair must be used solely to carry a person at a time on the seat.
4. Users should not sit in wheelchair while are using any means of transport.
5. The wheelchairs have a brake on the back wheels, which should be secured when the wheelchair is not moving.
6. DO NOT push on the footrests to sit or stand up, DANGER OF OVERTHROW. The footrests are used by the user when He or She is seated.
7. DO NOT touch moving parts while moving DANGER OF INJURY

MAINTENANCE:

Periodic review of the procedures below will help to maintain the life and effectiveness of your wheelchair.

WEEKLY

1. Check the wheel pressure by following the instructions on the wheels.
2. Check that the central bolt of the transverse arm is tight and in good condition.
3. Clean the fabric with soapy water or sufficient products.
4. Clean the metal parts of the chair with a dry cloth or with special products for chrome surfaces or painted surfaces.
5. Check the wheel bolts and adjust if necessary.
6. Check for damage to front and rear wheels.

QUARTERLY

1. Check and adjust if necessary: wheel forks, pulling wheels, propellers.
2. Grease or oil all components that are subject to friction or impending wear.
3. Tighten the screws and nuts.
4. General revision of the chair's status and repair to whatever is necessary.

ANNUALLY

1. We recommend a full review once a year to a qualified service provider.
2. For the safety of the patient, we recommend that the wheelchair will be checked by the manufacturer every 2 years. In case of repair, use only authentic spare parts.

CLEANING:

In addition to maintenance and inspection, both the frame and the upholstery can be thoroughly cleaned.

- To maintain its appearance, the fabric must be cleaned regularly.
- Dirt can be removed with a sponge and a mild detergent with warm water, sweeping in the direction of texture.
- Rinse with clean water before letting it dry.
- **DO NOT leave the wheelchair wet.**
- Do not place the fabric in a tumble dryer. If the dirt is deep, the surface should be cleaned with a soft brush.

- The frame should be cleaned regularly with a mild detergent.
- The wheelchairs with leather seats must be cleaned with dry cloth.
- Clean the wheels with warm water and mild soap. Pay attention in drying the wheels. it is dangerous use the wheelchair with wet wheels.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels or brakes.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

WARRANTY CLAIM FORM

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE::	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
ADDRESS:			
PHONE:			
			SIGN AND SIGNATURE

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The wheelchair that you have in your hands may differ from the one in the following photos. The assembly process is similar.

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First of all we check the contents of the package. Place the wheelchair as shown in the picture.

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By pushing the bars on the side of the seat, with one movement, the Wheelchair opens. Confirm that the cross bars are in Correct position.

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We move the black bar where it is already located on one back bar and drive its other end to the hole on the opposite parallel bar. This bar must end up parallel to the ground. Its holes must be intersected with the bar holes on the opposite side. Then we drive the screw fitting with the plastic head to the holes in such a way that it penetrates all three intersecting holes. Inside there are guides where by turning the bolt it reaches to the rear and secures. In case of disassembly, we do exactly the opposite.

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Hold the footrest with your hand and place it in the way shown. The pin has to pass through its hole.

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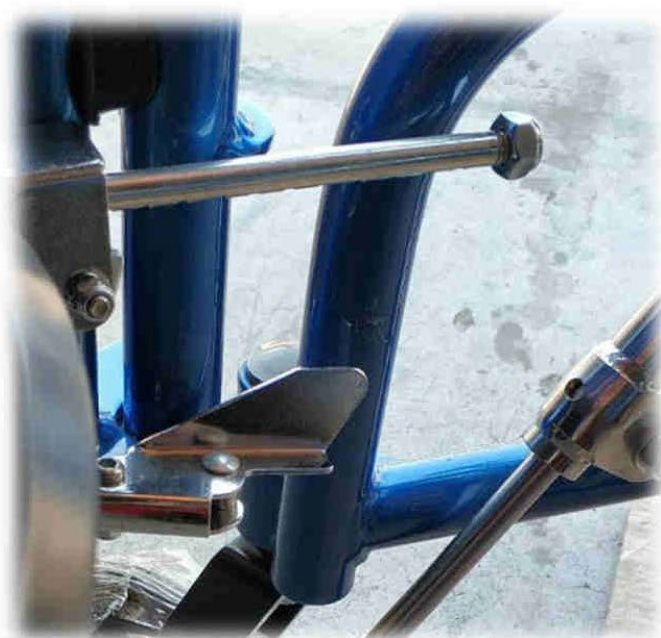
Holding the footrest firmly, place it so that the parallel with the hole part touches and touches the vertical tube where it ends up on the front wheel.

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Once the curved section of the bottom is tangential to the vertical tube, rotate the footrest counterclockwise until the lock switch is closed and locked.

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Apply the back piece in such a way that the pocket on the back of it is in the right direction as well as the safety clips to hold the parallel bars on the back where it is attached. If necessary, we can slide the back up and down along the parallel bars of the back for a more comfortable fit.

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The screws on the back of the back part must be loosened and the safety hooks must be positioned so that their curve hugs the bars on the back. As soon as we are sure of this, we rotate them until they are tightened enough.

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We repeat exactly the same procedure for fitting the seat fitting. Pay particular attention to the knee spacer bolt. It must be on the front side.

The seat is locked in the same way as the backrest.

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After confirming seat seating, we guide the leg separator to the metal hole we mentioned earlier. As soon as it reaches the desired depth, secure it by tightening the bolt.

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Continuing the fitting of the rest of the restraints, place the armrests right and left of the seat. Then place the "T" fitting where it holds the head pad. The "T" is inserted into the upper part of the back of the back. There is also the tightening bolt and height adjustment bolt. The head pad is adjustable from a tilting side by means of a screw nut system where it restricts or releases a rotary ball where it regulates its inclination. Once we have the desired tilt, tighten the lock nut.

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The wheelchair includes shoulder supports where they prevent the user from sliding right and left and restrict it to safety. These are slid on the back of the wheelchair until the holes of their metal parts meet the holes in the back of the back. Then we pass the screws where they are included and lie on the back, adjust the width of the user's chest, and then tighten them

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From the 4 points shown in the photo we pass the edges of the safety belt. After that we are sure about the comfort and safety of the user, we make sure we use Velcro edges. The third photo shows the final shape of the belt as it should be. The correct application of the belt is very important for the correct user position.

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It is possible to adjust the gastricnemius muscle pad where it is in the middle of the footrest. This is because, depending on the height of the tibia of the user, the most appropriate application is required. The second photo shows this application with three different levels of tibia height for greater comfort and safety.

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On the front of the rear wheels there is the wheel locker of the wheelchair. With a forward motion, the metallic part of the brake holds and immobilizes the wheel.

With the rearward movement, the wheel is unlocked and the wheelchair can move again.

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