**ALIUMININĖ VAIKŠTYNĖ SU 4 RATUKAIS  
ALUMĪNIJA STAIGULIS AR 4 RITENTIŅIEM  
ALUMINIUMIST 4 RATTALINE RULLAATOR  
ALUMINIUM 4 WHEEL ROLLATOR  
ROLATOR 4 ROTI ALUMINIU  
ČTYŘKOLOVÉ HLINÍKOVÉ CHODÍTKO**

Naudojimo ir priežiūros instrukcija  
Lietošanas un kopšanas instrukcija  
Kasutusjuhend  
User manual  
Manual de utilizare  
Návod k obsluze

**FARGO**

**01-3520**

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**EN****PREFACE**

- Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition, We wish you much success and we hope Kid-Man product will make a positive impact to your daily living.

**INDICATIONS**


- Walking device for indoor and outdoor use perfect for compensation and improvement respectively of restricted ability of walking.


**GENERAL NOTES**

1. Name of products
2. Model
3. Maximum load
4. Manufacturer
5. Contact details
6. Serial number



## CONTENT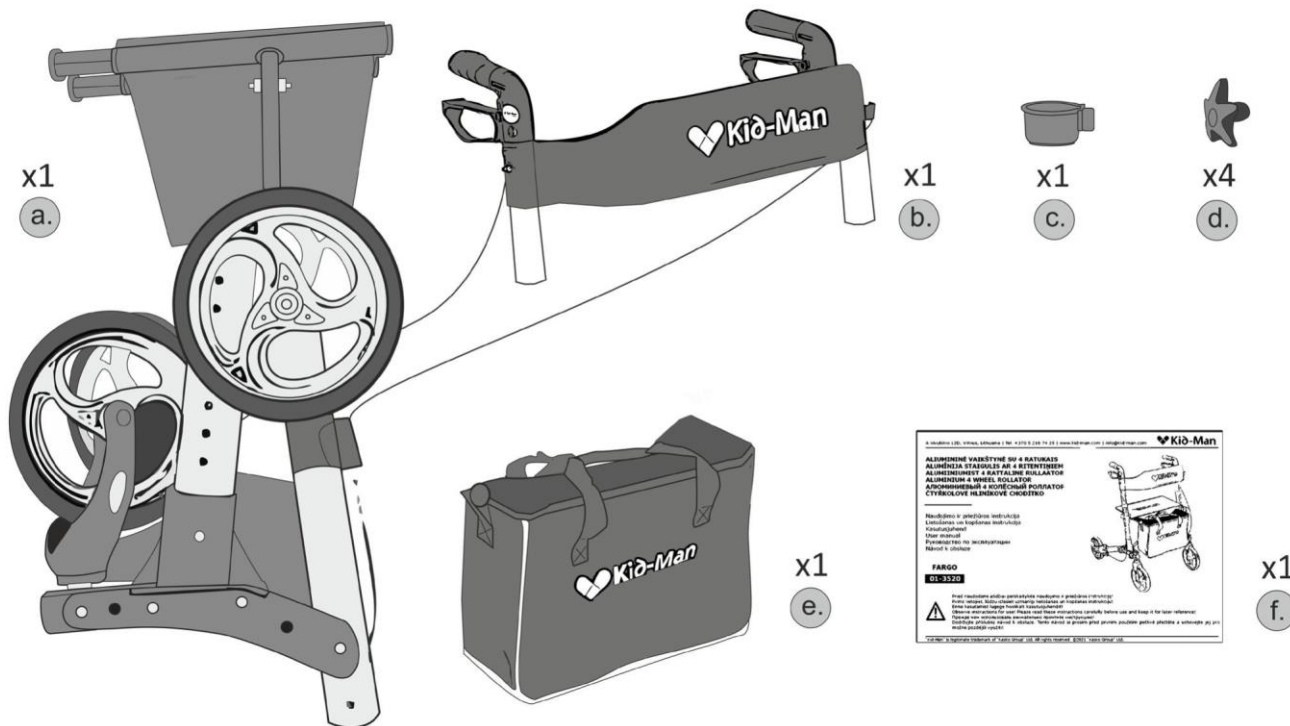


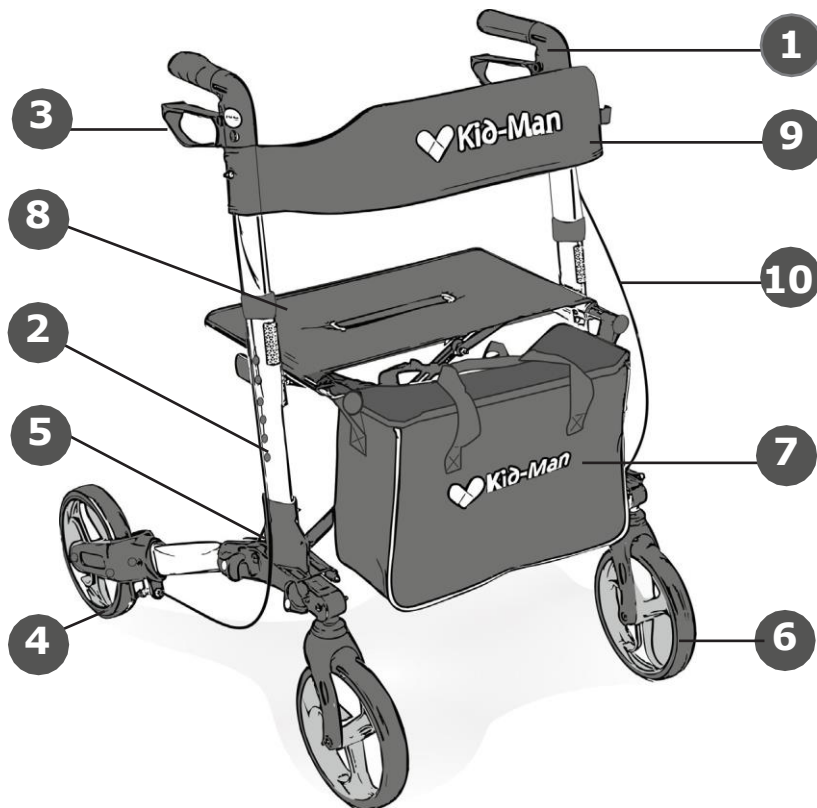**PARTS OF THE PRODUCT**

- 1.** Handle
- 2.** Adjustable tube of handgrips
- 3.** Brake
- 4.** Rear wheel
- 5.** Frame
- 6.** Front wheel
- 7.** Textile handbag
- 8.** Seat
- 9.** Backrest
- 10.** Brake wire


**SET UP**

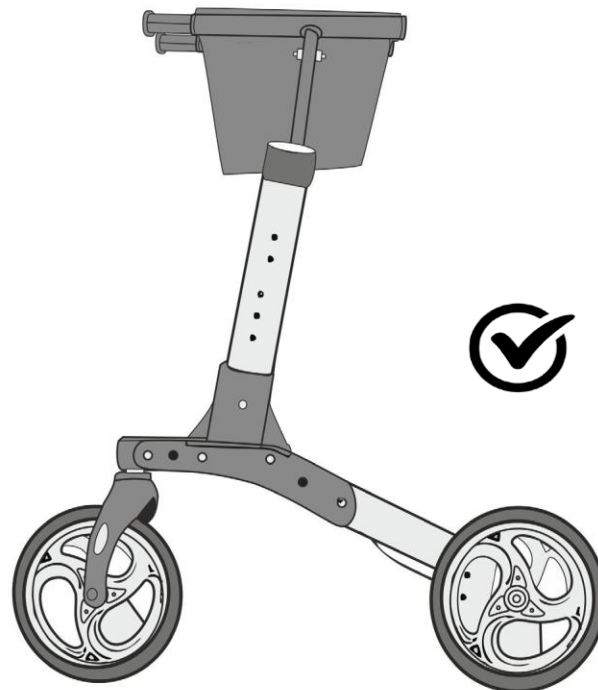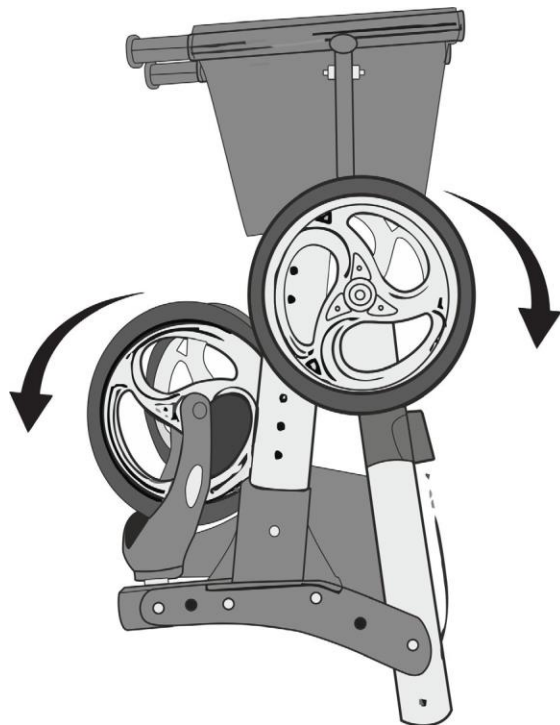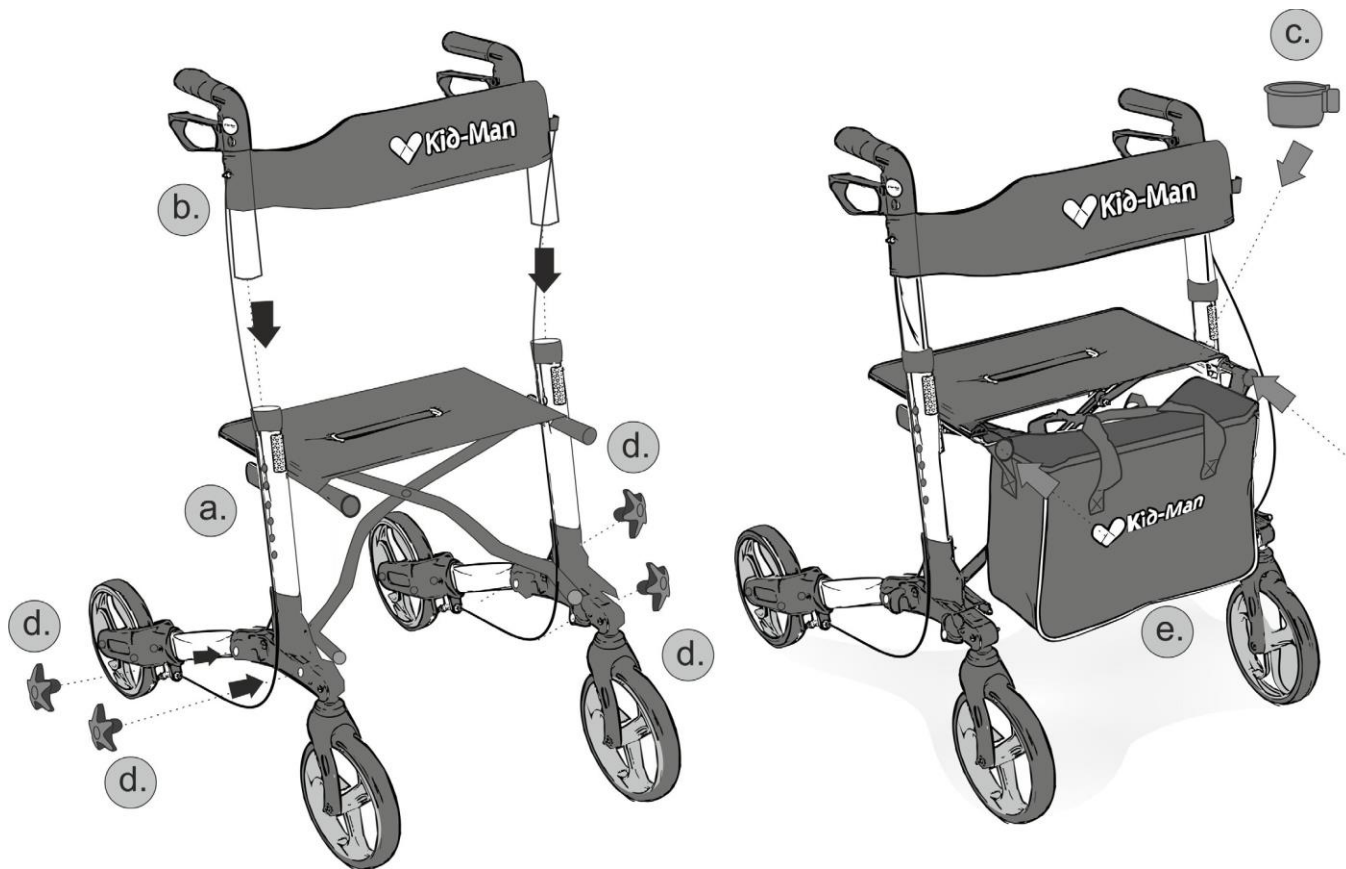
## PARKING AND BRAKING

- The efficiency of the brakes can be affected by abrasion of the wheels. Therefore test regularly the brakes. For desired braking strength adjust the brake either manually at the screw on the housing of the brake or with the suitable tool (not included) on the brake wire.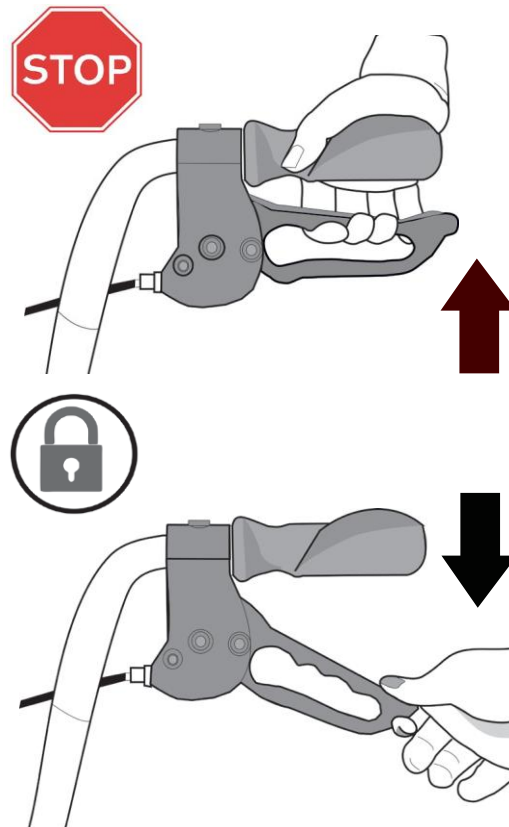

**FOLD AND UNFOLD THE ROLLATOR**

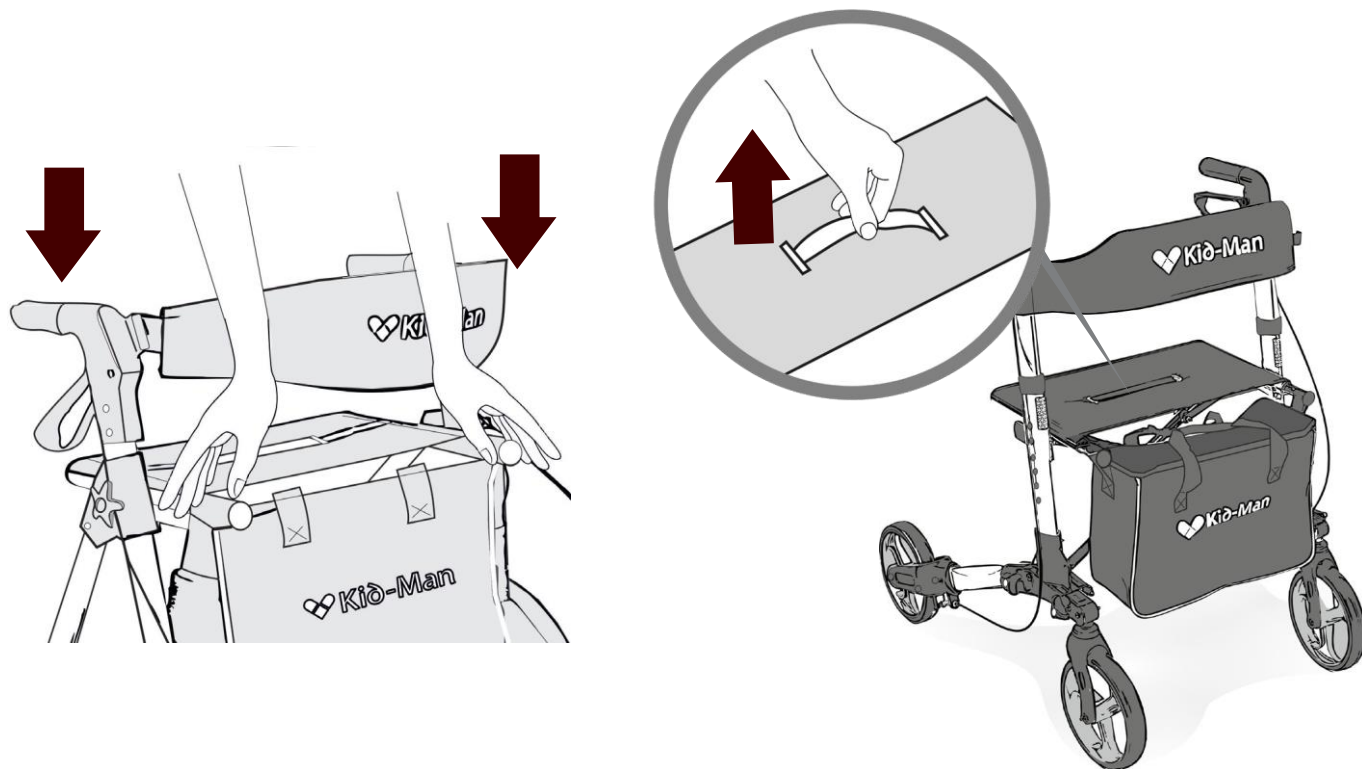
## WALKING

- When you walk with your rollator make sure you walk upright, with the rollator near you body. It will give you the correct body posture.

## HEIGHT ADJUSTMENT

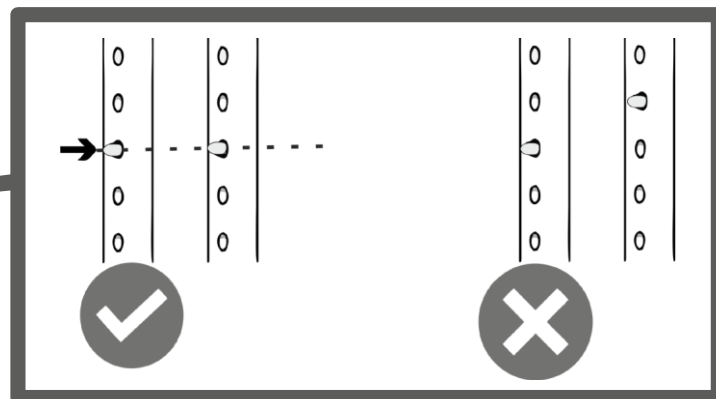
## SITTING

- Make sure the rollator is on the parking brake set before you sit on the seat of your rollator. When you sit down please make sure that your feet are between the rear wheels.

## STEPPER

- When you want to lift the rollator, use the stepper mounted next to rear wheels. Firstly, place your foot on the stepper. By pushing the stepper down, the front wheels are lifted thus allowing using to cross vertical obstacle. Then using the rollator normally, please keep you feet far from stepper.

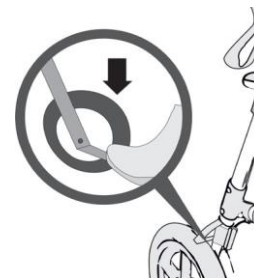## SAFETY WARNINGS

- The walker is only suitable for single occupancy.

- Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.

- Do not attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm to you or damage to the walker.

- It is forbidden to smoke or to be close to the flames, using a walker.

- The walker is not intended for use by the blind.

- Use slow speeds on gradients. Do not exceed the maximum gradient stated. Do not use escalators. Do not reverse down a gradient.

- It is forbidden to exceed the maximum safe load of product.

## CARE AND MAINTENANCE

- The product should be stored in a dry environment, away from direct sunlight. Should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent.

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**EN** We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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© Vivulskio 12D, Vilnius, Lithuania | Tel. +370 5 216 74 25 | www.kid-man.com | info@kid-man.com