MULTILINGUAL
MANUALS

GR EN

MOBIAKARE[®]
MEDICAL EQUIPMENT

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INTENDED USE:

These wheelchairs have been designed for physically or mentally handicapped people with limited mobility. It makes possible to transport people with mobility problems or limited mobility and can be used for movement by the patient himself (if it has 20-24" rear wheels) or with the help of an attendant.

When the wheelchair has a rear wheel with a diameter of 50 to 60 cm and hand rim on the wheel then the user can move independently

When the wheelchair has a rear wheel with a diameter of 13 to 40 cm and does not have a hand rim on the wheel then the user needs an attendant to move the Wheelchair through the handles located on the back of the Wheelchair.

SPECIFICATIONS

Please refer on **ANNEX A**

DIMENSIONS

Please refer on **ANNEX B**

ASSEMBLY

Please refer on **ANNEX C**

SYMBOLS

Please refer on **ANNEX D**

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the wheelchair by any removable parts.
- Sitting down in and standing up from the wheelchair may be performed independently, provided you have an adequate mobility and upper body strength.

IMPORTANT!

The construction of this wheelchair is designed to provide maximum safety. However, users may be at risk by using their wheelchairs unsuitable. For your own safety, the following rules must be observed. Nonspecific or incorrect adjustment work can increase the risk of accidents.

- Always wear light-colored clothes when moving in dark places so that you can be reached more easily from others.
- Be careful during your first course in this wheelchair.
- Meet your wheelchair.
- Make use of your wheelchair on level ground as well as uphill and downhill.
- Do not attempt to reach objects if you need to lift them from the floor reaching between your knees.
- Do not attempt to reach objects if you need to move on to the seat.
- Do not use on roads or motorways.
- Do not attempt to climb or descend with water, ice, or oil membrane.
- Do not use unauthorized parts, accessories, or adapters other than those approved by MOBIAK.
- Do not stand in the wheelchair.

BEFORE USE:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check, if the product is properly assembled.
3. Check If the seat is completely open. If not push the seat down from the lateral frame.
4. Check that the brake works correctly.
5. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.
6. If there are quick release axes, check that they have been secured correctly.
7. Check tires, tire pressures (if needed) and wheel locks (brakes). The brakes will not work properly if the pressure is too low or the gap between the tire and the wheel locks is too long.

WARNING!

The following methods must be used only from experienced patients. If you are not experienced, ask for help from an attendant.

SITTING DOWN TO THE WHEELCHAIR:

1. Position the wheelchair as close as possible to you.
2. Engage wheel locks.
3. Lean on the frame with arms and slowly go down towards the seat.

STANDING UP FROM THE WHEELCHAIR:

1. Position the wheelchair as close as possible alongside the seat to which you are transferring.
2. Engage wheel locks.
3. Must rely on both handles equally. Supporting only one hand (and unattended) can cause injury.
4. Shift body weight into the seat that you are going to transfer.

OTHER INSTRUCTIONS:

1. Always use lifts and ramps. If these are not available, you can reach your destination with the help of two escorts. They must lift the wheelchair by holding stable parts of the frame. A wheelchair should never be lifted while a passenger is seated. It should be pushed only.
2. Avoid stairs or obstacles, otherwise ask help from your attendant:
3. This wheelchair must be used solely to carry a person at a time on the seat.
4. Users should not sit in wheelchair while are using any means of transport.
5. The wheelchairs have a brake on the back wheels, which should be secured when the wheelchair is not moving.
6. DO NOT push on the footrests to sit or stand up, DANGER OF OVERTHROW. The footrests are used by the user when He or She is seated.
7. DO NOT touch moving parts while moving DANGER OF INJURY

MAINTENANCE:

Periodic review of the procedures below will help to maintain the life and effectiveness of your wheelchair.

WEEKLY

1. Check the wheel pressure by following the instructions on the wheels.
2. Check that the central bolt of the transverse arm is tight and in good condition.
3. Clean the fabric with soapy water or sufficient products.
4. Clean the metal parts of the chair with a dry cloth or with special products for chrome surfaces or painted surfaces.
5. Check the wheel bolts and adjust if necessary.
6. Check for damage to front and rear wheels.

QUARTERLY

1. Check and adjust if necessary: wheel forks, pulling wheels, propellers.
2. Grease or oil all components that are subject to friction or impending wear.
3. Tighten the screws and nuts.
4. General revision of the chair's status and repair to whatever is necessary.

ANNUALLY

1. We recommend a full review once a year to a qualified service provider.
2. For the safety of the patient, we recommend that the wheelchair will be checked by the manufacturer every 2 years. In case of repair, use only authentic spare parts.

CLEANING:

In addition to maintenance and inspection, both the frame and the upholstery can be thoroughly cleaned.

- To maintain its appearance, the fabric must be cleaned regularly.
- Dirt can be removed with a sponge and a mild detergent with warm water, sweeping in the direction of texture.
- Rinse with clean water before letting it dry.
- **DO NOT leave the wheelchair wet.**
- Do not place the fabric in a tumble dryer. If the dirt is deep, the surface should be cleaned with a soft brush.
- The frame should be cleaned regularly with a mild detergent.
- The wheelchairs with leather seats must be cleaned with dry cloth.
- Clean the wheels with warm water and mild soap. Pay attention in drying the wheels. it is dangerous use the wheelchair with wet wheels.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels or brakes.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

WARRANTY CLAIM FORM

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE::	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
ADDRESS:			
PHONE:			
SIGN AND SIGNATURE			

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

 **MOBIAC S.A**
ΚΑΘΙΑΝΑ ΑΚΡΟΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

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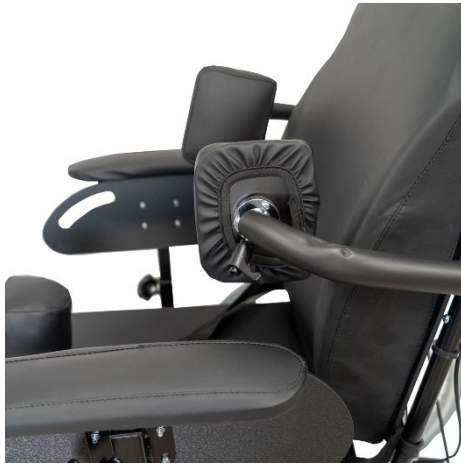
The wheelchair that you have in your hands may differ from the one in the following photos. The assembly process is similar.

Remove the product from the carton and unfold it.

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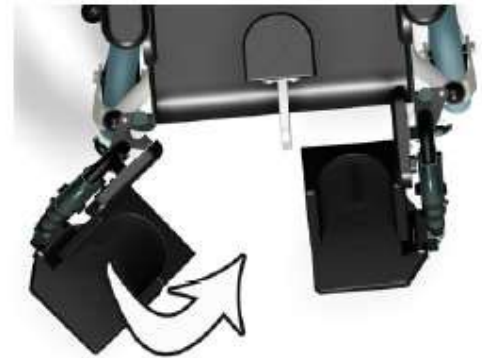
Place the footrests in the specified positions.

Rotate the footrest forward until the release lever locks into place (distinctive click sound).

To remove the footrests, press the release lever. Rotate the footrest back to the side and lift it up.

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Installation and adjustment of side panel

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Apply the brakes to secure the wheelchair.

Unscrew the screw and insert the side panel through the frame guide and adjust to the desired position. Retighten the screw.

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Installation and adjustment of headrest

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Place the headrest into the designated tube on the seatback.

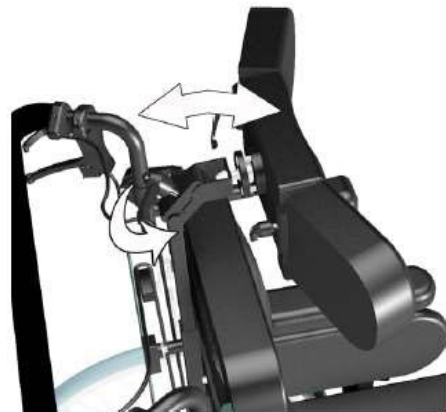
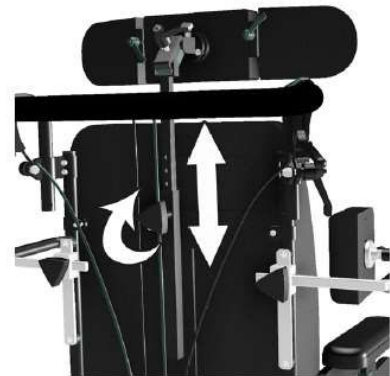
Move the headrest to the desired position. Finally, tighten the screw.

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To adjust the height of the headrest: Loosen the screw. Place the headrest in the designated tube on the back of the seat. Move the headrest to the desired position. Finally tighten the screw again. Move the headrest to the desired height. Finally tighten the screw again.

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To adjust the headrest depth: Loosen the screw. Unscrew the screw, bring the headrest along the bar to the desired depth. Finally tighten the screw again.

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Apply the brakes to secure the wheelchair. Drag the chest supports to the corresponding guide. Finally tighten the screws on both sides tightly.

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Apply the brakes to secure the wheelchair. The backrest cushion can be adapted to backrest with Velcro straps.

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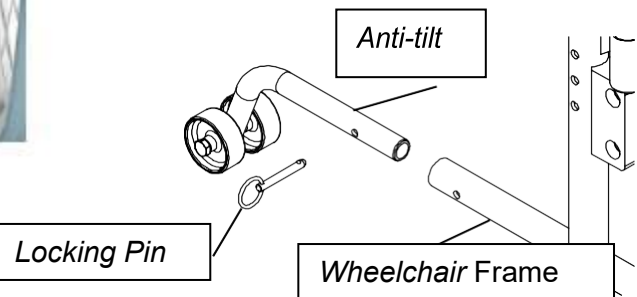
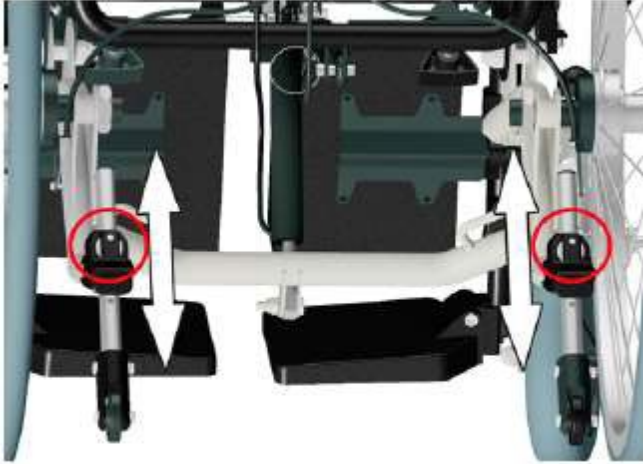
Apply the brakes to secure the wheelchair. The seat cushion can be adapted with Velcro straps.

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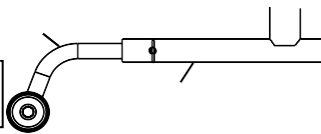
Insert the anti-tilt wheels positioned according to the chart above.

Secure the anti-tilt to the wheelchair frame with locking pins. (or "C" clips).

Ensure that the locking pins (or "C" clips) protrude through both the anti-tippers and the wheelchair frame.

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Always deactivate the anti-tipper before driving over a step or kerb.

Never step on the anti-tilt or use it as a tipping aid.

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Apply the brakes to secure the wheelchair. Release the lever and adjust the footrests up or down until the desired angle is reached. Tighten the lever again.

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Apply the brakes to secure the wheelchair. The knee support pad can be rotated 90°. Release the lever (also used to adjust the desired footrest angle), rotate the knee support pad and tighten the lever again.

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The length of the footrests can be adjusted in 4 positions. Loosen the fixing screw. Bring the footrest to the desired position and retighten the screw.

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The calf pad can be moved downwards or upwards. To do this, rotate the pad out and then move it to the desired position.

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Position adjustment of the calf pad

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You can move the calf pad forward or backward. To do this, loosen the screw and insert it into the desired hole.

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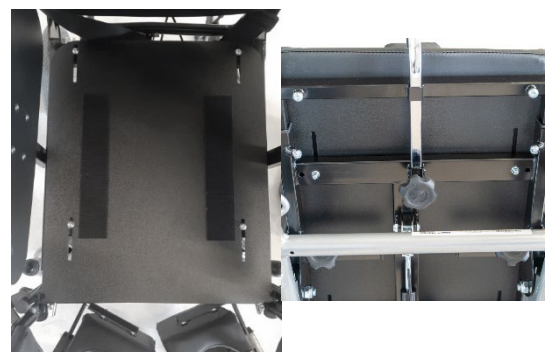
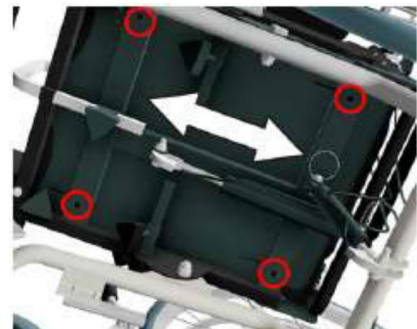
To adjust the depth of the seat you can move the seat section forwards or backwards. Remove the cushion and loosen the 4 seat screws on the bottom, holding against the screws on the top side of the seat. Then also adjust the depth position on the footrests via the rotary adapters, to allow the seat to move forward and backward. Now adjust it to the desired position. Then retighten the screws and rotary adapters.

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To adjust the height of the backrest, you can move the backrest section up or down. To do this, loosen the 4 guide screws after removing the back cushion. Now adjust the backrest to the desired position. Then retighten the screws and adjust the back cushion.

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Adjustment of the height of front wheels

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The height of the front wheels can be adjusted in 3 positions. Remove the bolted connection of the wheel axles. Bring the wheels to the desired position and tighten the screw connection again.

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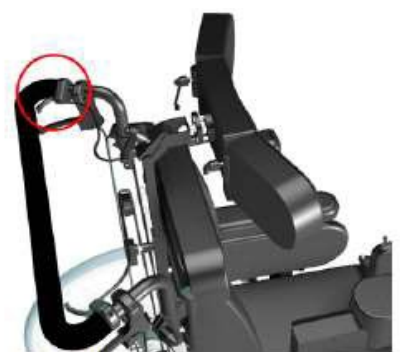
Apply the brakes to secure the wheelchair.

The seat can be tilted forward for easier entry and exit from the wheelchair. Press the small lever on the push handles and push the wheelchair forward. The seat tilts down at its front. To tilt the seat back, press the lever and push the push handle down.

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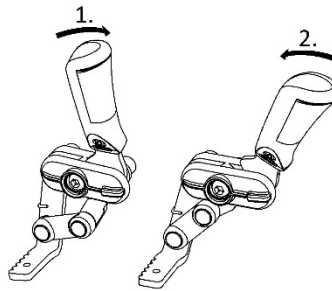
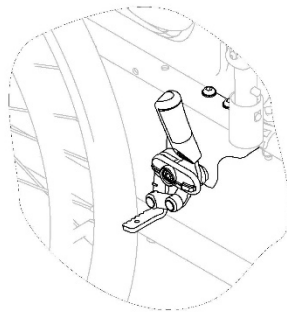
Apply the brakes to secure the wheelchair. The backrest can be reclined from -6° and up to 70° by an attendant. To do this, press the small lever on the push handles and adjust by pressing the backrest to the desired position.

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The brakes are used to immobilize the wheelchair when it is stationary to prevent it from rolling away.

To engage the brake, push the brake lever forwards with the palm. (1)

To disengage the brake, pull the brake lever backwards with the low fingers (2)

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Never engage the parking brakes while you are moving.

Do not engage the parking brakes to slow down the wheelchair.

Do not lean on the parking brakes for support or transfer.

Keep your fingers away from movable parts when using the brake, always keep your hand on the brake lever.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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