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INTENDED USE:

The Bed 0805424 is used for patients who have health problems and they need to stay lying in bed. They are designed for people who need to be treated for health reasons or disability in nursing homes, hospitals or even in healthcare facilities.

It was also developed as a supporting solution for the home care.





CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts

PICTURES OF CORRECT AND WRONG USE OF THE PRODUCT:

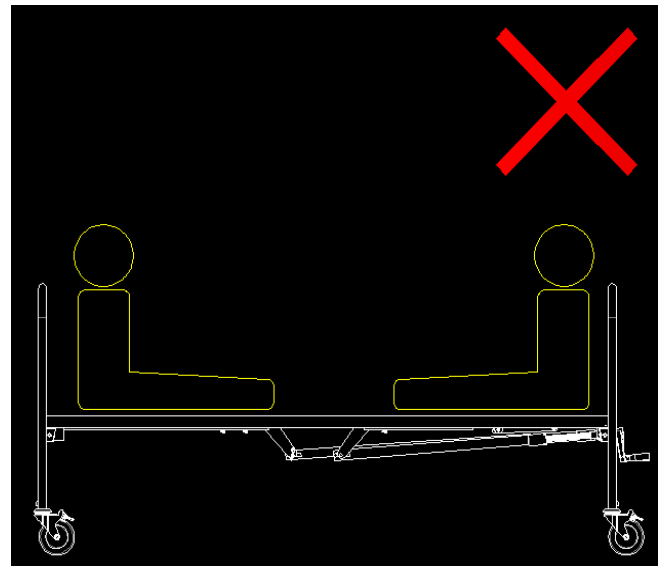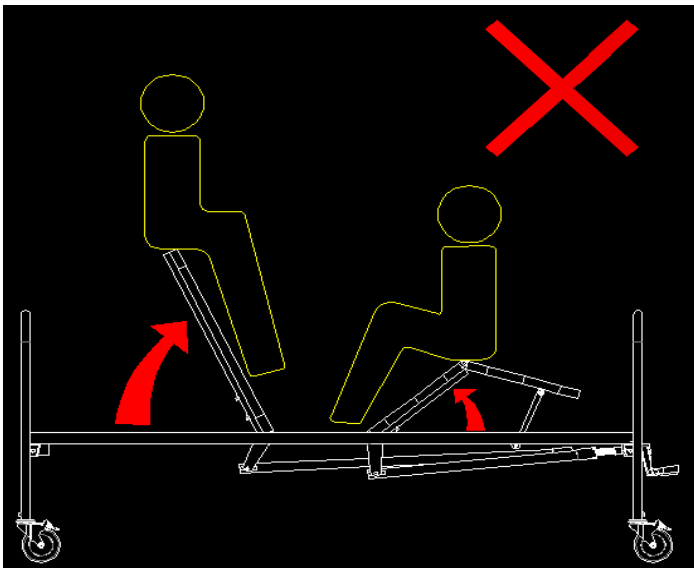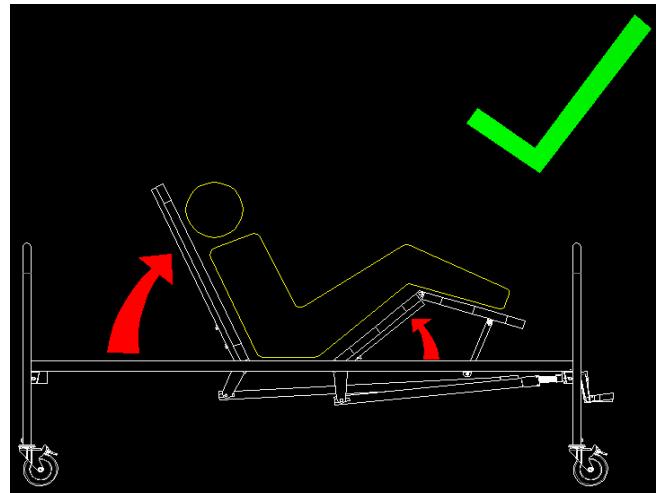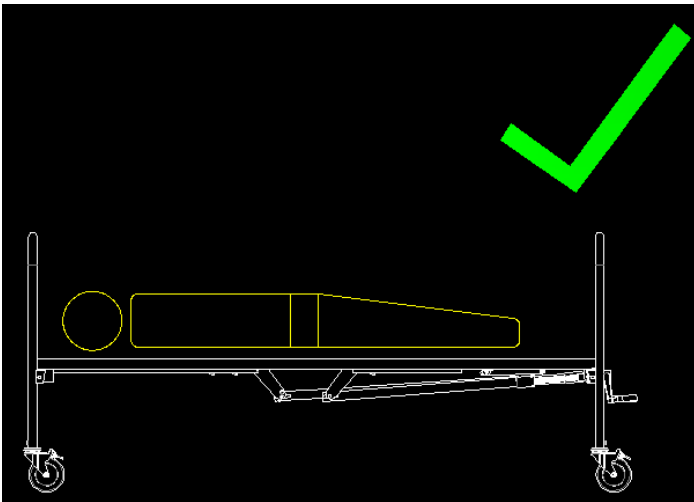DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity for any reason.
- When lowering the head or foot section, make sure that nobody inserts hands, legs or other body parts under the bed base
- Make sure that the floor surface supporting the bed is flat.

BEFORE USE:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
2. In case of damage do not use the product and contact your dealer for further instructions.
3. Check, if the product is properly assembled and all screws are secure and well screwed on.
4. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

MATTRESS SELECTION

Use mattresses or anti-decubitus mattresses according to the prescription of the treating Physician. The size of the mattress should not be larger than the frame of the bed. All MOBIAK mattresses and anti-decubitus are suitable.



ASSEMBLING:

NOTE: The illustrations for the assembly are indicative to make the assembly easier. May will be differences from reality. Assembly tools are included in the package.

PACKAGE CONTENTS:

	DESCRIPTION	QTY
1	Main Bed Frame	1
2	Crank for Backrest, (part is already assembled on the bed)	1
3	Crank for Legrest (part is already assembled on the bed)	1
4	Plastic Side Boards (Headrest & Footrest)	2
5	The Base with the Wheels	2
6	Foldable Rails, pair	1
7	I.V Stand	1
SCREWS KIT		
A	Screws for Installing the Rails	4
B	Pins (Long) & split pins for fixing the Crank on the bed frame.	2
C	Pins (Short) & split pins for fixing the levers of the back section, in the Bed Frame.	2


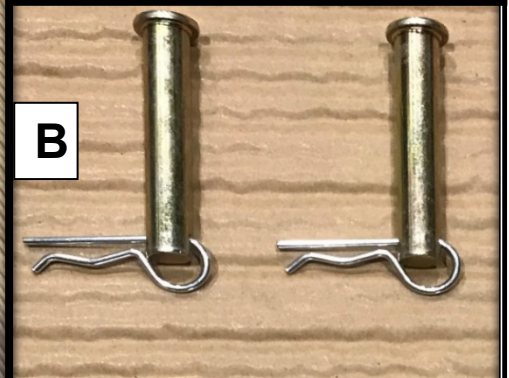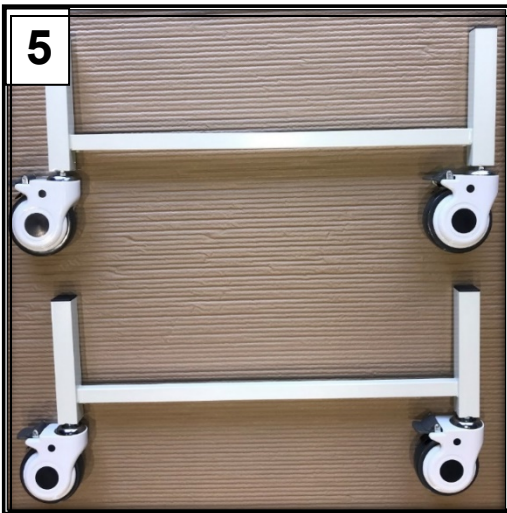
FOLLOW THE BELLOW STEPS:

6. Open the package and remove all parts of the product, except the main frame.
7. Check that the content Complies with the above content list.
8. Without removing the Main Frame from the carton, place the Base with the Wheels "5" in the frame slots.
9. Push down, so the part will fit in the frame slots.
10. Screw well with the tools that are in the package.

As the following picture show: # 1 to # 4

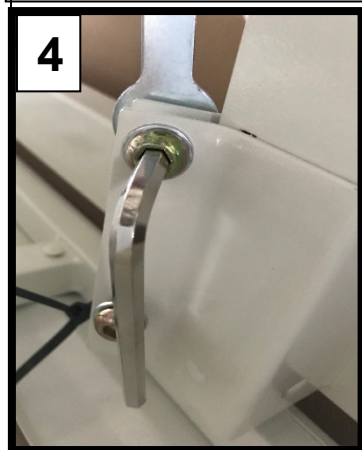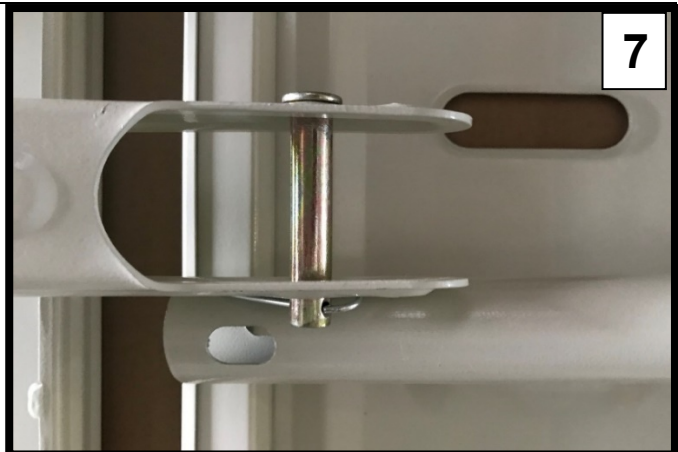FOLLOW THE BELLOW STEPS:

6. Install the short pins "C" to join the levers on the backrest part and secure. **As the pictures # 5 to # 6 show**

7. Attach the crank for the Back-rest using the "B" pins, **As the pictures # 7 to # 8 show**

8. Attach the crank for the Leg-rest using the "B" pins, **As the pictures # 9 to # 10 show.**

FOLLOW THE BELLOW STEPS:




9. Without removing the tire-up take the bed out of the carton and turn it to the right side.

10. Remove the tire-up and rotate the crank handles "2" & "3" clockwise to raise the sections slightly so that the rails "6" care more easily installed.

11. Install the rails "6" at the desired height position, with the safety handle to be on the side of the legs.

12. Screw and secure the rails with the "A" screws with the tools included in the package.

As the pictures #11 to #15 shows

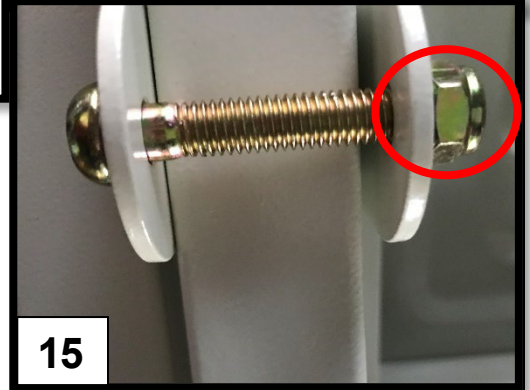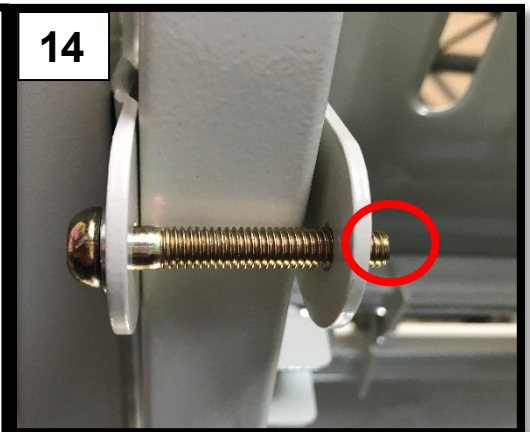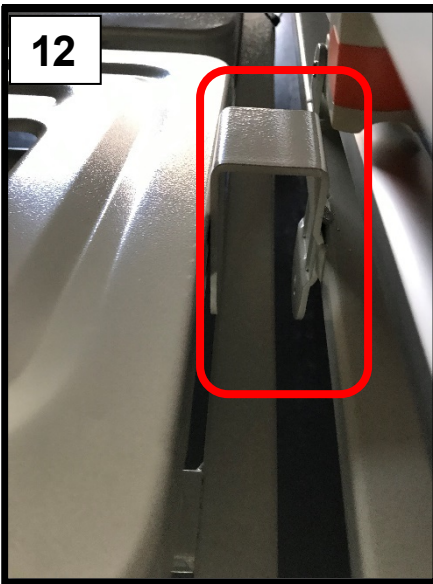FOLLOW THE BELLOW STEPS:

13. Install the Plastic Side Boards (headboard and footrest) "4" in the special iron sockets of the bed..

14. Press lightly down to secure.

15. Put the safety lock.

As the pictures #16 to #19 shows

The Bed is assembled and ready for use.



DISASSEMBLY:

Follow the steps described above in reverse order.

Brake activation: push the brake activation lever down with your foot.


Brake deactivation: push the brake deactivation lever down with your foot.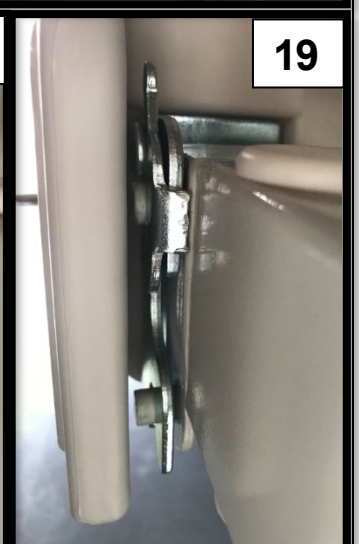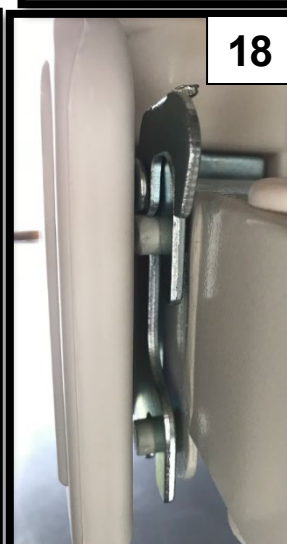

USE OF CRANK:

- v. Pull the handle out.
- vi. Rotate the handle clockwise to raise the section and counterclockwise to lower it
- vii. Tap the Handle inward after adjusting the slop to avoid any injuries.
- viii. The left crank adjusts the inclination in the back-rest and the right crank the leg-rest


As the pictures #20 to #23 shows.

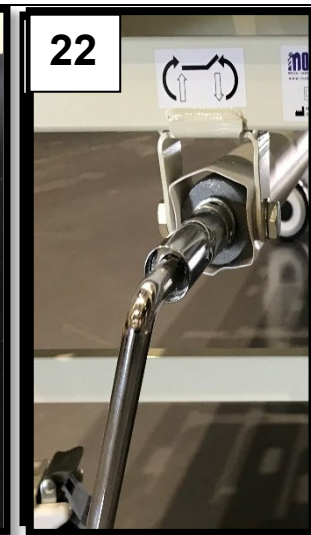I.V STAND:

Insert the IV. Stand “7” that are in the package, into the special slots on the edge of the bed.

As the pictures #24 to #25 shows.



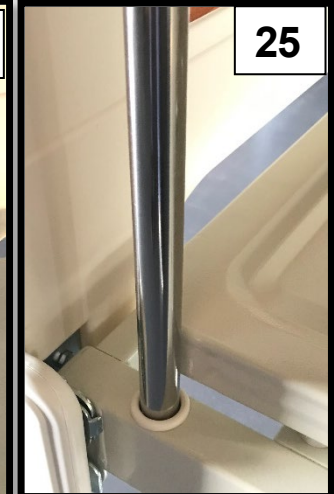
To place the Monkey Pole with plastic handle on the bed*:**

5. You must separately purchase the Monkey Pole 0805423 and Base 0805432.
6. Assemble the Monkey Pole 0805423 that comes in three parts.
7. Screw the Base 0805432 on the special bed socket that are on right or left.
8. Insert the Monkey pole 0805423 into the notched tube to stabilize it.

***** We can install the Monkey pole base 0805432 on beds with production date from March 15, 2021 and after.**

As the pictures #26 to #31 shows.

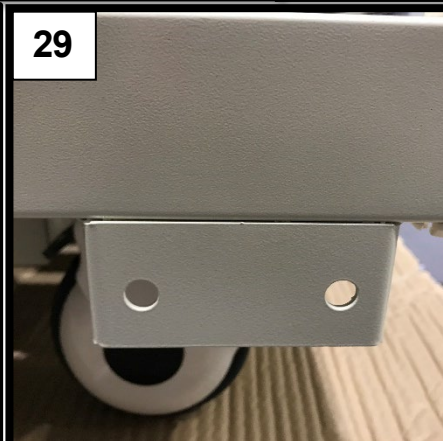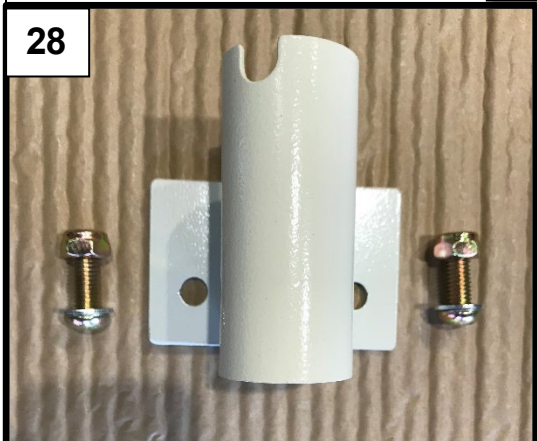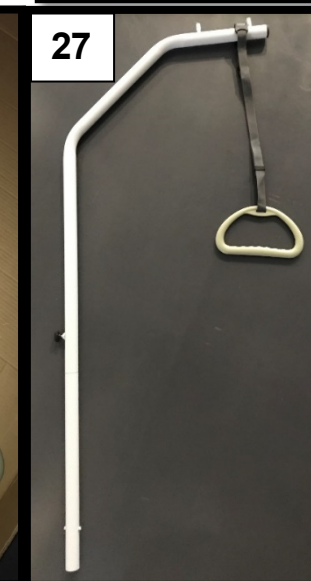CLEANING AND DISINFECTION:

Use only a wet cloth and mild soap on surfaces where they need cleaning.

Then dry thoroughly before you use the product.

DO NOT leave the product wet.

Do not use chemical cleaners to clean the frame. This could cause damage to the surface.

If you need to disinfect the product, use a common, mild disinfectant.

Protect the item from scratches, cuts and punctures.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, brakes, wood, or plastic parts.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user-customer or by dealer.

WARRANTY FORM:

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
DDRESS:			
PHONE:			
			SIGN AND SIGNATURE

MOBIAC S.A.
ΚΑΘΙΑΝΑ ΑΚΡΩΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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