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Table of contents

1. INTRODUCTION	19
<u>1.1. CONTACT</u>	19
<u>1.2. WHAT THE INSTRUCTION MANUAL IS ABOUT?</u>	20
<u>1.3. APPLICATION</u>	20
<u>1.4. CONTRAINDICATIONS</u>	21
2. WARNINGS AND CAUTIONS	21
<u>2.1. WARNINGS AND CAUTIONS</u>	21
3. TRANSPORT AND STORAGE	23
4. SYMBOLS AND MARKINGS	24
5. COMPONENT PARTS	25
6. PREPARATION FOR USE AND USE	26
<u>6.1. PREPARATION FOR USE AND CHECKING THE BEDSIDE CABINET</u>	26
<u>6.2. FIRST USE</u>	26
<u>6.3. USE</u>	26

<u>6.4. TABLE TOP MECHANISM</u>	27
<u>6.5. BRAKE SYSTEM</u>	29
7. TRAINING	29
8. PREPARING TO THE TRANSPORT	29
9. DECONTAMINATION	29
10. MAINTENANCE AND INSPECTIONS	31
<u>10.1. GENERAL INSPECTION</u>	31
<u>10.2. SERVICE LIFE</u>	32
<u>10.3. FAULT FINDING</u>	32
11. DISPOSAL OF PARTS	33
12. SPECIFICATION	33
<u>12.1. TECHNICAL DATA OF THE BEDSIDE CABINET</u>	33
13. WARRANTY	34

1. INTRODUCTION

Thank you for your trust and purchase of our product. Before using the bedside cabinet, please read this instruction manual carefully and make sure it is fully understood. In the case of any doubts concerning the installation, use or maintenance of the cabinet, please contact the seller or the manufacturer.

1.1. CONTACT

In order to get help with the installation, use or service of the product, as well as to report an unexpected operation or to obtain any information regarding service, warranty, sales or customer service concerning this product, please contact your dealer, distributor or (in case of doubt) with MOBIAK.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

1.2. WHAT THE INSTRUCTION MANUAL IS ABOUT?

This instruction manual applies to the below bedside cabinets General information

- **0805620 BEDSIDE CABINET, WOODEN (HOSPITAL USE)**
- **0805621 BEDSIDE CABINET, WOODEN WITH PULL-OUT TABLE (HOSPITAL USE)**
- **0805623 BEDSIDE CABINET, WOODEN WITH HEIGHT ADJUSTABLE TABLE (HOSPITAL USE)**

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

1.3. APPLICATION

The bedside cabinet is intended for medical care, designed to be positioned by the patient's bed containing storage shelves, open storage space and drawers where the patient can keep his/her personal effects.

The product is intended to be used in environment conditions of long-term care in medical facilities (e.g. health care places, rehabilitation center), during care provided in a home setting or patient care under medical supervision in a hospital or other medical unit.

The cabinet is designed for adult users. The lower (or upper) age limit is not defined, and the ability to use the cabinet depends on the mental state of the user.

It is the carer's responsibility to determine that the patient is both mentally and physically capable of occupying the bedside cabinet with minimal risk of personal injury.

- A risk assessment must always be performed on the suitability of the patient.
- The user must at least have sufficient control of trunk movements and should not show any balance disorders - risk of affecting the bedside cabinet stability.
- **If there are any doubts to use the product should be consulted with a health care professional (e.g. physiotherapist, doctor). Make sure the product is suitable for your condition or dysfunction.**

1.4. CONTRAINDICATIONS

The contraindications for using the bedside cabinet include:

- Level of mental development that makes it impossible to safely operate the bedside cabinet functions,
- Confusion, agitation or unstable emotional state of the patient,
- If there are any doubts to integrity or functionality of bedside cabinet.

Consider the presence of other contraindications that are specific for the patient of the care environment.

2. WARNINGS AND CAUTIONS

Warnings in this instruction manual indicate potential hazards, disregard of which could lead to injury or death.

Cautions in this instruction manual identify potential hazards, disregard of which could result in damage to the equipment.

2.1. WARNINGS AND CAUTIONS

- **READ THE INSTRUCTION MANUAL CAREFULLY** before use or installation.
- **THE USER IS OBLIGED TO FOLLOW THIS INSTRUCTION.**
- The bedside cabinet is not intended for children. If used by a child, ensure that a risk assessment has been conducted – taking into account the child's proportions and the dimensions of the bedside cabinet and table top.
- The bedside cabinet table top is not suitable for loading more than 15 kg of evenly distributed weight or 5kg load on end of table top (20cm from edge) – in case of any doubt, please contact the local distributor of manufacturer.
- The bedside cabinet should be used in acceptance with its intended purpose.

- The bedside cabinet cannot be used if any parts are missing.
- Before each use of the device, check and lock both wheels.
- Wheels should be locked/unlocked with the use of foot, not with a hand.
- There should not be obstacles (that would make its proper operation difficult) in the place, where the bedside cabinet is used.

- The bedside cabinet and table top (in option) should be adjusted and used on flat, horizontal surfaces – all wheels ought to touch the ground.
- Do not exceed the safe working load of the table top (only in cabinet's version with the table top)!
- When adjusting the tilting table top (in option) make sure that mechanism is moved away from the back board of the bedside cabinet.
- Leaving fingers between moving parts of the bedside cabinet may injured them and cause an accident.
- During adjustment and maintenance, make sure that any parts of the body are not located in the zone of potential risk of injury (movable parts: drawers, table top in option).
- Do not lean on the bedside cabinet!
- Remember that there is a risk of accident or damage to the bedside cabinet if product is repaired on your own!
- **The bedside cabinet is not intended for transporting goods.**
- Inspections, repairs and disinfections may only be carried out by specially trained persons.
- The use of accessories that have not been designed for use with the bedside cabinet is forbidden - a hazard could be introduced due to product combination incompatibility.
- The use of additional mechanical or electrical devices that are not intended for use with the bedside cabinet is unacceptable.
- Due to the small space under the top board of bedside cabinet with adjustable tilting table top (in option), special attention should be paid to young children, user fingers could be trapped between the its components and injured or damaged.
- Only original parts supplied by the manufacturer may be used.
- It is forbidden to modify the bedside cabinet without the consent of the manufacturer – this poses a risk of danger.
- Hold parts of the bedside cabinet's during transport!
- During use pay attention to the distance between the bedside cabinet and the walls – risk of product damaging.
- Before each use check the space around the bedside cabinet, to ensure if it allows free use.

- The bedside cabinet should be used and kept away from heat sources and open flames (e. g. cigarettes, electric fire, heaters, etc.) – close proximity could pose a fire hazard.
- The bedside cabinet is to be installed and put in to service in accordance with the information provided in these instructions for use.
- **ALL THE ABOVE-MENTIONED WARNINGS AND CAUTIONS MUST BE STRICTLY OBSERVED.**

Warning

3. TRANSPORT AND STORAGE

Observe the following instructions when transporting and storing the bedside cabinet:

- Always store the bedside cabinet on a flat, level surface.
- Wheel brakes should be locked.
- The bedside cabinet should be protected (e. g. with the use of foil or cardboard package) against the ingress of liquids, dirt, dust, etc.
- Do not store the bedside cabinet on its side.

Environmental conditions:

	Operational conditions	Transport/storage conditions
Ambient temperature:	from +10°C to +40°C	from -20°C to +50°C
Humidity:	30% - 75%	30% - 75%
Atmospheric pressure:	from 800 to 1060 hPa	from 800 to 1060 hPa
Altitude above sea level:	≤ 2000 m	≤ 2000 m

- The bedside cabinet is not intended for transporting the goods. Do not transport the goods on top of bedside table between rooms – due to the risk of injury to the user/caregiver. In order to prevent the risk of cross contamination, when removing the bedside cabinet from its place of use by the end user, make sure that all actions (connected with the bedside cabinet) are carried out with the use of disposable gloves. Next to, properly dispose the glove, unless it can be verified that the bedside cabinet and all accessories have been properly disinfected and cleaned.
- If the bedside cabinet is removed from its place of use by the end user, before handing the cabinet over for storage, make sure that has been cleaned and disinfected – in acceptance with your local infection control rules and/or rules that are specified in this instruction manual (see section 11).

- Avoid exposing the bedside cabinet to direct sunlight – direct sunlight may cause colour fading over time (including the labels).
- Avoid placing the bedside cabinet in a moisture rich environment – a long-term exposure to moisture may have a detrimental effect on parts of the bedside cabinet.
- When using the bedside cabinet's functions, make sure that no furniture or other things (such as a bed) are not an obstacle.

- Make sure that the bedside cabinet is positioned in an appropriate distance from walls/other furniture in order to prevent the damage to the equipment when operating the bedside cabinet.

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5. COMPONENT PARTS

List of components of the bedside cabinet

0805620

1. Top board
2. The drawer
3. Shelf (double-sided)
4. Storage closet
5. Four castors (two castors with brakes)

0805621

1. Top board
2. The drawer (double-sided)
3. Shelf (double-sided)
4. Double Storage closet (double-sided)
5. Four castors (two castors with brakes)
6. Table top

0805623

1. Top board
2. The drawer (double-sided)
3. Shelf (double-sided)
4. Double Storage closet (double-sided)
5. Four castors (two castors with brakes)
6. Adjustable tilting tabletop

*(The photos of the products are indicative)

- During operation works, pay special attention to the risk of hands injury.
- The bedside cabinet cannot be used when there are missing parts.

6. PREPARATION FOR USE AND USE

6.1. PREPARATION FOR USE AND CHECKING THE BEDSIDE CABINET

- Make sure that you have read and fully understood the instruction manual before use of the bedside cabinet.
- Make sure that a risk assessment has been conducted in acceptance with local health and safety regulations in order to protect personnel and users from risks during use.
- The bedside cabinet cannot be used in case any parts are missing.

The bedside cabinet is delivered fully assembled. Before it is put into use carrying out the following checks:

- Have all packaging elements been removed?
- Is the bedside cabinet clear of obstructions?
- Is the bedside cabinet set on the flat and horizontal floor?
- Has a risk assessment been performed on the suitability of the bedside cabinet (and any ancillary equipment) for the users?
- Is the wheel lock working properly?
- Has the bedside cabinet (if necessary) been cleaned and disinfected before use?

It is the carer's responsibility to determine that the patient is both mentally and physically capable of operating the bedside cabinet with minimal risk of personal injury.

If the carer is to operate the bedside cabinet ensure that the occupant is made aware of the action(s) about to take place.

6.2. FIRST USE

Before using the bedside cabinet for the first time, a risk assessment must be performed on the basis of the user's condition. This assessment should include, but it is not limited to:

- Possibility of user entrapment,
- Possibility of interference by young children (and adults),
- Users who lack capacity,
- Unauthorized persons,
- Physical and mental condition of users,
- Housing conditions.

If children, adults with reduced cognitive/learning abilities or (even) pets pose a potential risk of intentional or unintentional adjustment or use of the bedside cabinet, consider its suitability for use during the initial risk assessment of the patient/product.

6.3. USE

Before the first use of the bedside cabinet, make sure that:

- the bedside cabinet has room temperature,
- the bedside cabinet has been cleaned and disinfected (see section 11),
- brakes on the castors have been applied,
- all mechanical functions work properly,
- the bedside cabinet is placed horizontally on a flat surface so all the castors touch the ground,
- objects such as a bed, other furniture or walls are not obstacles and there is enough space.

- Leaving hands or other objects between moving parts of the bedside cabinet may injury/damage them or cause an accident.
- Before each use of the bedside cabinet, check and lock both wheels.
- It is forbidden to start and use the product with defects that may pose a risk to users or other persons.
- Before operating the table top (in option) ensure that the patient will not be injured during unfolding the table top

Ensure the bedside cabinet is positioned an appropriate distance from walls / other furniture to prevent damage or patient injury when operating the bedside cabinet.


6.4. TABLE TOP MECHANISM

Bedside cabinet 0805621 is equipped with a folded table top.

To unfold the table top lift the table top until it latches into place.

To fold the table top unlock simultaneously both latches under table top (see the drawing) and fold table top.

It is recommended to hold the table top while lowering it.

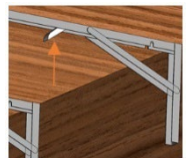Bedside cabinet 0805623 is equipped with an adjustable tilting table top.

To unfold the table top:

- Grasp the handle or upper part of mechanism frame and pull it towards you (in opposite direction from the cabinet).
- Ensure, that hinges on the mechanism back are stable and fully opened.
- Holding the table top handle pull it up parallel to the mechanism frame.
- Gently place it on supports rods positioning the table top horizontally. Ensure, that mechanism frame is unfolded.

The table top height may be adjusted after the table top is pulled out. **The height adjustment** is made by lifting the handle located under the upper part of the frame, attached to the spring to release the strut lock. When lifting the handle the spring will support lifting the table top.

To lower the table top lift the handle to release the gas strut lock and push the mechanism down.

The table top **rotation is adjusted** by unlatching (pulling) the spindle above the table top and set the required position.

To fold the table top:

- Set the table top in horizontal position if it is tilted.
- Then using the strut handle lower the mechanism to the lowest position.
- Pull the mechanism frame towards the bedside cabinet.

It's recommended to fold the mechanism while holding it in upper part of frame with both hands (simultaneously using the spring handle) and press it down.


Warning

Adjusting the table top height and tilting is possible only after unfolding mechanism! Its recommended to unfold table top before, otherwise it could cause hazard of hands injury.

6.5. BRAKE SYSTEM

The bedside cabinet is manoeuvrable via the aid of four wheels (inc. two individually braked wheels), however it is not designed for goods transport.

- To block the brake, press the brake pedal.
- To release the brake, lift the brake pedal.

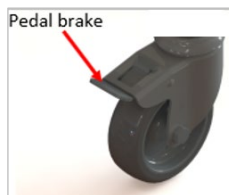For safety reasons, wheels should be blocked with the foot (not with the use of a hand), and the manufacturer recommends wearing adequate footwear. During normal use, both wheels should be locked.

Wheels should be locked / unlocked by foot, not by hand.

7. TRAINING

Professional personnel should be appropriately familiarized with the functionality of the bedside cabinet, its limitations and the target user group before use. The user's ability to adjust the bedside cabinet in an independent manner should be determined in acceptance with the risk assessment. The user should be familiarized with the functionality of the bedside cabinet by a trained person as soon as possible – preferably before the use of this product. It is the responsibility of the end user to ensure they have received sufficient training to use the bedside cabinet safely and correctly.

8. PREPARING TO THE TRANSPORT

Ensure a risk assessment in line with local health and safety policy is undertaken to ensure that staff are not put at risk when preparing the bedside cabinet for the transport.

- Apply the castor brakes.
- Ensure the instructions for use is safely stored.
- Secure the moveable parts (table top, drawers, shelves) with tape, stretch foil or similar to prevent them from unexpected movement.
- Protect the bedside cabinet (e. g. with the use of foil) against the ingress of liquids, dirt, dust, etc.

9. DECONTAMINATION

Infection control and routine cleaning should be carried out in accordance with the local infection control schedule or recommendations from the local regulatory body.

Repeated cleaning and disinfection according to the below instructions will not result in loss of general safety and essential performance.

- Regular cleaning and disinfection of the bedside cabinet will help to prevent the risk of infection to the occupant and/or the carer.
- Before transferring the bedside cabinet and to another user, ensure it has been cleaned and disinfected using the method as detailed below to help prevent the risk of cross infection.

These instructions apply to the bedside cabinet.

General cleaning:

- The bedside cabinet should be cleaned by starting with the cleanest parts of the device and systematically moving to the dirtiest parts. Extra care should be taken around areas where excess dirt or dust may gather (shelves, table top mechanism, etc.).
- The cloth should be changed during the cleaning process if it becomes soiled.
- Wipe down with a clean cloth moistened with a mild detergent and dilute with warm water (40°C).
- Wipe down with a clean cloth moistened with cold clean water, then a clean dry cloth, and allow to fully dry before use.

Disinfection:

- Mop up any fluid with paper towels.
- Wipe bedside cabinet down using a clean cloth moistened with cold clean water.
- Wipe down with a 0.1% Chlorine solution (1,000 ppm) in cold water.
- Rinse with cold clean water and a clean cloth and allow to fully dry before use.
- Always ensure the cleaned parts are allowed to dry.

In cases of blood spills or other bodily fluids it is recommended that a 1% Chlorine solution (10,000 ppm) is used instead.

Note: If any of the stages stated above are omitted or combined it will reduce the effectiveness of the clean.

Note: Bleach, solvents or similar surface cleaners are not recommended due to the risk of damaging the bedside cabinet.

- During decontamination use appropriate protective cover to minimize contact of these measures with the skin. Always check what neutralizing agent is recommended by the manufacturer.
- Decontamination procedure performed by an unauthorized person pose a threat to the person as well as the environment.
- The bedside cabinet manufacturer assumes no responsibility for any loss or damage caused by improperly conducted decontamination.

The use of neat bleach or similar surface cleaners is not recommended as damage may be caused to the cleaned surfaces.

10. MAINTENANCE AND INSPECTIONS

Only authorized service personnel is entitled to repair the bedside cabinet or interfere with its structure. MOBIAK is not responsible for repairs carried out by unauthorized service technicians. Failure to observe the rule may result in the manufacturer's warranty becoming void. Bedside cabinet can be inspected by trained person and service personnel. **All types of bedside cabinet must be serviced at least once a year.** MOBIAK recommends that the caregiver perform frequent visual and operational inspections and its functionality. If there are any signs of damage or the bedside cabinet is not performing as it should withdraw it from service until the cabinet has been repaired and is fit for use again.

Periodically check to ensure that:

- The bedside cabinet operates as per its intended purpose.
- No parts are missing and all fasteners are tightened carefully.
- Parts do not show signs of excessive wear.
- The wooden frame and table top mechanism (in option) is mechanically operational and sound.
- The bedside cabinet is cleaned following the guidelines in this instructions for use.

10.1. GENERAL INSPECTION

The manufacturer recommends that the bedside cabinet are serviced once yearly, as a minimum. Please act accordingly with the following instructions:

- Failure to perform inspections at the stated frequency could adversely affect the basic operation of the bedside cabinet and (consequently) put the patient at risk.
- Modification of the bedside cabinet wooden frame is not

- allowed without the permission of the manufacturer - A hazard could be introduced.
- The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or resulting from non-compliance with the instructions contained in these Instruction For Use.
- Only manufacturer approved components can be used – if in doubt contact the manufacturer or your local distributor.

To maintain the cabinet's essential performance the following checks must be performed:

- Check if all fasteners are tight and that none are missing.
- Check if the drawers and the top table (optional) are mechanically operational.
- Check if all labels and stickers are present and intact.
- Check if the castors lock function works correctly and that when locked castors do not swivel or roll.
- Check if the bedside cabinet does not show signs of excessive wear.
- If any gaps appear to be outside of specification remove the bedside cabinet from use until the dimension of the gap in question has been confirmed.

10.2. SERVICE LIFE

The service life of the bedside cabinet is 10 years*. On the basis that the bedside cabinet is serviced and maintained in acceptance with the information detailed in these instructions for use.

At the end of service life, the bedside cabinet should be withdrawn from use in accordance with local waste management policy.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

10.3. FAULT FINDING

The most common failures that may occur within the service life of the bedside cabinet are described below. If a fault does occur please try the following suggestions, as these may help in diagnosing the fault or contact the service department.

Description of the failure	Possible cause	Remedy
The wheels are permanently locked/unlocked	Damage to wheels	Withdraw from use and contact the service department or supplier
The movable section of table top (in option) do not work correctly	Damage to the table top mechanism	Withdraw from use and contact the service department or supplier

During the adjustment and maintenance attention should be paid to ensure that no part of the body is found in the potentially hazardous section (e.g on the guide rods).

11. DISPOSAL OF PARTS

When the bedside cabinet has come to the end of its useful life follow local recycling policies.

The steel, plastic and wooden components are also to be separated and disposed of following the local recycling policy as these can also be recovered and recycled.

The bedside cabinet is to be decontaminated before disposal to avoid risk of cross contamination.

12. SPECIFICATION

12.1. TECHNICAL DATA OF THE BEDSIDE CABINET

	0805621	0805623	0805620
External dimensions (width × height × depth) [mm]	597x880x487		456x810x456
Width with extended table top (width × height × depth) [mm]	1.130	1.300	-
Upper drawer (width × height × depth) [mm]	350x95x425		320x97x332
Middle shelf (width × height × depth) [mm]	388x233x460		384x192x420
Bottom cabinet (width × height × depth) [mm]	388x385x425		384x355x412
Shelf in the bottom cabinet (width × height × depth) [mm]	388x183x420		384x170x380
Table top height [mm]	885	735-980	-
Table top dimensions (width × depth) [mm]	345x635	340x570	-
Table top tilt	-	+/- 45°	-
Table top maximum load/Maximum load on 20cm of the end of the table top [kg]	15/5	15/5	-
Overall weight [kg]	35,1	40,9	30,2

The bedside cabinet data identify the maximum angles and dimensions which can be achieved in normal use by each part with reference to horizontal.

The above-listed maximum load of table top refer to an equal distribution of load. Applying a greater load than recommended to the end of the table top may cause a fall of cabinet. Additional weight that recommended could damage components or cause the bedside cabinet to become unstable, potentially causing injury.

13. WARRANTY

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, brakes, wood, or plastic parts.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
ADDRESS:			
PHONE:			
			SIGN AND SIGNATURE

MOBIAC S.A
KAGIANA AKPOTHPIOY, XANIA - KPHTH
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com